

Mounting Description

Abduction block for Sliding seat



Tools: awl, 6mm drill & star screwdriver





1. Place the abduction block in center and on top of the seat plate, and find the required position for the locking adapter.

Make marks, using an awl, for the screw holes of the adapter, which is to be mounted on the **UNDERSIDE** of the seat plate.

2. Drill two holes through the seat plate. Fix the adapter on the **UNDERSIDE** mounting the screws through the top of the seat plate using a star screwdriver.

3. The depth of the abduction block can be adjusted by moving the star screws on the underside.

4. Mount the abduction block by sliding the right bar into the adapter. Fix it by tightening the wing screw.

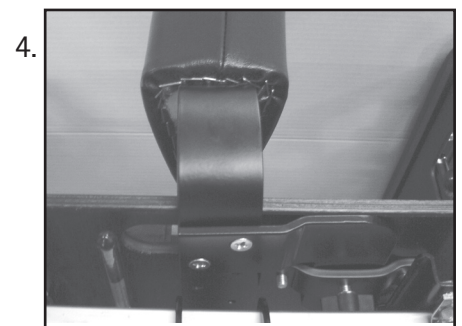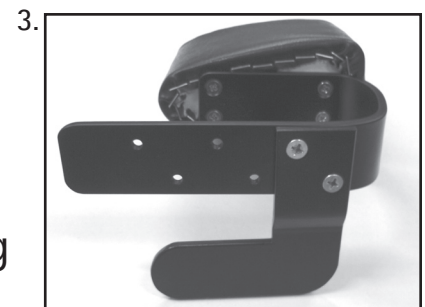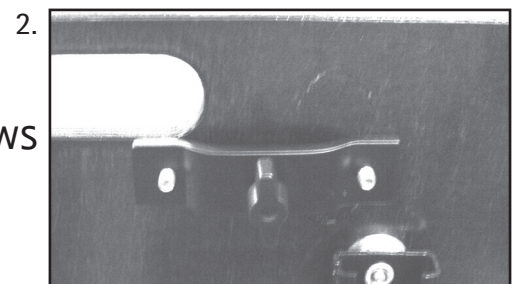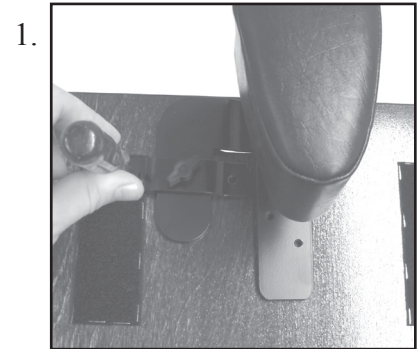

Item no.	Item
22103	Abduction block for sliding seat Small 60
22104	Abduction block for sliding seat Medium 70
22105	Abduction block for sliding seat Large 80