


## Monteringsbeskrivelse Hoftebelte Netti Svipp Mounting description Hip belt Netti Svipp

1

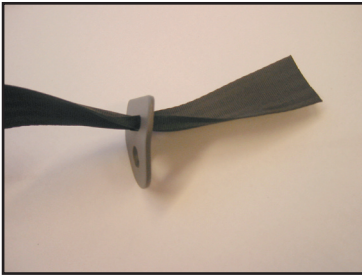

### Norsk

Trekk beltet gjennom hoftebeltefestet (Bilde 1) og tre deretter beltet tilbake gjennom belteklemmen (Bilde 2 og 3).


Plasser foring på innsiden av bakhjulsbraketten (Bilde 4) og tre skruen gjennom hoftebeltefeste og inn i bakhjulsbraketten (Bilde 5). Plasser mutter på innsiden og stram skruen. Pass på at hoftebeltefestet ligger 45° oppover (Bilde 6). Dette gjøres på begge sider.

Juster deretter lengden på beltet ved hjelp av justering i belteklemme (Bilde 3) på hver side.

3



2

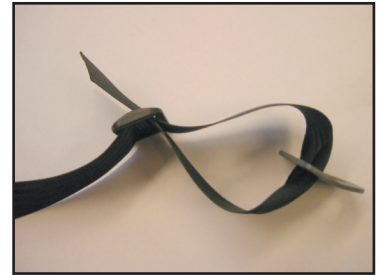

4



5




### English

Tread the belt through the hip belt bracket (Pic. 1) and then tread the belt back through the beltclamp (Pic. 2 and 3)


Place the sleeve on the inside of the main wheel bracket (Pic. 4) and thread the screw through the hip belt bracket and through the main wheel bracket (Pic 5). Put the nut on the inside and fasten good and make sure that the hip belt bracket is in a 45° angle (Pic. 6). Repeat the same operation on the other side.

Adjust the length on the hip belt using the belt clamp (Pic 3) on each side.

7

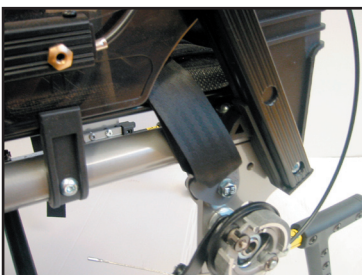

6

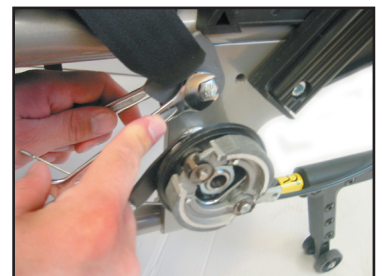

8

