

## Mounting of knee and leg support

When mounting the knee and leg support, please use the following procedure:

1. Put the cushion in the wheelchair.
2. Let the user sit into the chair.
3. Push the knee and leg support(s) in from the side between the seat plate and the seat cushion.
4. Adjust the knee and leg support to a position until finding a position optimal for the user and according to the advice of therapist.
5. When the position is decided, mark the holes on the seat plate. Drill 6.5 mm holes in the seat plate with a distance according to the holes on the holder.
6. The long holes in the holder make it possible to adjust the knee and leg support side ways. For large adjustments, new holes have to be made in the seat plate.
7. A stop- washer shall be used on both ends of the screws.
8. To avoid the screws to tear on the cushion, cover the screws with plastic tape.
9. When moving the user sideways, simply remove the knee and leg support by pulling it straight up. Attach it by fixing it into the same hole.

