

Mounting Description Side Support Correction



Item	Item number
Side support Correction Left	27374
Side Support Correction Right	27375

Tools: 5mm Allen key

Mount the adapter in required height on the backrest tube. The set screws should be on the inside of the tube. Fix both screws.

Sense

The adapter is mounted directly onto the backrest tube.

Netti models

Un-tighten the straps on the velcro backrest & twist the backrest material around the backrest tube to give room for the adapter. Then adjust the straps as normal.




Sense




Netti I, II, III, Svipp, 4U CED

NB! When Side Support Correction is used on the Netti models, we recommend to use the Flip backrest cushion.

Lead the side support into the hole in the adapter, & tighten the adjustment lever. This is also how to adjust the side support in & out of the chair.

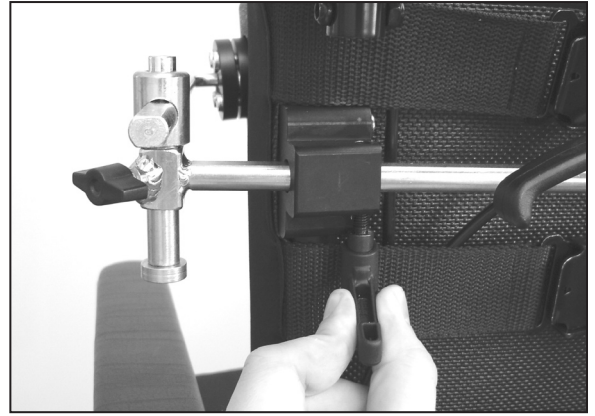

To adjust the side support in the length of the chair, use an Allen Key.

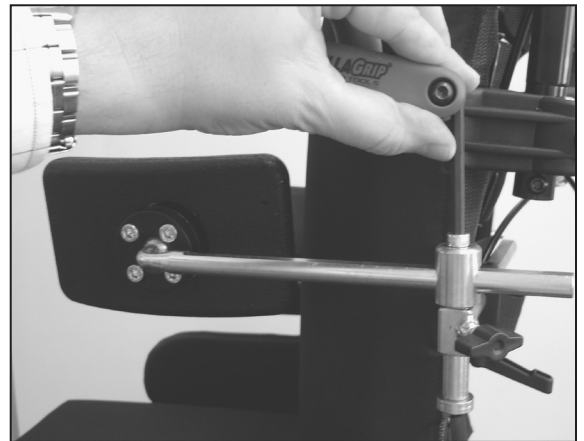

To adjust the angle of the side support use an Allen key on the four screws on the outside of the side support.



The side support is released by loosening the adjustment wheel. Then lift the support straight up, and swing it out to the side.

