


Product Info

Date: 13/1 2006	Product info.: 1-2006	Written by: Fredrik Delås
Item: Abduction block	Delivery: 1. February	

We have now come up with a new abduction block with much more flexibility and user friendly solutions!

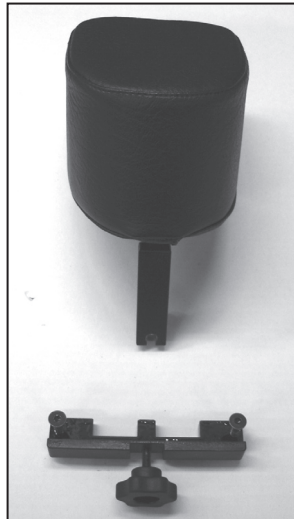


The new abduction block can easily be mounted on both comfort seat plate and the sliding seat. It is also much easier to mount and take off while the user is sitting in the wheelchair.

The abduction block is designed a little bigger, and is made from polyurethane foam covered with artificial leather. The block is put together so that it makes no pressure points when the block is compressed.

The abduction block is adjustable in height and depth. This is to optimize the individual adjustment possibilities and to make it compatible with the different cushions on the market.

The depth adjustability makes it possible to place the abduction block where it is supposed to be; in front between the knees. Most abduction blocks on the market today are placed in the groin of the user, and often lead to a non comfortable pressure.

To get an optimum result of the abduction block, I recommend to always use it in combination with a hip belt. The hip belt positions the pelvis, and the abduction block reduces the abduction of the legs.

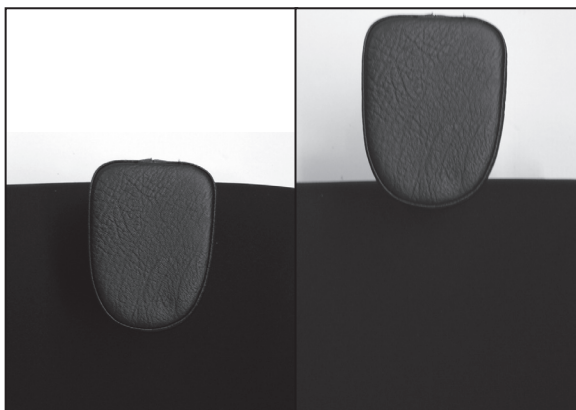

Item number	Item
27490	Abduction block Small 80mm
27491	Abduction block Medium 110mm
27492	Abduction block Large 140mm