


# SPARE PART CATALOG NETTI I

Version 2/08 -

# INDEX


	Page
Chair frame	3
Seat cushion - complete	4
Seat cushion - outer cover	5
Backrest - fixed angles	6
Backrest - recline	7
Backrest cushion	8
Push handle	9
Head support A	10
Head support B	11
Head support C	12
Arm support - left	13
Arm support - right	14
Comfort pads arm support	15
Arm support bracket	16
Extension piece foot support	17
Universal foot supports - complete	18
Universal foot supports - parts	19
Angle adjustable foot supports - complete	20
Angle adjustable foot supports - parts	21
Bearing house	23
Front castors	24
Brake	25
Gas cylinder - recline	26
Main wheels	27
Anti tip	28

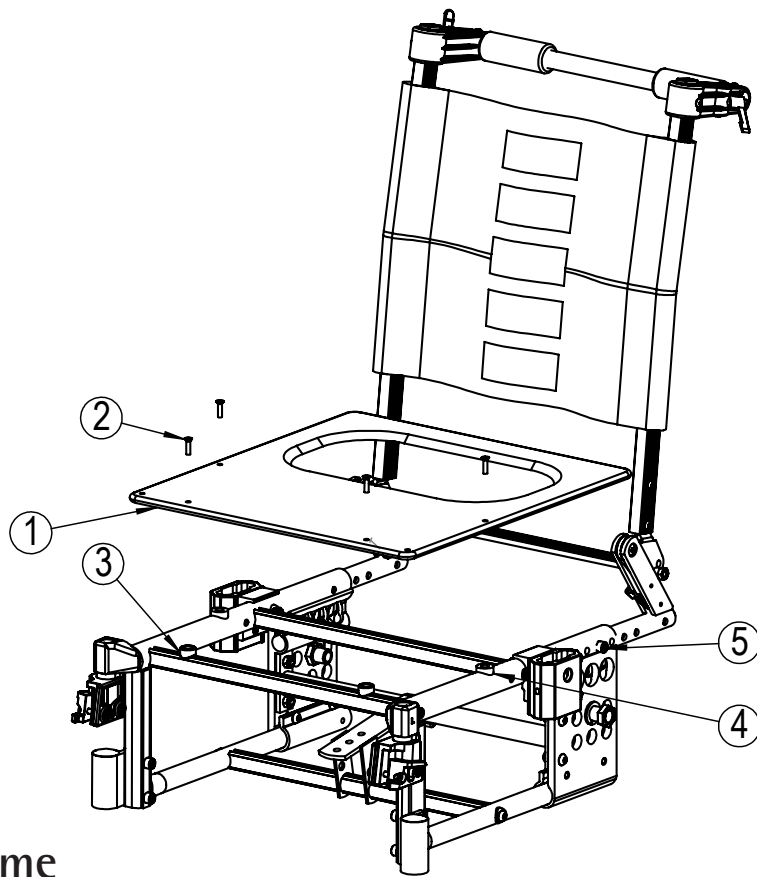

## Chair frame


Position	Description	Item no.	Pcs. pr. chair
1	Seat plate. Width: 35cm Depth: 40cm	20176	1
1	Seat plate. Width: 35cm Depth: 43cm	21040	1
1	Seat plate. Width: 35cm Depth: 46cm	20612	1
1	Seat plate. Width: 38cm Depth: 40cm	20910	1
1	Seat plate. Width: 38cm Depth: 43cm	21041	1
1	Seat plate. Width: 38cm Depth: 46cm	20758	1
1	Seat plate. Width: 40cm Depth: 40cm	20911	1
1	Seat plate. Width: 40cm Depth: 43cm	21042	1
1	Seat plate. Width: 40cm Depth: 46cm	20550	1
1	Seat plate. Width: 43cm Depth: 40cm	20195	1
1	Seat plate. Width: 43cm Depth: 43cm	21043	1
1	Seat plate. Width: 43cm Depth: 46cm	20551	1
1	Seat plate. Width: 45cm Depth: 40cm	20196	1
1	Seat plate. Width: 45cm Depth: 43cm	21044	1
1	Seat plate. Width: 45cm Depth: 46cm	20552	1
1	Seat plate. Width: 50cm Depth: 40cm	20197	1
1	Seat plate. Width: 50cm Depth: 43cm	21045	1
1	Seat plate. Width: 50cm Depth: 46cm	20553	1
1	Seat plate solid wood. Width: 55cm Depth: 40cm	20759	1
1	Seat plate solid wood. Width: 55cm Depth: 46cm	20635	1
1	Seat plate solid wood. Width: 60cm Depth: 40cm	20616	1
1	Seat plate solid wood. Width: 60cm Depth: 46cm	20617	1
2*	Screw M5x35	22098	4
3*	Frame Seat plate Washer High	24141	2
4*	Frame Seat plate Washer Low	22084	2
5*	Screw M8x40	21095	6

\* Available upon request.

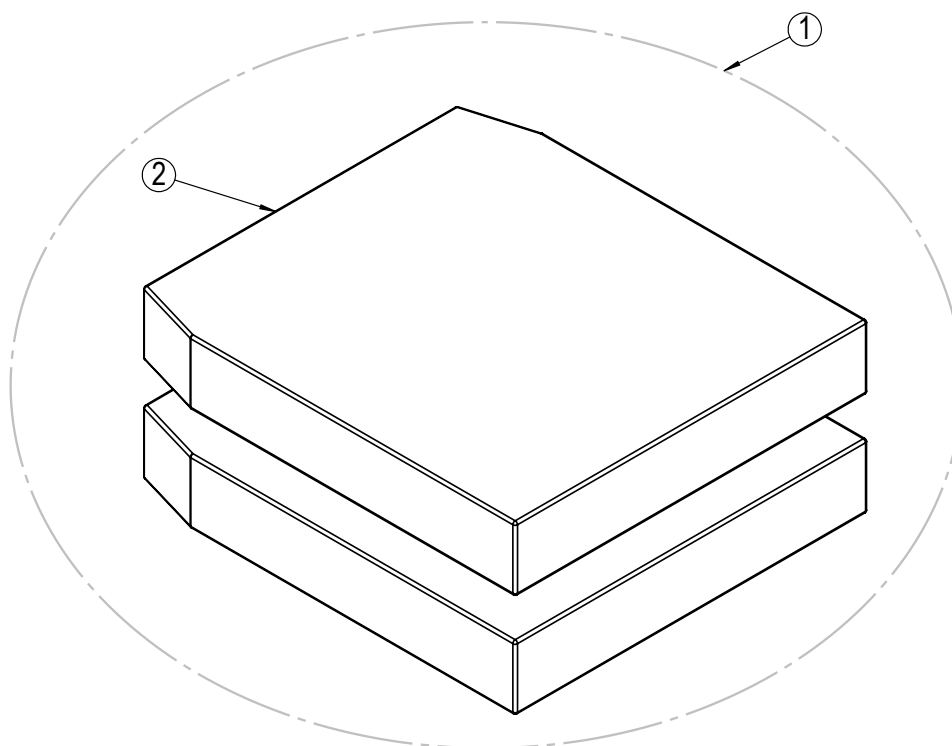


## Seat cushion - complete

Position	Description	Item no.	Pcs. pr. chair
1	Seat cushion, Uno Seat Width:35cm Depth:40cm, Mars	840377	1
1	Seat cushion, Uno Seat Width:35cm Depth:45cm, Mars	840378	1
1	Seat cushion, Uno Seat Width:35cm Depth:50cm, Mars	840379	1
1	Seat cushion, Uno Seat Width:38cm Depth:40cm, Mars	840380	1
1	Seat cushion, Uno Seat Width:38cm Depth:45cm, Mars	840381	1
1	Seat cushion, Uno Seat Width:38cm Depth:50cm, Mars	840382	1
1	Seat cushion, Uno Seat Width:40cm Depth:40cm, Mars	840385	1
1	Seat cushion, Uno Seat Width:40cm Depth:45cm, Mars	840386	1
1	Seat cushion, Uno Seat Width:40cm Depth:50cm, Mars	840387	1
1	Seat cushion, Uno Seat Width:43cm Depth:40cm, Mars	840390	1
1	Seat cushion, Uno Seat Width:43cm Depth:45cm, Mars	840391	1
1	Seat cushion, Uno Seat Width:43cm Depth:50cm, Mars	840392	1
1	Seat cushion, Uno Seat Width:45cm Depth:40cm, Mars	840395	1
1	Seat cushion, Uno Seat Width:45cm Depth:45cm, Mars	840396	1
1	Seat cushion, Uno Seat Width:45cm Depth:50cm, Mars	840397	1
1	Seat cushion, Uno Seat Width:50cm Depth:40cm, Mars	840400	1
1	Seat cushion, Uno Seat Width:50cm Depth:45cm, Mars	840401	1
1	Seat cushion, Uno Seat Width:50cm Depth:50cm, Mars	840402	1
1	Seat cushion, Uno Seat Width:55cm Depth:40cm, Mars	840405	1
1	Seat cushion, Uno Seat Width:55cm Depth:45cm, Mars	840406	1
1	Seat cushion, Uno Seat Width:55cm Depth:50cm, Mars	840407	1
1	Seat cushion, Uno Seat Width:60cm Depth:40cm, Mars	840410	1
1	Seat cushion, Uno Seat Width:60cm Depth:45cm, Mars	840411	1
1	Seat cushion, Uno Seat Width:60cm Depth:50cm, Mars	840412	1

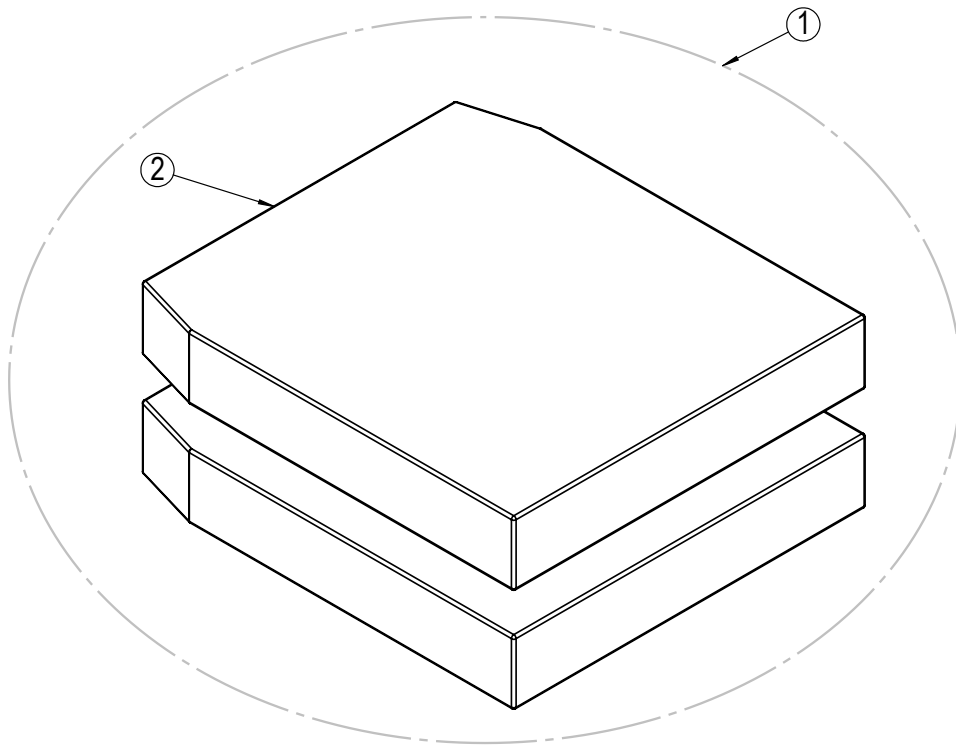

## Seat cushion - outer cover


Position	Description	Item no.	Pcs. pr. chair
2	Outer cover, Uno Seat Width:35cm Depth:40cm, Mars	3543540	1
2	Outer cover, Uno Seat Width:35cm Depth:45cm, Mars	3543545	1
2	Outer cover, Uno Seat Width:35cm Depth:50cm, Mars	3543550	1
2	Outer cover, Uno Seat Width:38cm Depth:40cm, Mars	3543840	1
2	Outer cover, Uno Seat Width:38cm Depth:45cm, Mars	3543845	1
2	Outer cover, Uno Seat Width:38cm Depth:50cm, Mars	3543850	1
2	Outer cover, Uno Seat Width:40cm Depth:40cm, Mars	3544040	1
2	Outer cover, Uno Seat Width:40cm Depth:45cm, Mars	3544045	1
2	Outer cover, Uno Seat Width:40cm Depth:50cm, Mars	3544050	1
2	Outer cover, Uno Seat Width:43cm Depth:40cm, Mars	3544340	1
2	Outer cover, Uno Seat Width:43cm Depth:45cm, Mars	3544345	1
2	Outer cover, Uno Seat Width:43cm Depth:50cm, Mars	3544550	1
2	Outer cover, Uno Seat Width:45cm Depth:40cm, Mars	3544540	1
2	Outer cover, Uno Seat Width:45cm Depth:45cm, Mars	3544545	1
2	Outer cover, Uno Seat Width:45cm Depth:50cm, Mars	3544550	1
2	Outer cover, Uno Seat Width:50cm Depth:40cm, Mars	3545040	1
2	Outer cover, Uno Seat Width:50cm Depth:45cm, Mars	3545045	1
2	Outer cover, Uno Seat Width:50cm Depth:50cm, Mars	3545050	1
2	Outer cover, Uno Seat Width:55cm Depth:40cm, Mars	3545540	1
2	Outer cover, Uno Seat Width:55cm Depth:45cm, Mars	3545545	1
2	Outer cover, Uno Seat Width:55cm Depth:50cm, Mars	3545550	1
2	Outer cover, Uno Seat Width:60cm Depth:40cm, Mars	3546040	1
2	Outer cover, Uno Seat Width:60cm Depth:45cm, Mars	3546045	1
2	Outer cover, Uno Seat Width:60cm Depth:50cm, Mars	3546050	1

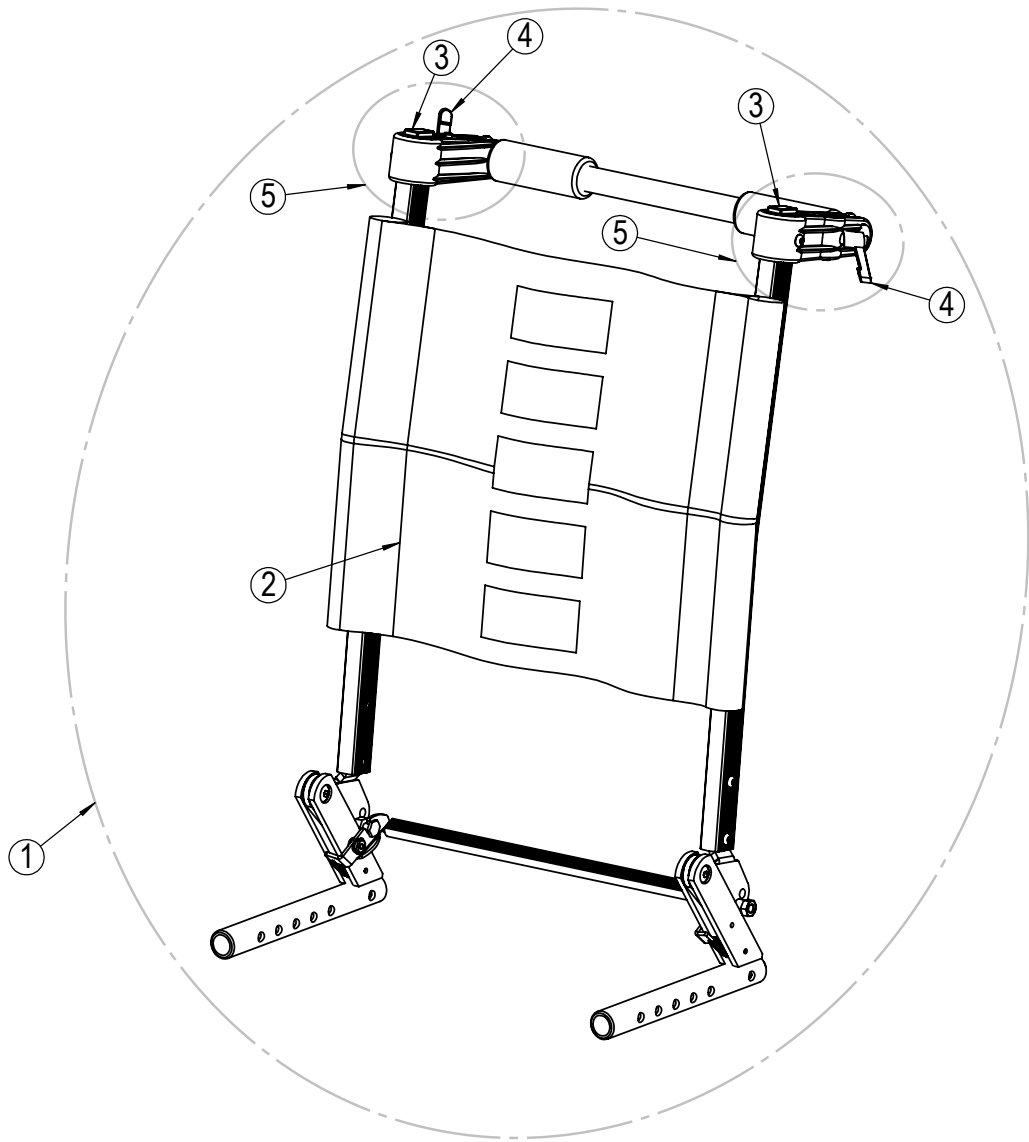

## Backrest - fixed angles

Position	Description	Item no.	Pcs. pr. unit
1	Backrest 35cm fixed Netti I/II complete	21892	1
1	Backrest 38cm fixed Netti I/II complete	21893	1
1	Backrest 40cm fixed Netti I/II complete	21894	1
1	Backrest 43cm fixed Netti I/II complete	21895	1
1	Backrest 45cm fixed Netti I/II complete	21896	1
1	Backrest 50cm fixed Netti I/II complete	21897	1
1	Backrest 55cm fixed Netti I/II complete	21898	1
1	Backrest 60cm fixed Netti I/II complete	21899	1
2	Velcro 35-55cm	20699	1
2	Velcro 55-60cm	2865	1
3*	Square plug 25x15	21847	2
4*	Clamp lever M6x16	20097	2


\* Available upon request

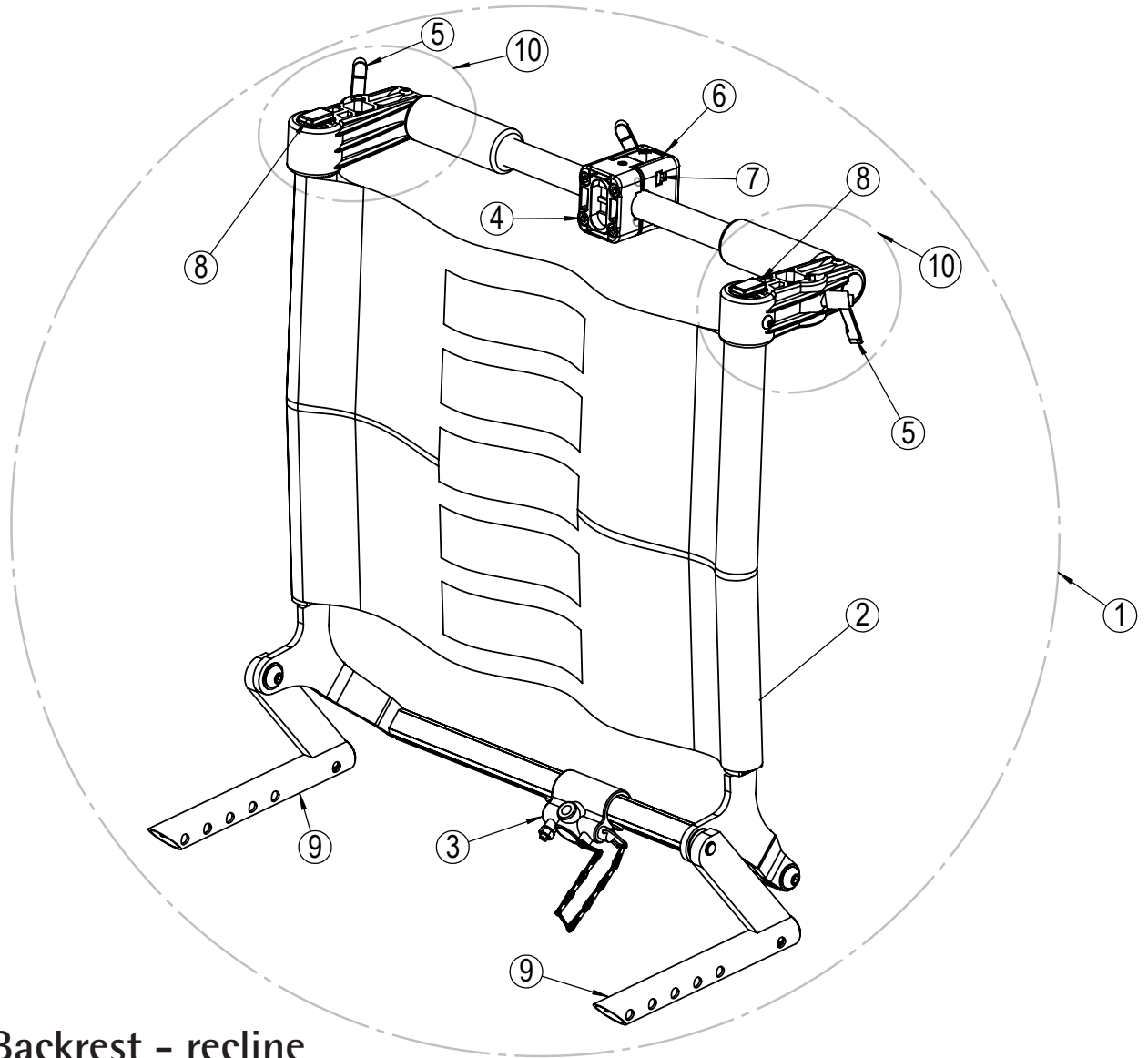

## Backrest - recline


Position	Description	Item no.	Pcs. pr. unit
1	Backrest with recline 35cm Netti I complete	26912	1
1	Backrest with recline 38cm Netti I complete	26912	1
1	Backrest with recline 40cm Netti I complete	26912	1
1	Backrest with recline 43cm Netti I complete	26912	1
1	Backrest with recline 45cm Netti I complete	26912	1
1	Backrest with recline 50cm Netti I complete	26912	1
1	Backrest with recline 55cm Netti I complete	26912	1
1	Backrest with recline 60cm Netti I complete	26912	1
2	Velcro 35-50cm	20699	1
2	Velcro 50-60cm	2865	1
3	Cylinder hinge	20433	1
4	Screw M6x40	20431	5
5*	Clamp lever M6x16	20097	3
6	Head support adapter complete	23005	1
7*	Square nut M6	20296	4
8*	Square plug 25x15	21847	2
9	Backrest hinge	22062	2
10	Backrest push bow bracket complete	26051	2

\* Available upon request

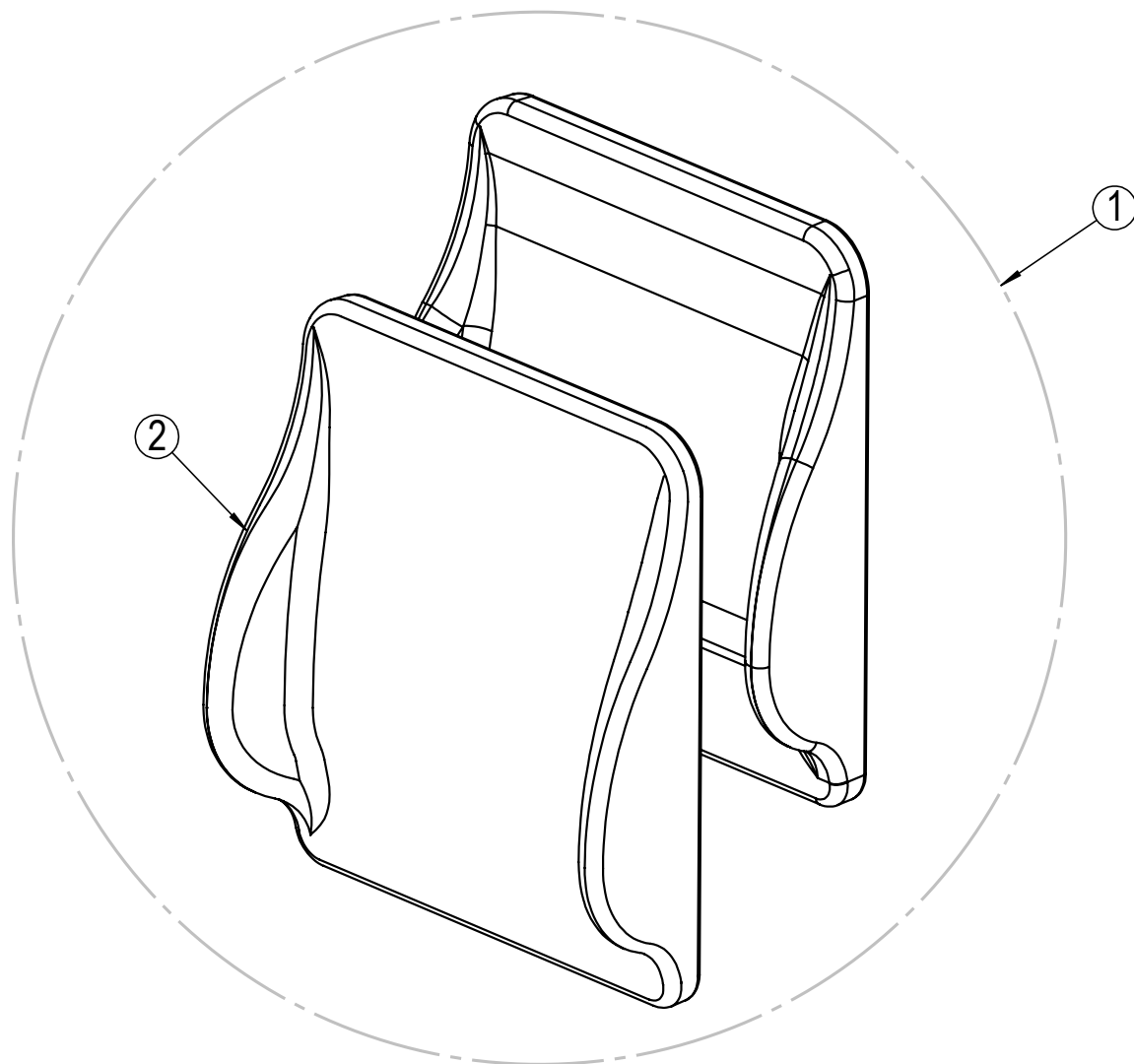

## Backrest cushion


Position	Description	Item no.	Pcs. pr. chair
1	Backrest cushion, Uno Back Width:35cm Height:50cm, Mars	3735035	1
1	Backrest cushion, Uno Back Width:38cm Height:50cm, Mars	3735038	1
1	Backrest cushion, Uno Back Width:40cm Height:50cm, Mars	3735040	1
1	Backrest cushion, Uno Back Width:43cm Height:50cm, Mars	3735043	1
1	Backrest cushion, Uno Back Width:45cm Height:50cm, Mars	3735045	1
1	Backrest cushion, Uno Back Width:50cm Height:50cm, Mars	3735050	1
1	Backrest cushion, Uno Back Width:55cm Height:50cm, Mars	3735055	1
1	Backrest cushion, Uno Back Width:60cm Height:50cm, Mars	3735060	1
2	Outer cover, Uno Back Width:35cm Height:50cm, Mars	3745035	1
2	Outer cover, Uno Back Width:38cm Height:50cm, Mars	3745038	1
2	Outer cover, Uno Back Width:40cm Height:50cm, Mars	3745040	1
2	Outer cover, Uno Back Width:43cm Height:50cm, Mars	3745043	1
2	Outer cover, Uno Back Width:45cm Height:50cm, Mars	3745045	1
2	Outer cover, Uno Back Width:50cm Height:50cm, Mars	3745050	1
2	Outer cover, Uno Back Width:55cm Height:50cm, Mars	3745055	1
2	Outer cover, Uno Back Width:60cm Height:50cm, Mars	3745060	1

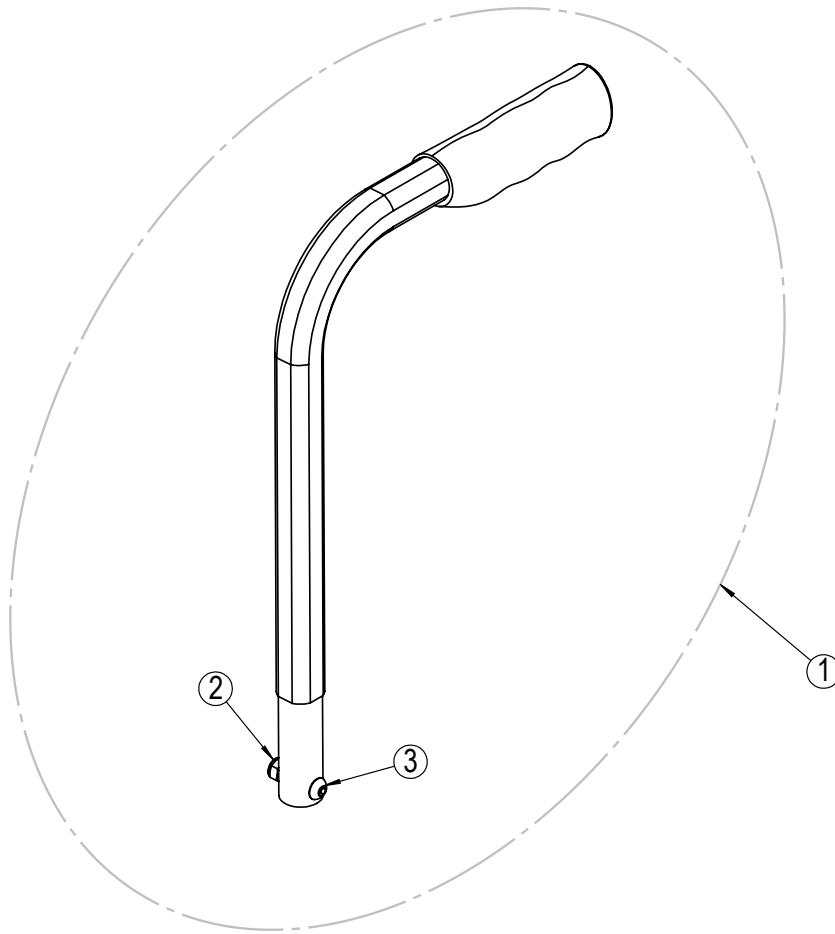

## Push handle

Position	Description	Item no.	Pcs. pr. chair
1	Push handle standard 300mm complete	21754	2
1	Push handle long 500mm complete	21808	2
2*	Lock nut M6	20058	2
3*	Screw M6x25	20992	2


\* Available upon request

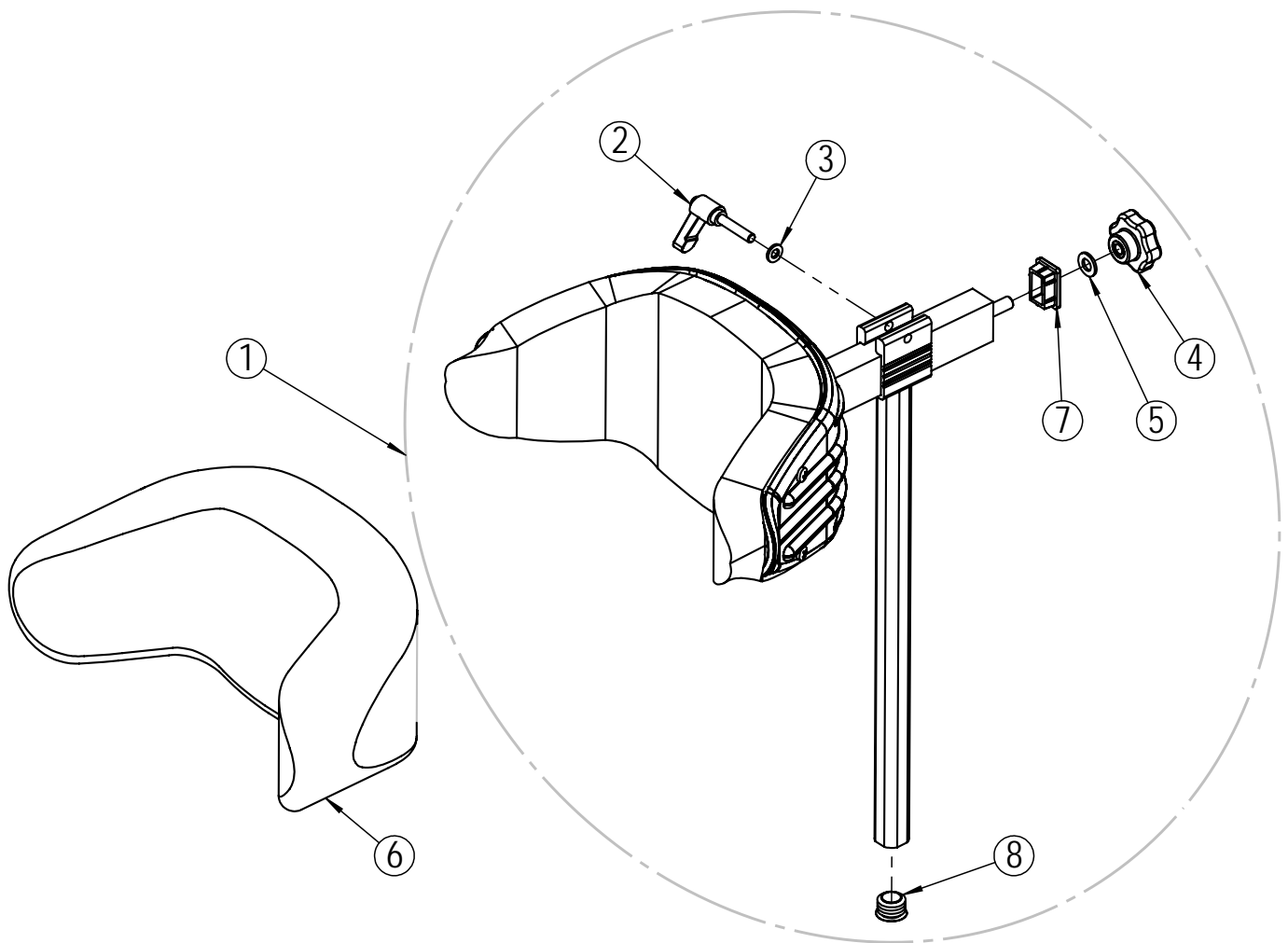

## Head support A

Position	Description	Item no.	Pcs. pr. chair
1	Head support A with Mars	20575	1
2*	Clamp lever M6x30	20282	1
3*	Washer M6	20063	1
4*	Star wheel M8	26684	1
5*	Washer M8	29305	1
6	Hygiene cover Mars Head support A	20126	1
7*	Square plug horizontal bar	20852	1
8*	Plastic plug vertical bar	20509	1


\* Available upon request.

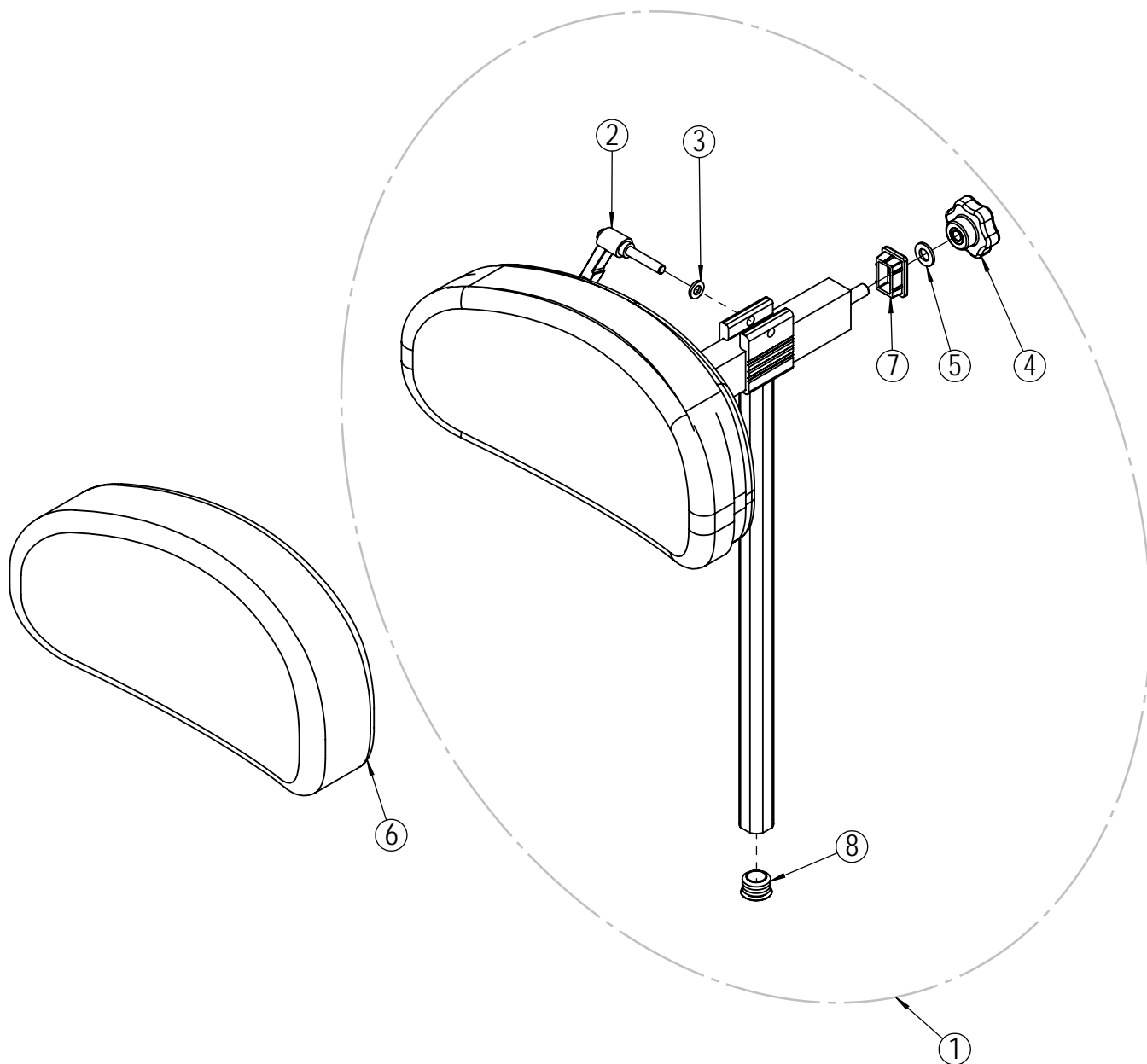

## Head support B

Position	Description	Item no.	Pcs. pr. chair
1	Head support B with Mars	20574	1
2*	Clamp lever M6x30	20282	1
3*	Washer M6	20063	1
4*	Star wheel M8	26684	1
5*	Washer M8	29305	1
6	Hygiene cover Mars Head support B	20629	1
7*	Square plug horizontal bar	20852	1
8*	Plastic plug vertical bar	20509	1


\* Available upon request.

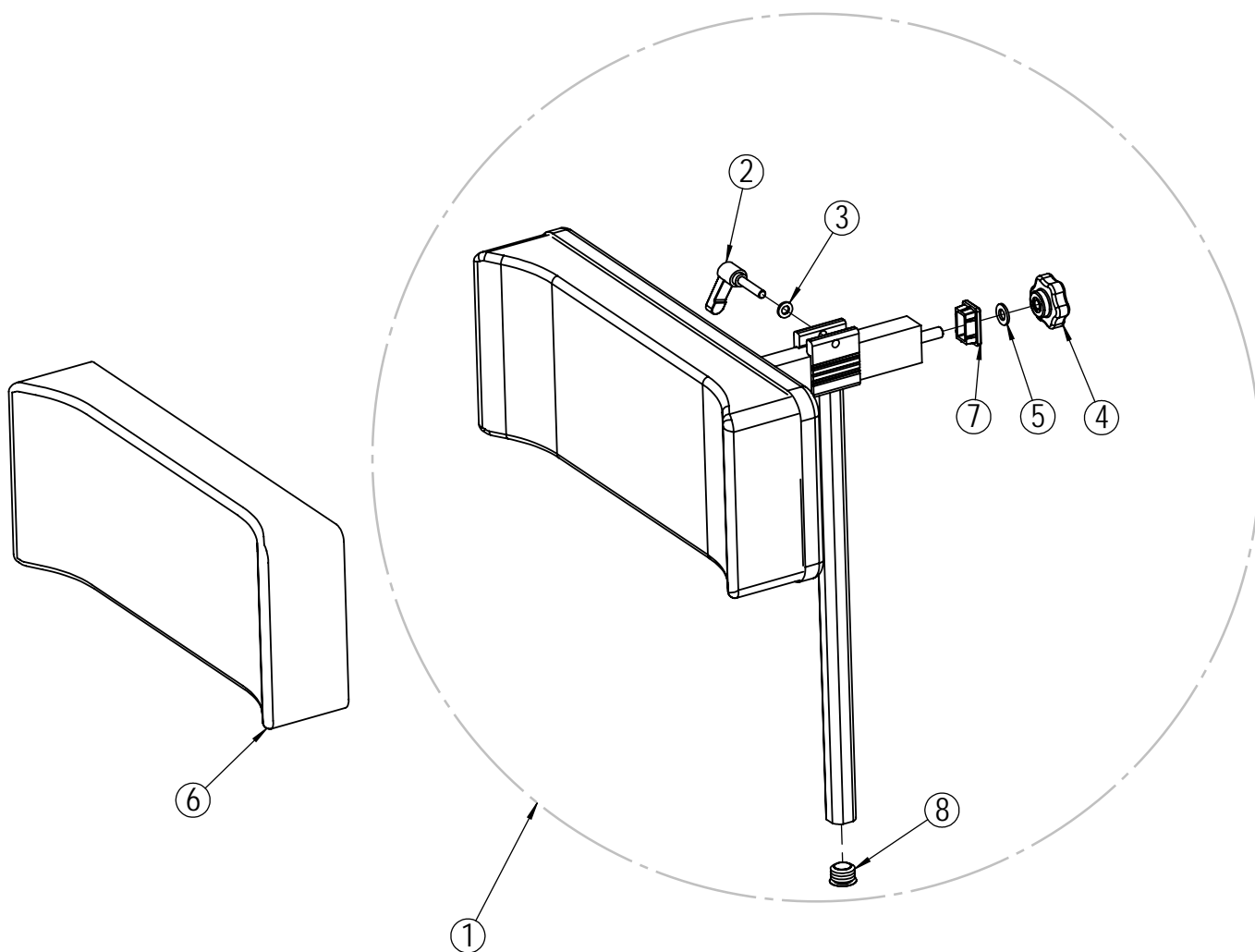

## Head support C

Position	Description	Item no.	Pcs. pr. chair
1	Head support C with Mars	20573	1
2*	Clamp lever M6x30	20282	1
3*	Washer M6	20063	1
4*	Star wheel M8	26684	1
5*	Washer M8	29305	1
6	Hygiene cover Mars Head support C	20628	1
7*	Square plug horizontal bar	20852	1
8*	Plastic plug vertical bar	20509	1


\* Available upon request.

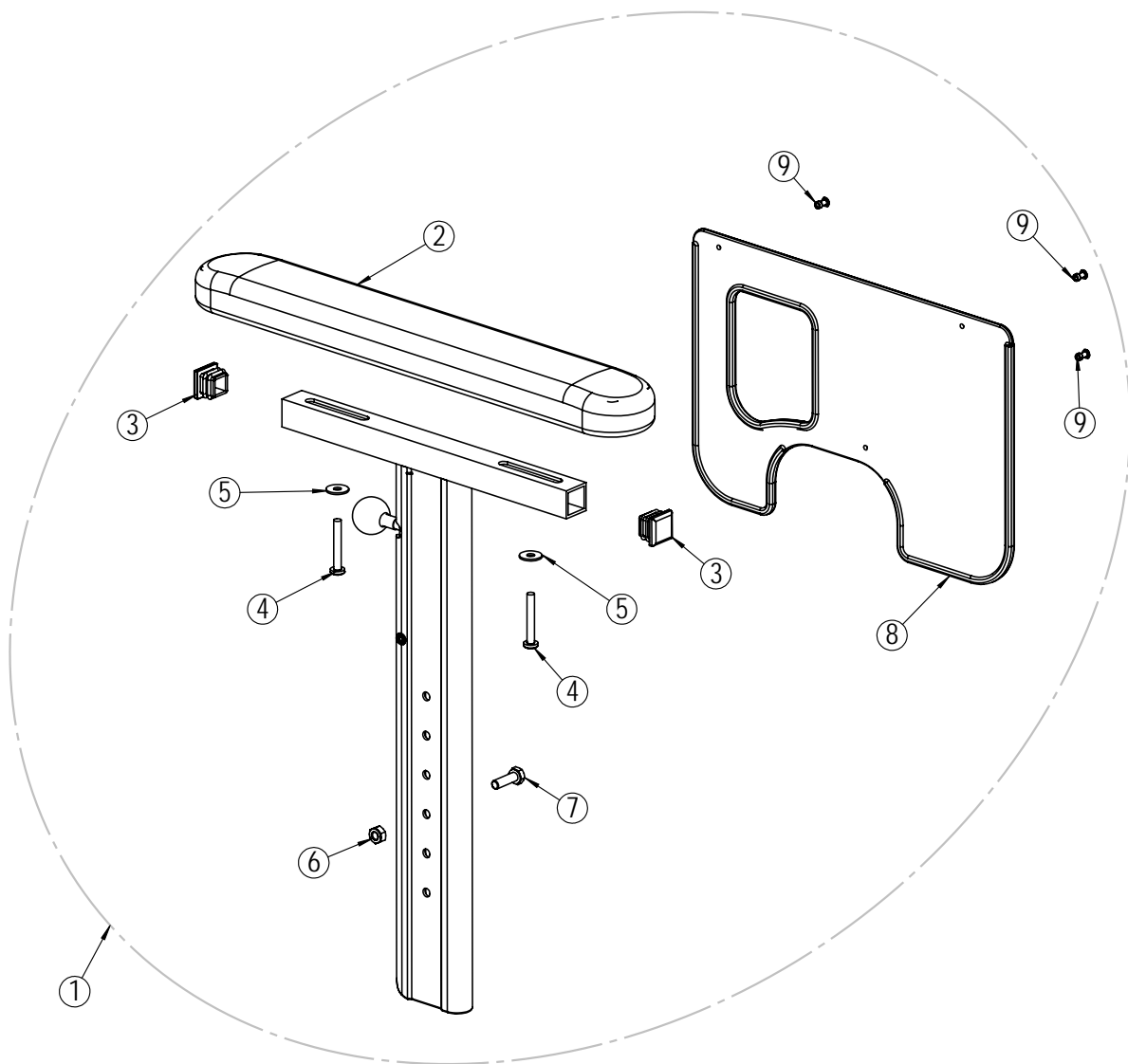

### Arm support - left

Position	Description	Item no.	Pcs. pr. chair
1	Arm support left complete	21681	1
2	Arm support pad standard 385x58 PU with screws	20835	1
3*	Square plug 20x20x12 mm	20633	2
4*	Screw M5x35	21323	2
5*	Washer M5	23594	2
6*	Screw M6x20	20406	1
7*	Nut M6	20407	1
8	Skirt guard arm support	20239	1
9*	Pop black $\varnothing 3.2 \times 9.7 \text{ mm}$	20060	3


\* Available upon request

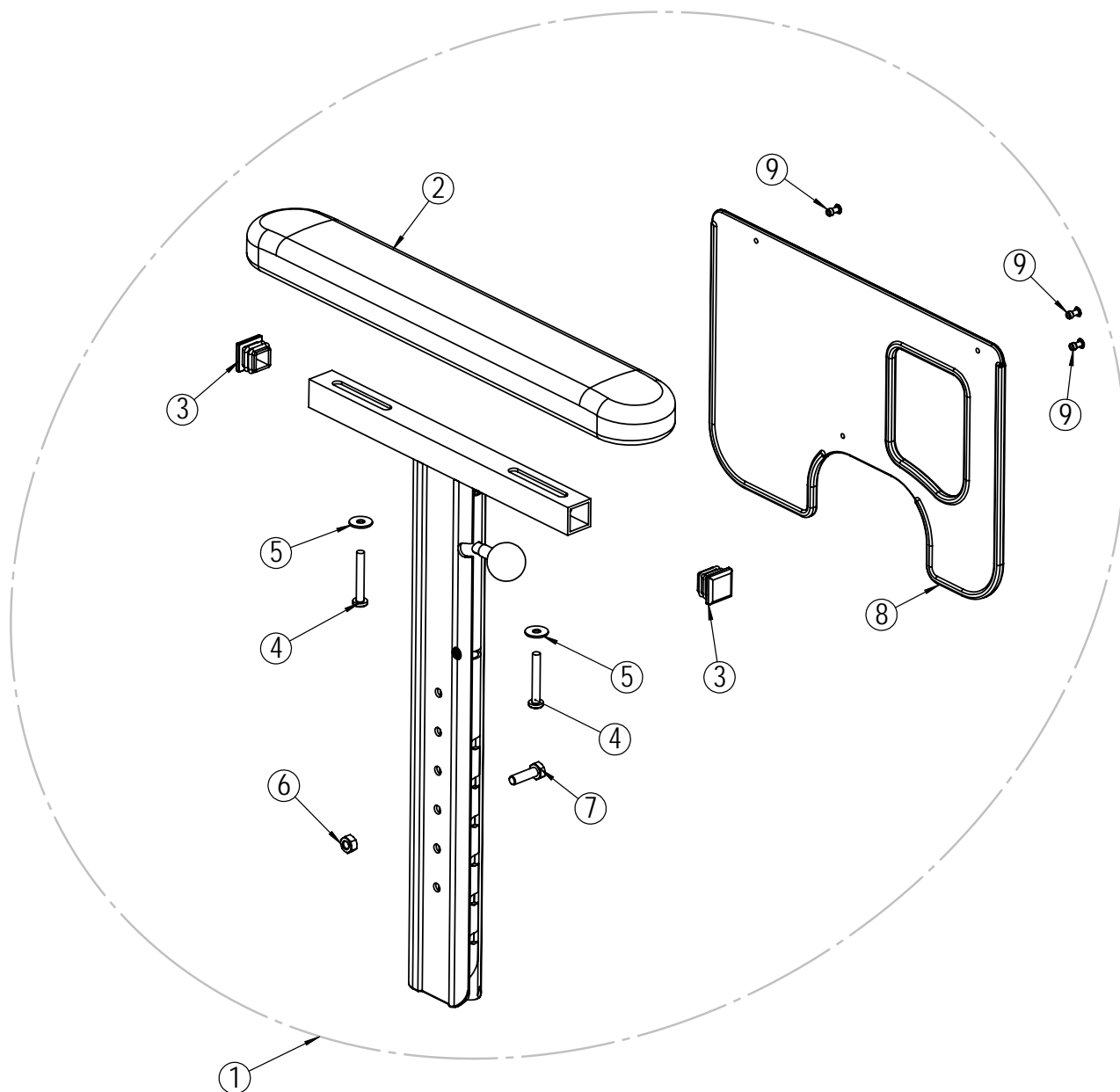

## Arm support - right


Position	Description	Item no.	Pcs. pr. chair
1	Arm support right complete	21682	1
2	Arm support pad standard 385x58 PU with screws	20835	1
3*	Square plug 20x20x12 mm	20633	2
4*	Screw M5x35	21323	2
5*	Washer M5	23594	2
6*	Screw M6x20	20406	1
7*	Nut M6	20407	1
8	Skirt guard arm support	20239	1
9*	Pop black $\varnothing$ 3.2x9.7mm	20060	3

\* Available upon request

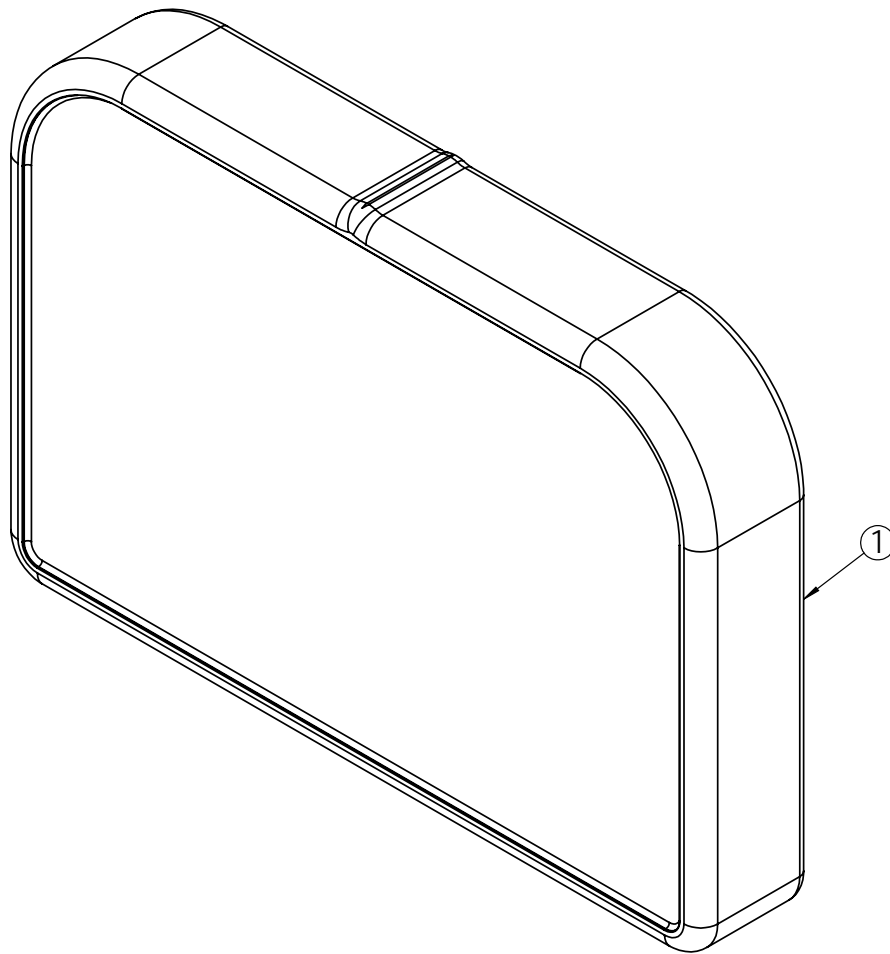

## Comfort pads arm support


Position	Description	Item no.	Pcs. pr. chair
1	Comfort pads. 2,5cm	22149	2
1	Comfort pads. 3,5cm	22150	2
1	Comfort pads. 3,5cm	22151	2

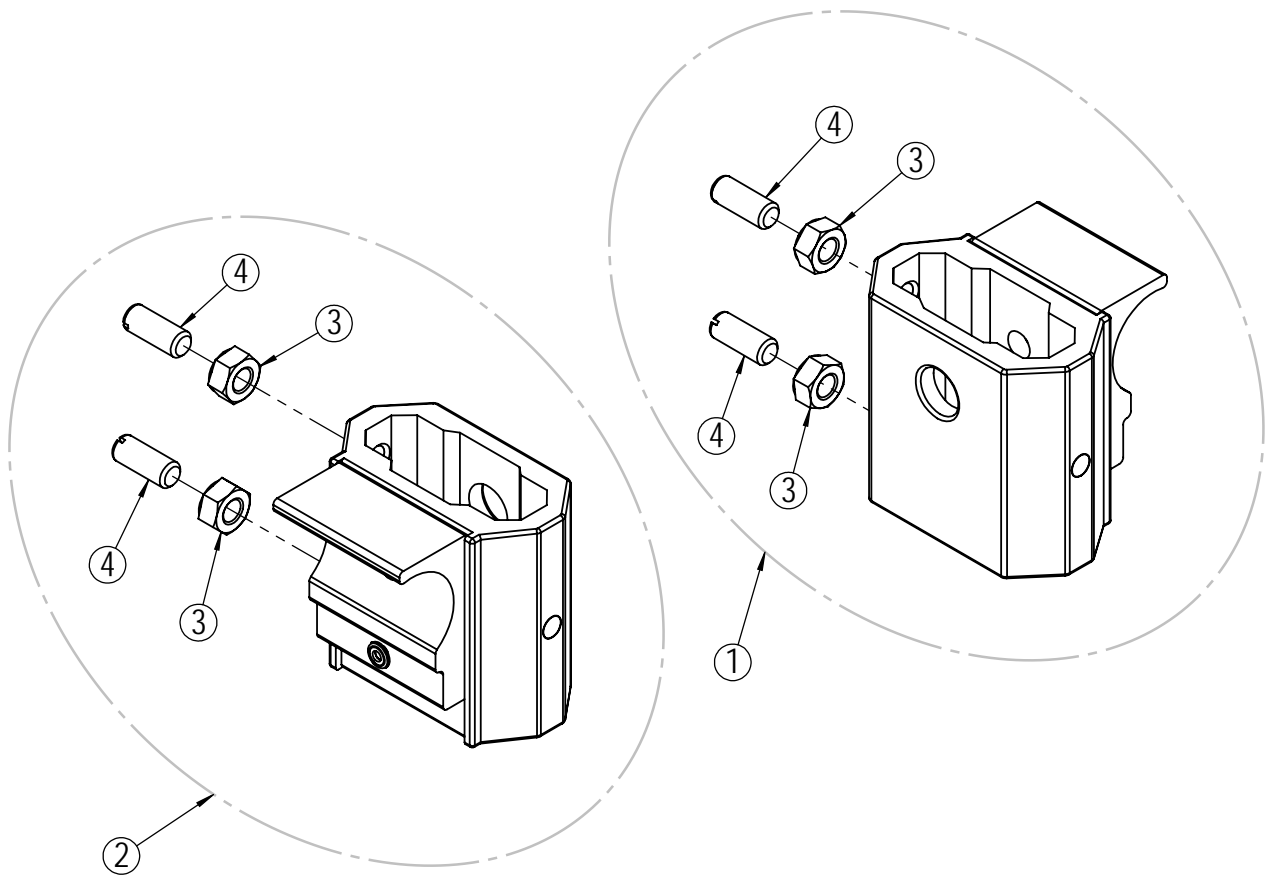

## Arm support bracket


Position	Description	Item no.	Pcs. pr. chair
1	Arm support bracket right complete	20404	1
2	Arm support bracket left complete	20401	1
3*	Nut M8	20408	4
4*	Nylon screw M8x20	20857	4

\* Available upon request

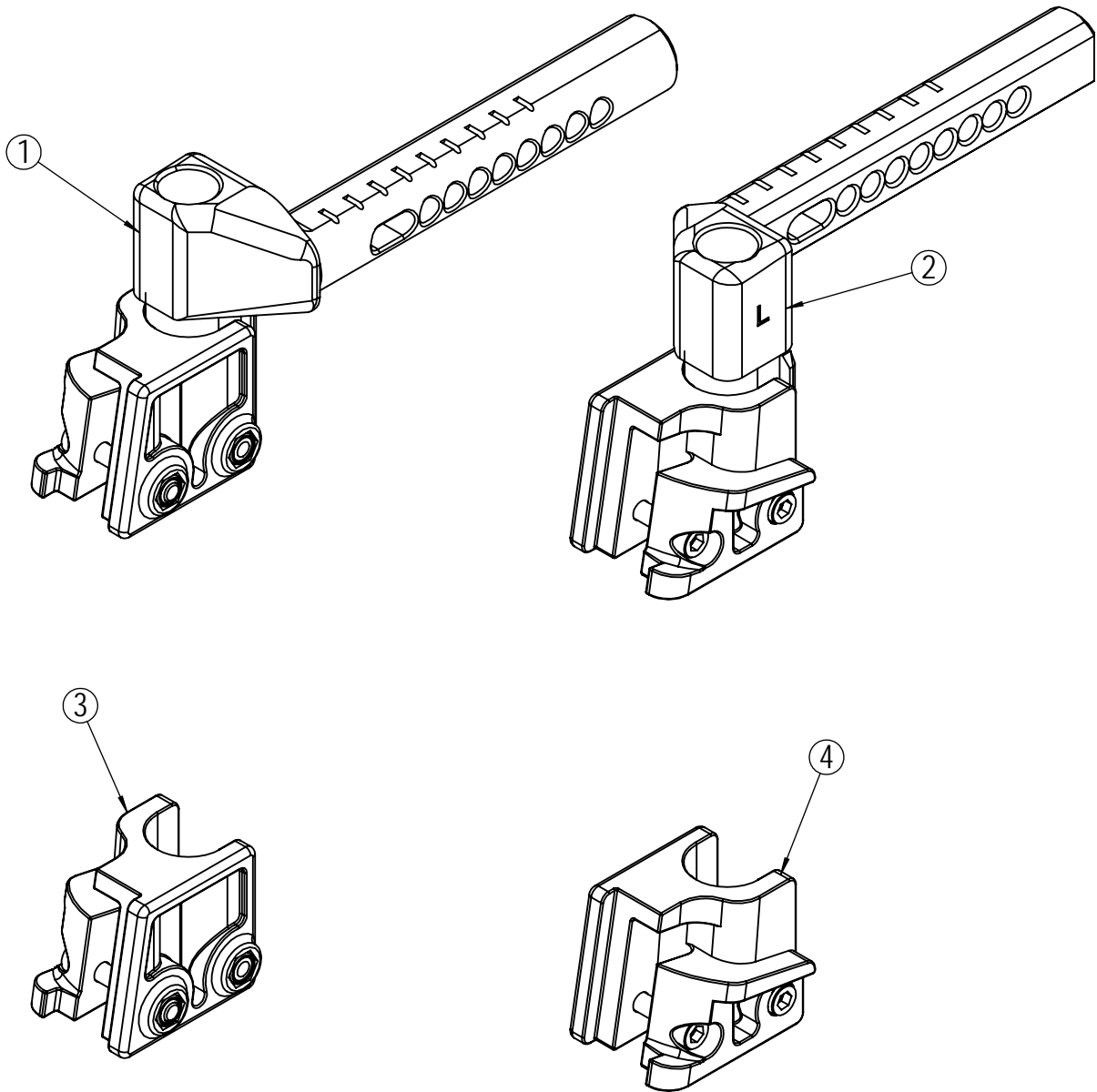


### Extension piece - foot support

Position	Description	Item no.	Pcs. pr. chair
1	Extension piece. Silver. Right. Complete.	27384	1
2	Extension piece. Silver. Left. Complete.	27385	1
3	Locking bracket with screw. Right.	29555	1
4	Locking bracket with screw. Left.	29556	1

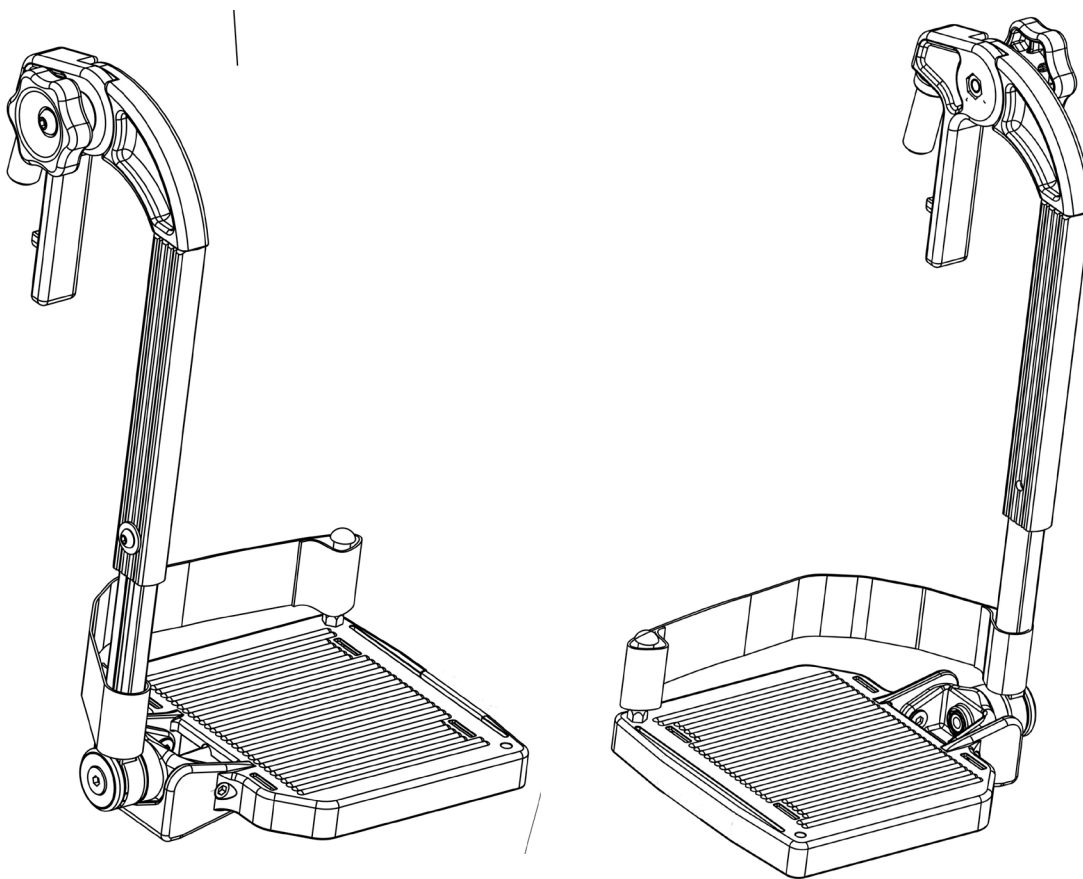

## Universal foot support - complete


Position	Description	Item no.	Pcs. pr. chair
1	Universal foot support. Silver. 35cm. Right.	27220	1
2	Universal foot support. Silver. 35cm Left.	27221	1
1	Universal foot support. Silver. 38cm. Right.	27222	1
2	Universal foot support. Silver. 38cm Left.	27223	1
1	Universal foot support. Silver. 40cm. Right.	27224	1
2	Universal foot support. Silver. 40cm Left.	27225	1
1	Universal foot support. Silver. 43cm. Right.	27226	1
2	Universal foot support. Silver. 43cm Left.	27227	1
1	Universal foot support. Silver. 45cm. Right.	27228	1
2	Universal foot support. Silver. 45cm Left.	27229	1
1	Universal foot support. Silver. 50cm. Right.	27230	1
2	Universal foot support. Silver. 50cm Left.	27231	1
1	Universal foot support. Silver. 55cm. Right.	27232	1
2	Universal foot support. Silver. 55cm Left.	27233	1
1	Universal foot support. Silver. 60cm. Right.	27234	1
2	Universal foot support. Silver. 60cm Left.	27235	1

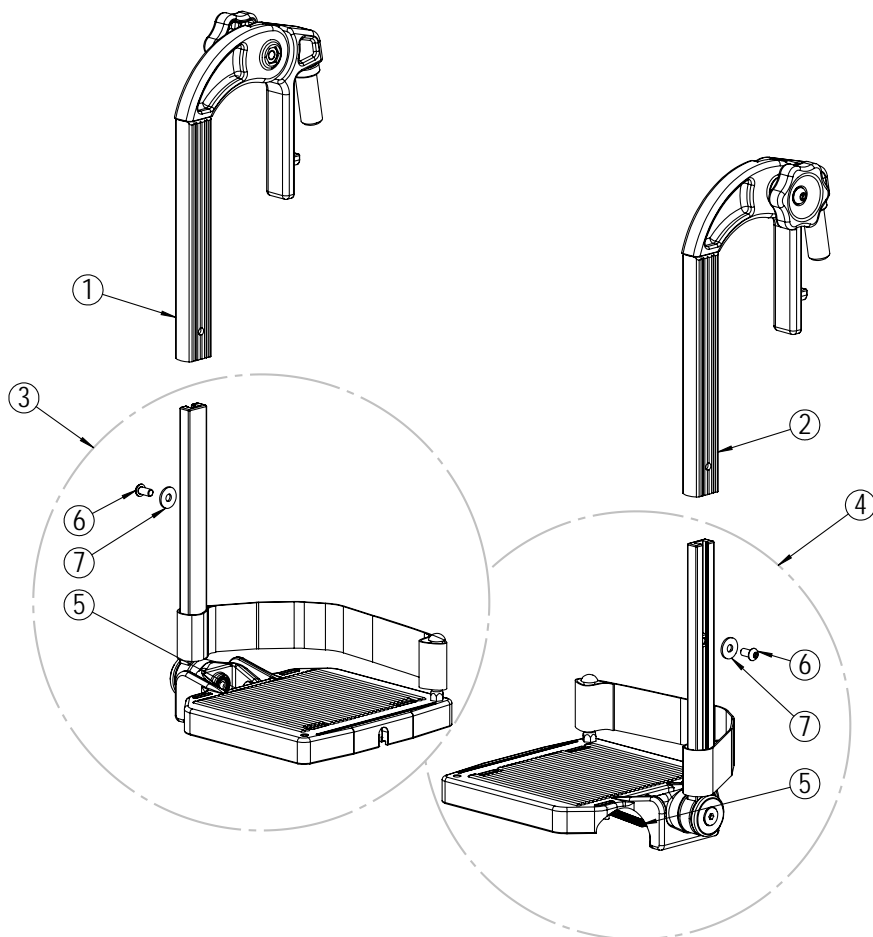

## Universal foot support - parts

Position	Description	Item no.	Pcs. pr. chair
1	Universal foot support. Swing element. Silver. Right.	25824	1
2	Universal foot support. Swing element. Silver. Left.	26723	1
3	Footplate with heelstrap. 35cm. Right.	27875	1
4	Footplate with heelstrap. . 35cm. Left.	27899	1
3	Footplate with heelstrap. . 38cm. Right.	27876	1
4	Footplate with heelstrap. . 38cm. Left.	27900	1
3	Footplate with heelstrap. . 40cm. Right.	27877	1
4	Footplate with heelstrap. . 40cm. Left.	27901	1
3	Footplate with heelstrap. . 43cm. Right.	27878	1
4	Footplate with heelstrap. . 43cm. Left.	27902	1
3	Footplate with heelstrap. . 45cm. Right.	27879	1
4	Footplate with heelstrap. . 45cm. Left.	27903	1
3	Footplate with heelstrap. . 50cm. Right.	27880	1
4	Footplate with heelstrap. . 50cm. Left.	27904	1
3	Footplate with heelstrap. . 55cm. Right.	27881	1
4	Footplate with heelstrap. . 55cm. Left.	27905	1
3	Footplate with heelstrap. . 60cm. Right.	27882	1
4	Footplate with heelstrap. . 60cm. Left.	27906	1
5*	Footplate with heelstrap. fjær 1x6x65	27858	2
6*	Screw M6x12	24272	2
7*	Washer 6,4x18x1,5	20356	2

\* Available upon request

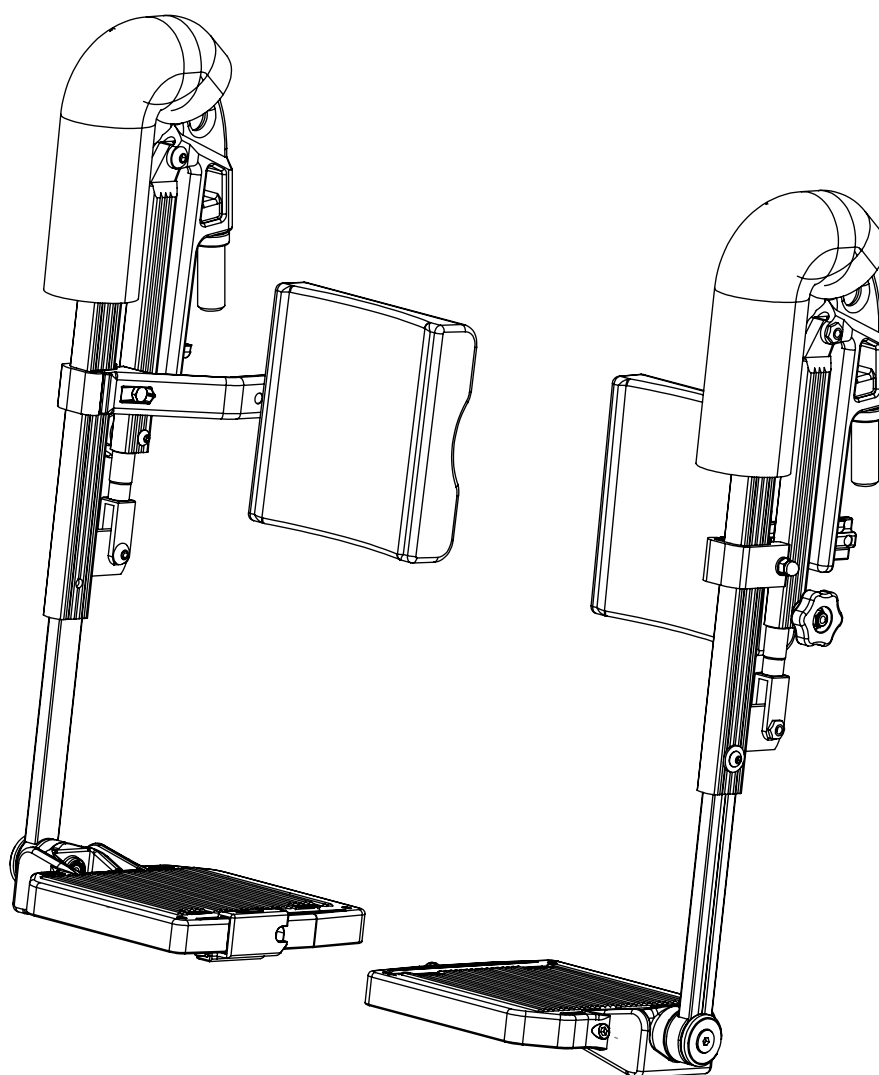


## Angle adjustable foot supports - complete


Position	Description	Item no.	Pcs. pr. chair
1	Angle adjustable foot support. Black. 35cm. Right.	29500	1
2	Angle adjustable foot support. Black. 35cm. Left.	29508	1
1	Angle adjustable foot support. Black. 40cm. Right.	29502	1
2	Angle adjustable foot support. Black. 40cm. Left.	29510	1
1	Angle adjustable foot support. Black. 43cm. Right.	29503	1
2	Angle adjustable foot support. Black. 43cm. Left.	29511	1
1	Angle adjustable foot support. Black. 45cm. Right.	29504	1
2	Angle adjustable foot support. Black. 45cm. Left.	29512	1
1	Angle adjustable foot support. Black. 50cm. Right.	29505	1
2	Angle adjustable foot support. Black. 50cm. Left.	29513	1
1	Angle adjustable foot support. Black. 55cm. Right.	29506	1
2	Angle adjustable foot support. Black. 55cm. Left.	29514	1
1	Angle adjustable foot support. Black. 60cm. Right.	29507	1
2	Angle adjustable foot support. Black. 60cm. Left..	29515	1

## Angle adjustable foot supports - parts

Position	Description	Item no.	Pcs. pr. chair
3	Angle adjustable foot support. Black. Right.	29489	1
4	Angle adjustable foot support. Black. Left.	29490	1
5	Knee upholstery. Mars.	23864	2
6	Calf pad with bracket. 35cm	27964	1
6	Calf pad with bracket. 40cm	27965	1
6	Calf pad with bracket. 43cm	27966	1
6	Calf pad with bracket. 45cm	27966	1
6	Calf pad with bracket. 50cm	27967	1
6	Calf pad with bracket. 55cm	27968	1
6	Calf pad with bracket. 60cm	27969	1
7	Foot plate. 35cm Right.	28621	1
8	Foot plate. 35cm Left.	28649	1
7	Foot plate. 40cm Right.	28623	1
8	Foot plate. 40cm Left.	28651	1
7	Foot plate. 43cm Right.	28624	1
8	Foot plate. 43cm Left.	28652	1
7	Foot plate. 45cm Right.	28625	1
8	Foot plate. 45cm Left.	28653	1
7	Foot plate. 50cm Right.	28626	1
8	Foot plate. 50cm Left.	28654	1
7	Foot plate. 55cm Right.	28627	1
8	Foot plate. 55cm Left.	28655	1
7	Foot plate. 60cm Right.	28628	1
8	Foot plate. 60cm Left.	28656	1
9*	Foot plate spring	27858	2
10*	Screw M6x12	24272	2
11*	Washer	20356	2
12	Mechanical adjustment piece with screw. Black. Right.	29558	1
13	Mechanical adjustment piece with screw. Black. Left.	29559	1
14*	Screw M6x40	20103	2
15*	Adjustment wheel	27388	2
16	Foot plate hinge complete	28523	2
17	Calf support bracket with screw, washer and lock nut	25353	2
18*	Nut M8	29761	2
19*	Foot support locking cone	20539	2
20*	Nut M6	20058	2

\* Available upon request.

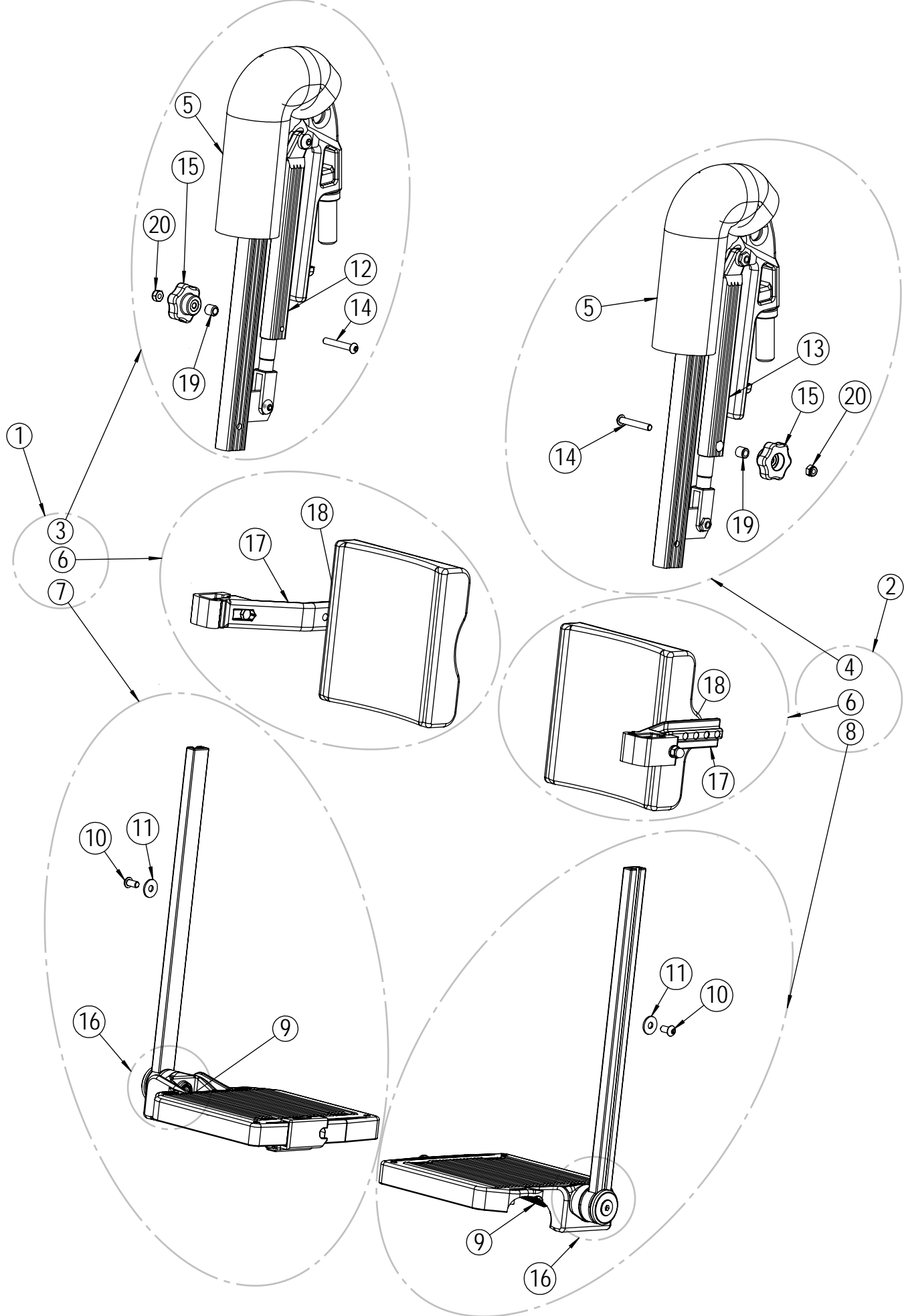


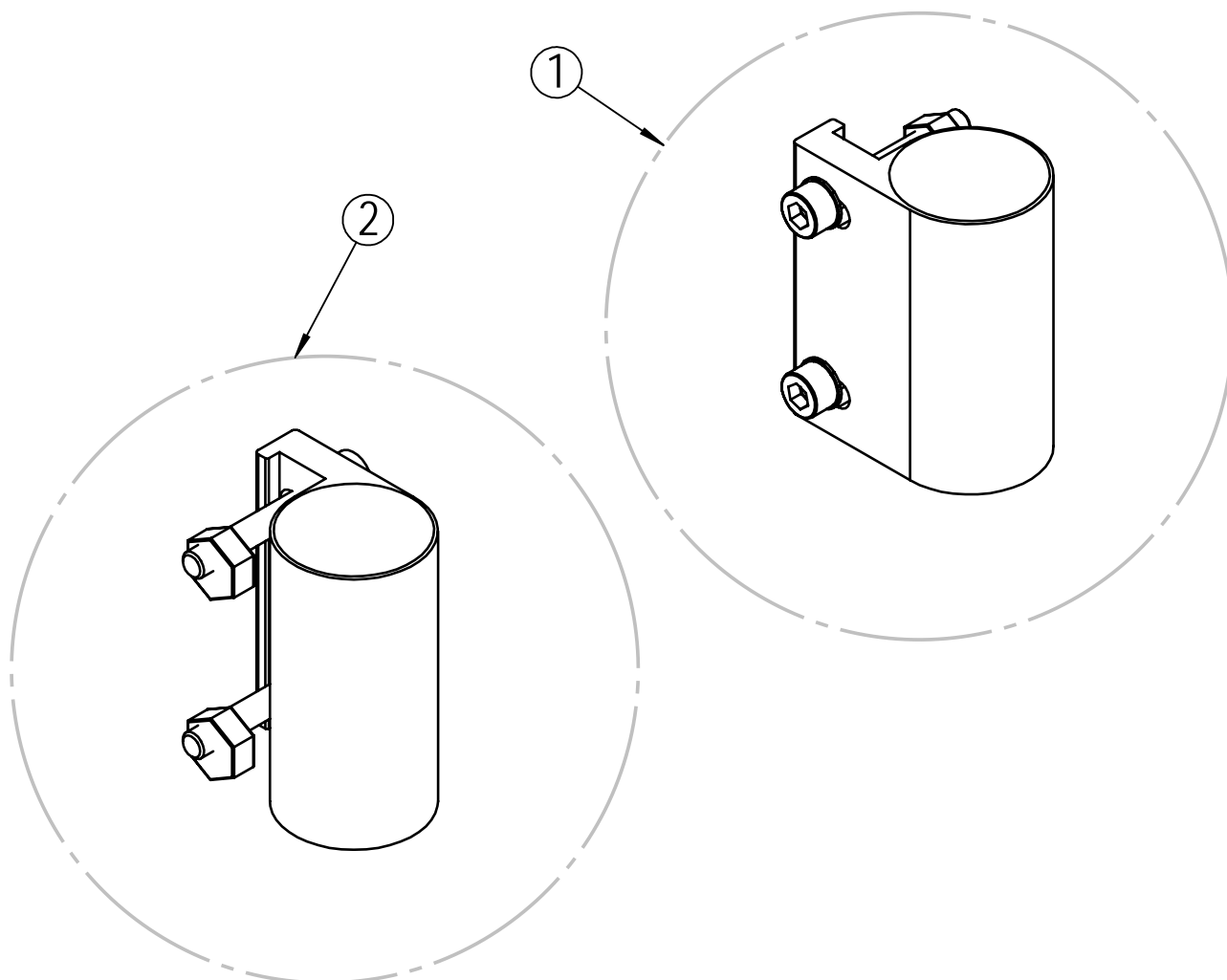

## Bearing house

Position	Description	Item no.	Pcs. pr. chair
1	Bearing house A complete	26445	1
2	Bearing house B complete	26446	1


\* Available upon request

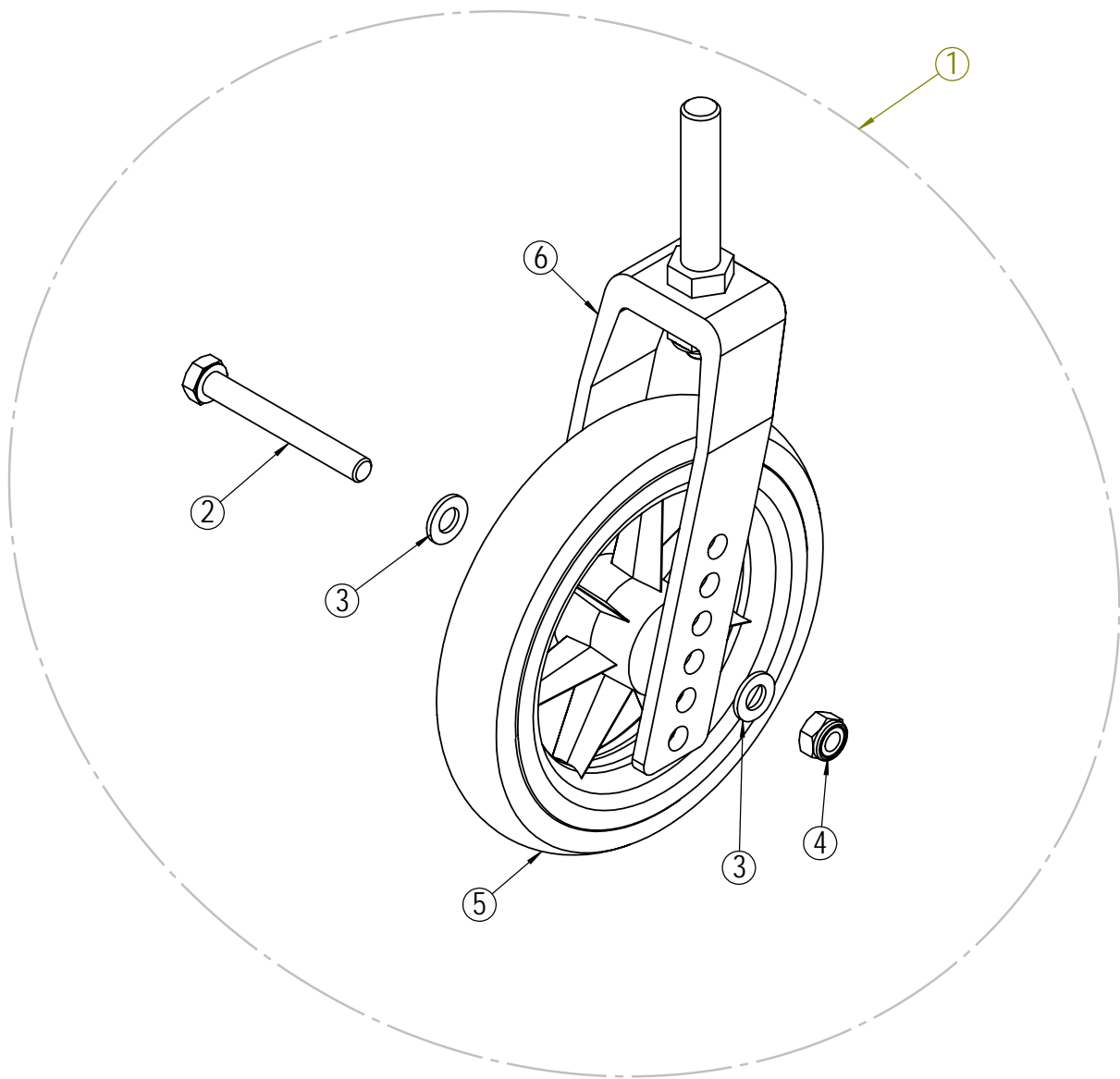

## Front castors


Position	Description	Item no.	Pcs. pr. chair
1	Fork standard with 6" front castors complete	20538	2
1	Fork standard with 7" front castors complete	20535	2
2*	Screw M8x65	20419	2
3*	Washer M8	20068	4
4*	Lock nut M8	20412	2
5	Front castor 6"	20817	2
5	Front castor 7"	20976	2
6	Front castor standard with axle	20429	2

\* Available upon request.

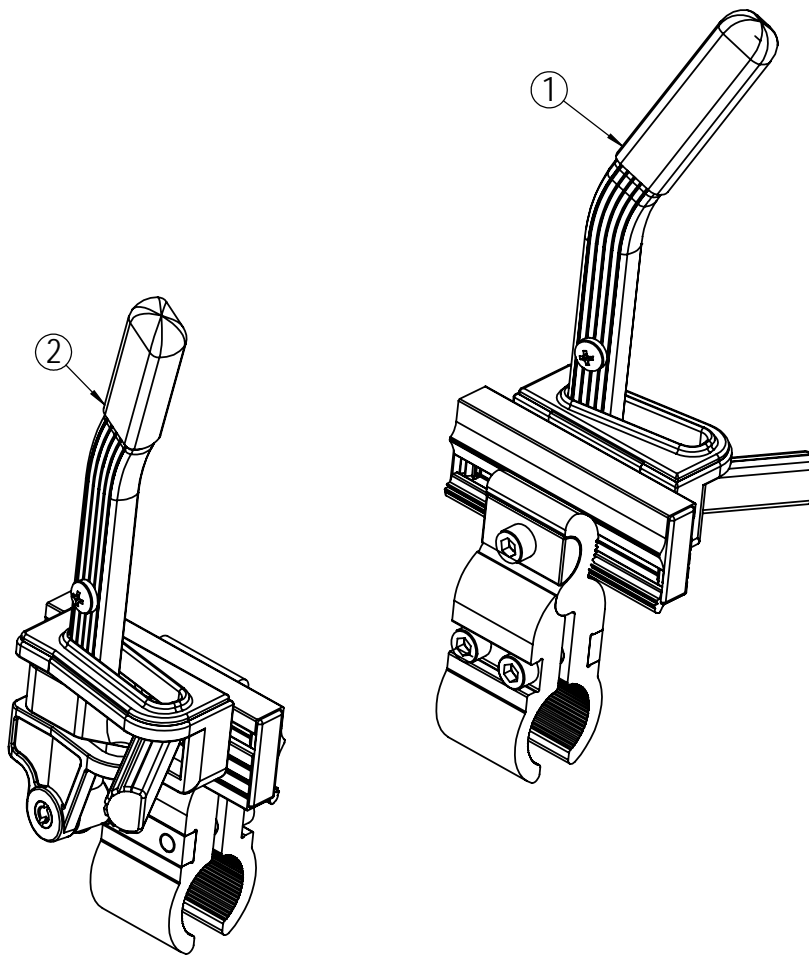


## Brake

Position	Description	Item no.	Pcs. pr. chair
1	Brake Universal Netti I/Svipp Left	26824	1
2	Brake Universal Netti I/Svipp Right	26817	1

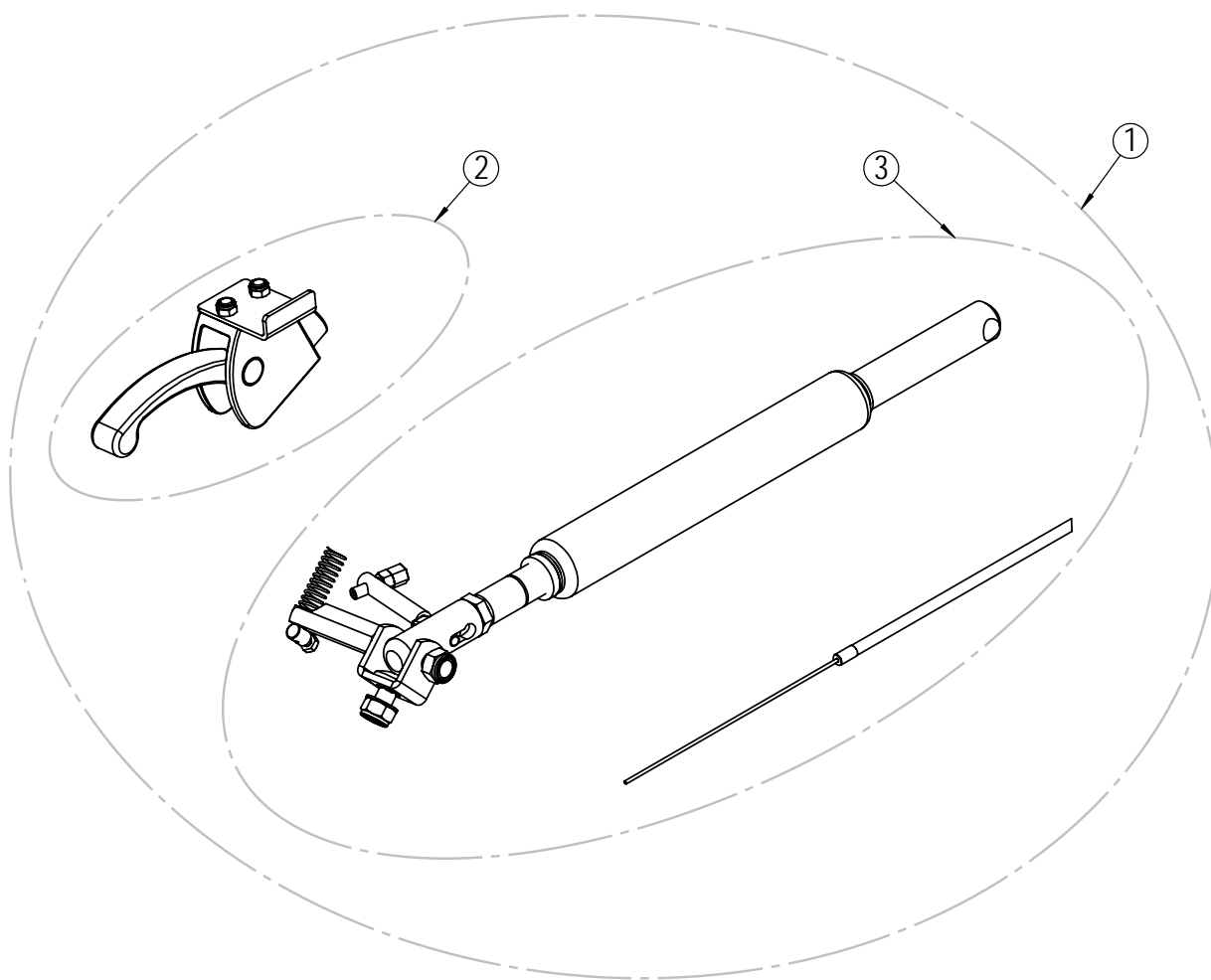


## Gas cylinder - recline

Position	Description	Item no.	Pcs. pr. chair
1	Recline cylinder complete with release handle right	40843	1
1	Recline cylinder complete with release handle left	40844	1
2	Release handle right	26190	1
2	Release handle left	26191	1
3	Recline cylinder with wire complete Netti I	40845	1

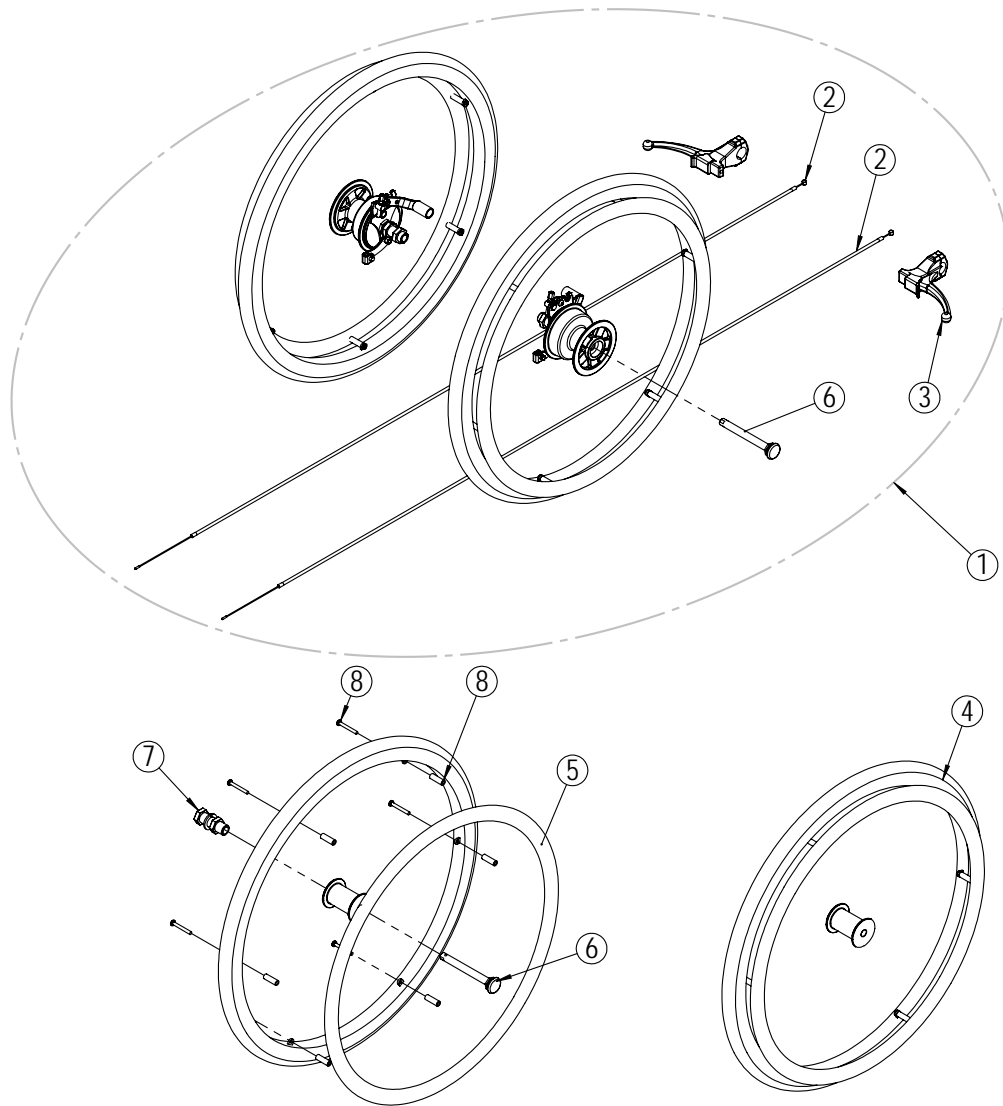

## Main wheels

Position	Description	Item no.	Pcs. pr. chair
1	Main wheel 22" x 1" with drum brake - kit	24365	1
1	Main wheel 24" x 1" with drum brake - kit	24310	1
2	Wire 1250 mm complete	41310	2
3	Brake handle	41313	2
4	Main wheel 22" x 1" with quick release	25371	2
4	Main wheel 24" x 1" with quick release	25372	2
5	Push rim 22" plast	20451	2
5	Push rim 24" plast	20450	2
6	Quick release Ø12 x 98mm	20449	2
7	Wheel bushing complete	20329	2
8	Screw (6 pcs.) & distance sleeves (6 pcs.) - kit	21111	2

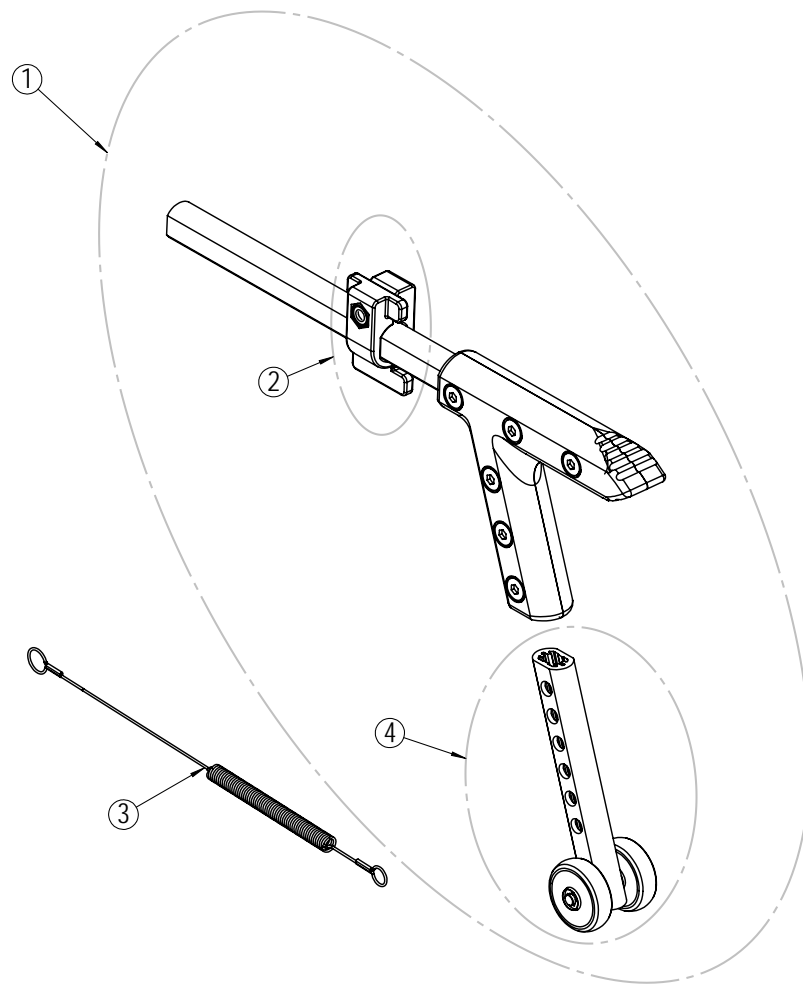

## Anti tip

Position	Description	Item no.	Pcs. pr. chair
1	Anti tip long/short	25419	2
2	Anti tip lock bracket complete	26980	2
3*	Anti tip spring 445mm	24242	2
4*	Anti tip vertical bar with wheels standard	26118	2

\* Available upon request