


SPARE PART CATALOG NETTI II

Version 2/08 -

INDEX


| | Page |
|-------------------------------------------|------|
| Chair frame | 3 |
| Seat cushion - complete | 4 |
| Seat cushion - outer cover | 5 |
| Backrest - fixed angles | 6 |
| Backrest cushion | 7 |
| Push handle | 8 |
| Head support A | 9 |
| Head support B | 10 |
| Head support C | 11 |
| Arm support - left | 12 |
| Arm support - right | 13 |
| Comfort pads arm support | 14 |
| Arm support bracket | 15 |
| Extension piece foot support | 16 |
| Universal foot supports - complete | 17 |
| Universal foot supports - parts | 18 |
| Angle adjustable foot supports - complete | 19 |
| Angle adjustable foot supports - parts | 20 |
| Bearing house | 22 |
| Front castors | 23 |
| Brake | 24 |
| Gas cylinder - recline | 25 |
| Main wheels | 26 |
| Anti tip | 27 |

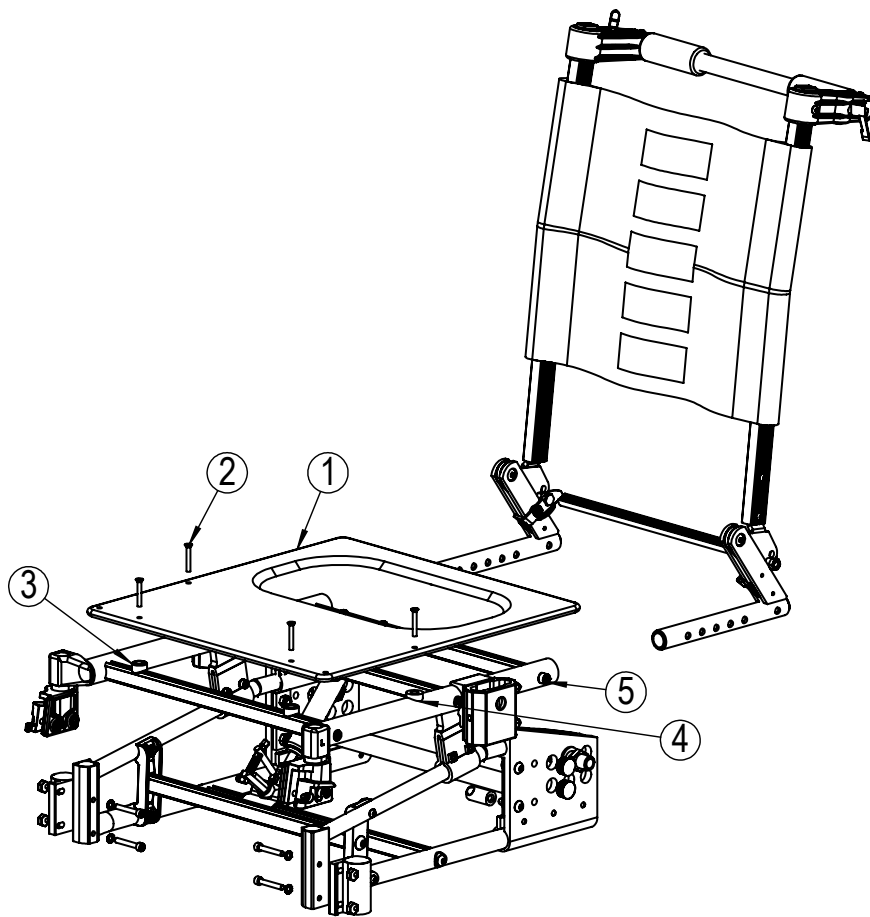

Chair frame


| Position | Description | Item no. | Pcs. pr. chair |
|----------|-------------------------------------|----------|----------------|
| 1 | Seat plate. Width: 35cm Depth: 40cm | 20176 | 1 |
| 1 | Seat plate. Width: 35cm Depth: 43cm | 21040 | 1 |
| 1 | Seat plate. Width: 35cm Depth: 46cm | 20612 | 1 |
| 1 | Seat plate. Width: 38cm Depth: 40cm | 20910 | 1 |
| 1 | Seat plate. Width: 38cm Depth: 43cm | 21041 | 1 |
| 1 | Seat plate. Width: 38cm Depth: 46cm | 20758 | 1 |
| 1 | Seat plate. Width: 40cm Depth: 40cm | 20911 | 1 |
| 1 | Seat plate. Width: 40cm Depth: 43cm | 21042 | 1 |
| 1 | Seat plate. Width: 40cm Depth: 46cm | 20550 | 1 |
| 1 | Seat plate. Width: 43cm Depth: 40cm | 20195 | 1 |
| 1 | Seat plate. Width: 43cm Depth: 43cm | 21043 | 1 |
| 1 | Seat plate. Width: 43cm Depth: 46cm | 20551 | 1 |
| 1 | Seat plate. Width: 45cm Depth: 40cm | 20196 | 1 |
| 1 | Seat plate. Width: 45cm Depth: 43cm | 21044 | 1 |
| 1 | Seat plate. Width: 45cm Depth: 46cm | 20552 | 1 |
| 1 | Seat plate. Width: 50cm Depth: 40cm | 20197 | 1 |
| 1 | Seat plate. Width: 50cm Depth: 43cm | 21045 | 1 |
| 1 | Seat plate. Width: 50cm Depth: 46cm | 20553 | 1 |
| 2* | Screw M5x35 | 22098 | 4 |
| 3* | Frame Seat plate Washer High | 24141 | 2 |
| 4* | Frame Seat plate Washer Low | 22084 | 2 |
| 5* | Screw M8x40 | 21095 | 6 |

* Available upon request.

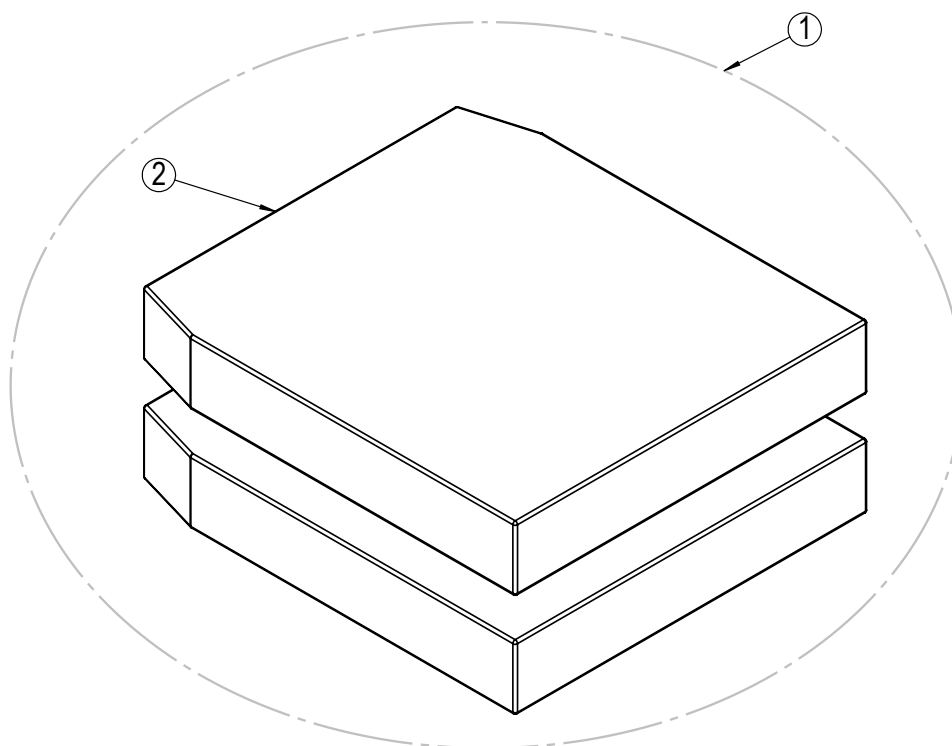


Seat cushion - complete

| Position | Description | Item no. | Pcs. pr. chair |
|----------|----------------------------------------------------|----------|----------------|
| 1 | Seat cushion, Uno Seat Width:35cm Depth:40cm, Mars | 840377 | 1 |
| 1 | Seat cushion, Uno Seat Width:35cm Depth:45cm, Mars | 840378 | 1 |
| 1 | Seat cushion, Uno Seat Width:35cm Depth:50cm, Mars | 840379 | 1 |
| 1 | Seat cushion, Uno Seat Width:38cm Depth:40cm, Mars | 840380 | 1 |
| 1 | Seat cushion, Uno Seat Width:38cm Depth:45cm, Mars | 840381 | 1 |
| 1 | Seat cushion, Uno Seat Width:38cm Depth:50cm, Mars | 840382 | 1 |
| 1 | Seat cushion, Uno Seat Width:40cm Depth:40cm, Mars | 840385 | 1 |
| 1 | Seat cushion, Uno Seat Width:40cm Depth:45cm, Mars | 840386 | 1 |
| 1 | Seat cushion, Uno Seat Width:40cm Depth:50cm, Mars | 840387 | 1 |
| 1 | Seat cushion, Uno Seat Width:43cm Depth:40cm, Mars | 840390 | 1 |
| 1 | Seat cushion, Uno Seat Width:43cm Depth:45cm, Mars | 840391 | 1 |
| 1 | Seat cushion, Uno Seat Width:43cm Depth:50cm, Mars | 840392 | 1 |
| 1 | Seat cushion, Uno Seat Width:45cm Depth:40cm, Mars | 840395 | 1 |
| 1 | Seat cushion, Uno Seat Width:45cm Depth:45cm, Mars | 840396 | 1 |
| 1 | Seat cushion, Uno Seat Width:45cm Depth:50cm, Mars | 840397 | 1 |
| 1 | Seat cushion, Uno Seat Width:50cm Depth:40cm, Mars | 840400 | 1 |
| 1 | Seat cushion, Uno Seat Width:50cm Depth:45cm, Mars | 840401 | 1 |
| 1 | Seat cushion, Uno Seat Width:50cm Depth:50cm, Mars | 840402 | 1 |

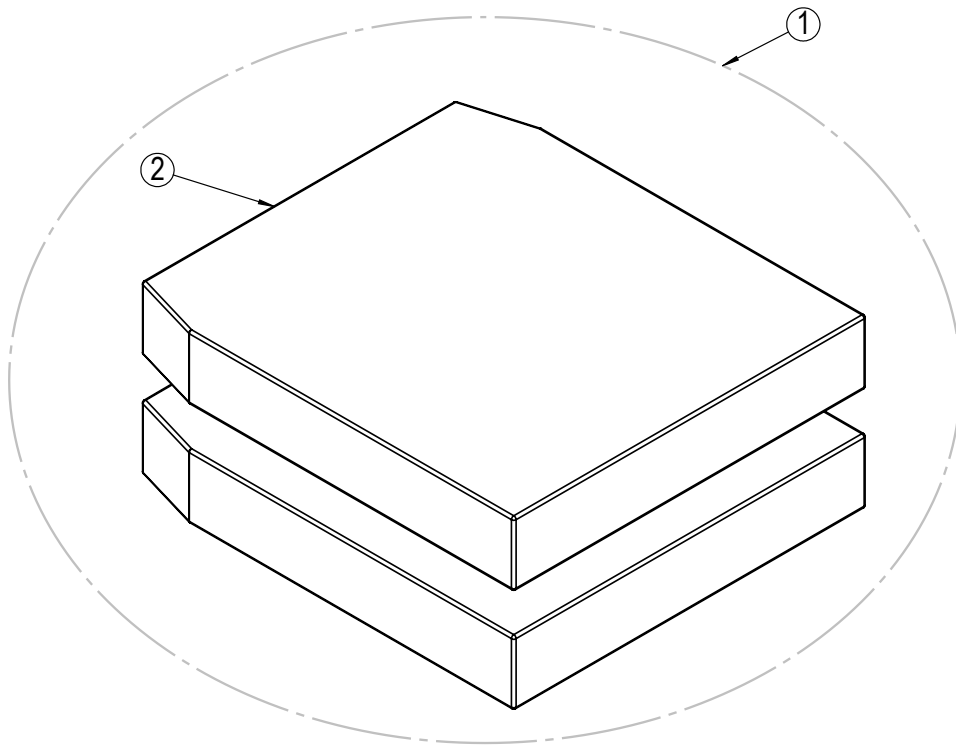

Seat cushion - outer cover


| Position | Description | Item no. | Pcs. pr. chair |
|----------|---------------------------------------------------|----------|----------------|
| 2 | Outer cover, Uno Seat Width:35cm Depth:40cm, Mars | 3543540 | 1 |
| 2 | Outer cover, Uno Seat Width:35cm Depth:45cm, Mars | 3543545 | 1 |
| 2 | Outer cover, Uno Seat Width:35cm Depth:50cm, Mars | 3543550 | 1 |
| 2 | Outer cover, Uno Seat Width:38cm Depth:40cm, Mars | 3543840 | 1 |
| 2 | Outer cover, Uno Seat Width:38cm Depth:45cm, Mars | 3543845 | 1 |
| 2 | Outer cover, Uno Seat Width:38cm Depth:50cm, Mars | 3543850 | 1 |
| 2 | Outer cover, Uno Seat Width:40cm Depth:40cm, Mars | 3544040 | 1 |
| 2 | Outer cover, Uno Seat Width:40cm Depth:45cm, Mars | 3544045 | 1 |
| 2 | Outer cover, Uno Seat Width:40cm Depth:50cm, Mars | 3544050 | 1 |
| 2 | Outer cover, Uno Seat Width:43cm Depth:40cm, Mars | 3544340 | 1 |
| 2 | Outer cover, Uno Seat Width:43cm Depth:45cm, Mars | 3544345 | 1 |
| 2 | Outer cover, Uno Seat Width:43cm Depth:50cm, Mars | 3544550 | 1 |
| 2 | Outer cover, Uno Seat Width:45cm Depth:40cm, Mars | 3544540 | 1 |
| 2 | Outer cover, Uno Seat Width:45cm Depth:45cm, Mars | 3544545 | 1 |
| 2 | Outer cover, Uno Seat Width:45cm Depth:50cm, Mars | 3544550 | 1 |
| 2 | Outer cover, Uno Seat Width:50cm Depth:40cm, Mars | 3545040 | 1 |
| 2 | Outer cover, Uno Seat Width:50cm Depth:45cm, Mars | 3545045 | 1 |
| 2 | Outer cover, Uno Seat Width:50cm Depth:50cm, Mars | 3545050 | 1 |

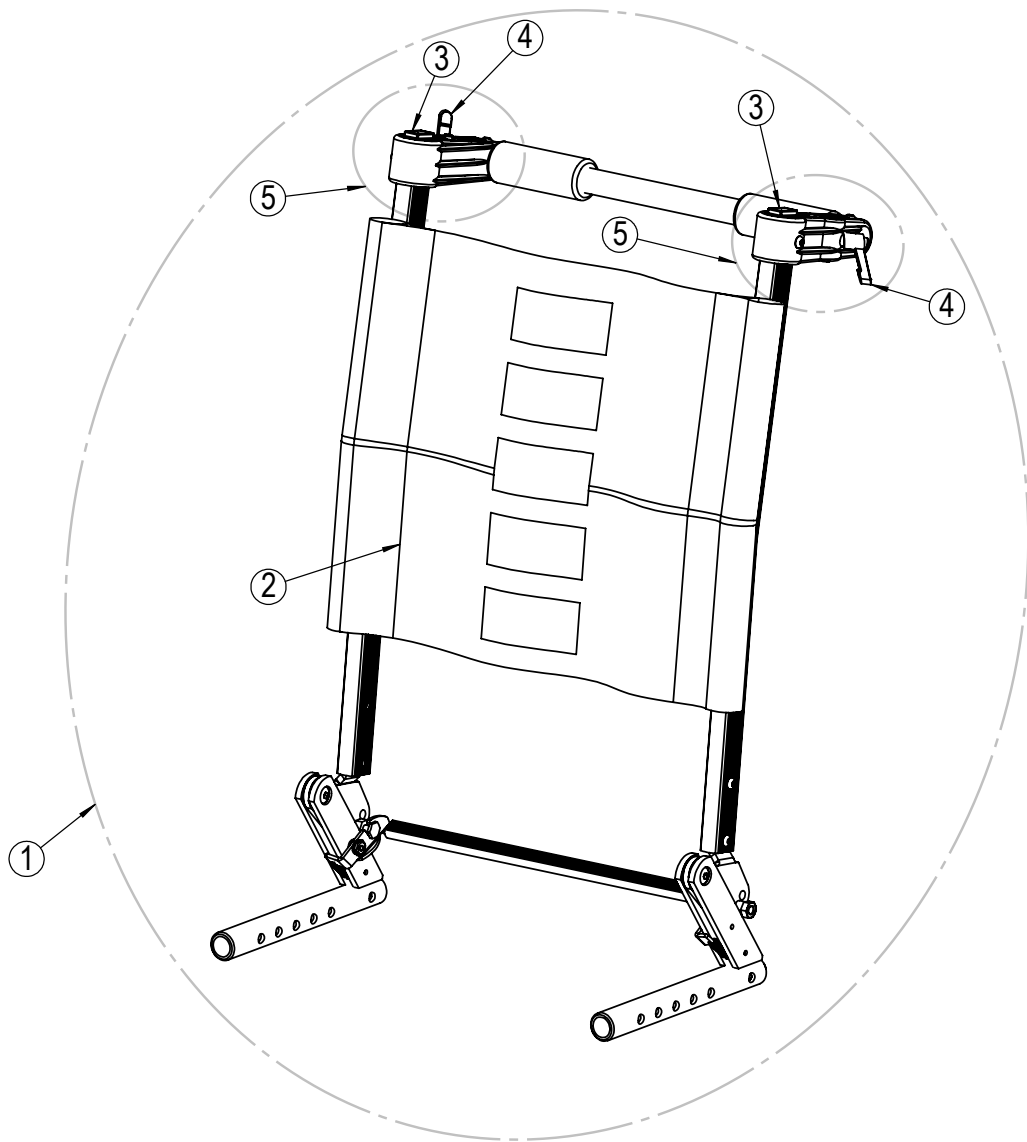

Backrest - fixed angles


| Position | Description | Item no. | Pcs. pr. chair |
|----------|-----------------------------------------|----------|----------------|
| 1 | Backrest 35cm fixed Netti I/II complete | 21892 | 1 |
| 1 | Backrest 38cm fixed Netti I/II complete | 21893 | 1 |
| 1 | Backrest 40cm fixed Netti I/II complete | 21894 | 1 |
| 1 | Backrest 43cm fixed Netti I/II complete | 21895 | 1 |
| 1 | Backrest 45cm fixed Netti I/II complete | 21896 | 1 |
| 1 | Backrest 50cm fixed Netti I/II complete | 21897 | 1 |
| 2 | Velcro 35-55cm | 20699 | 1 |
| 3* | Square plug 25x15 | 21847 | 2 |
| 4* | Clamp lever M6x16 | 20097 | 2 |
| 5* | Backrest push bar bracket complete | 26051 | 2 |

* Available upon request

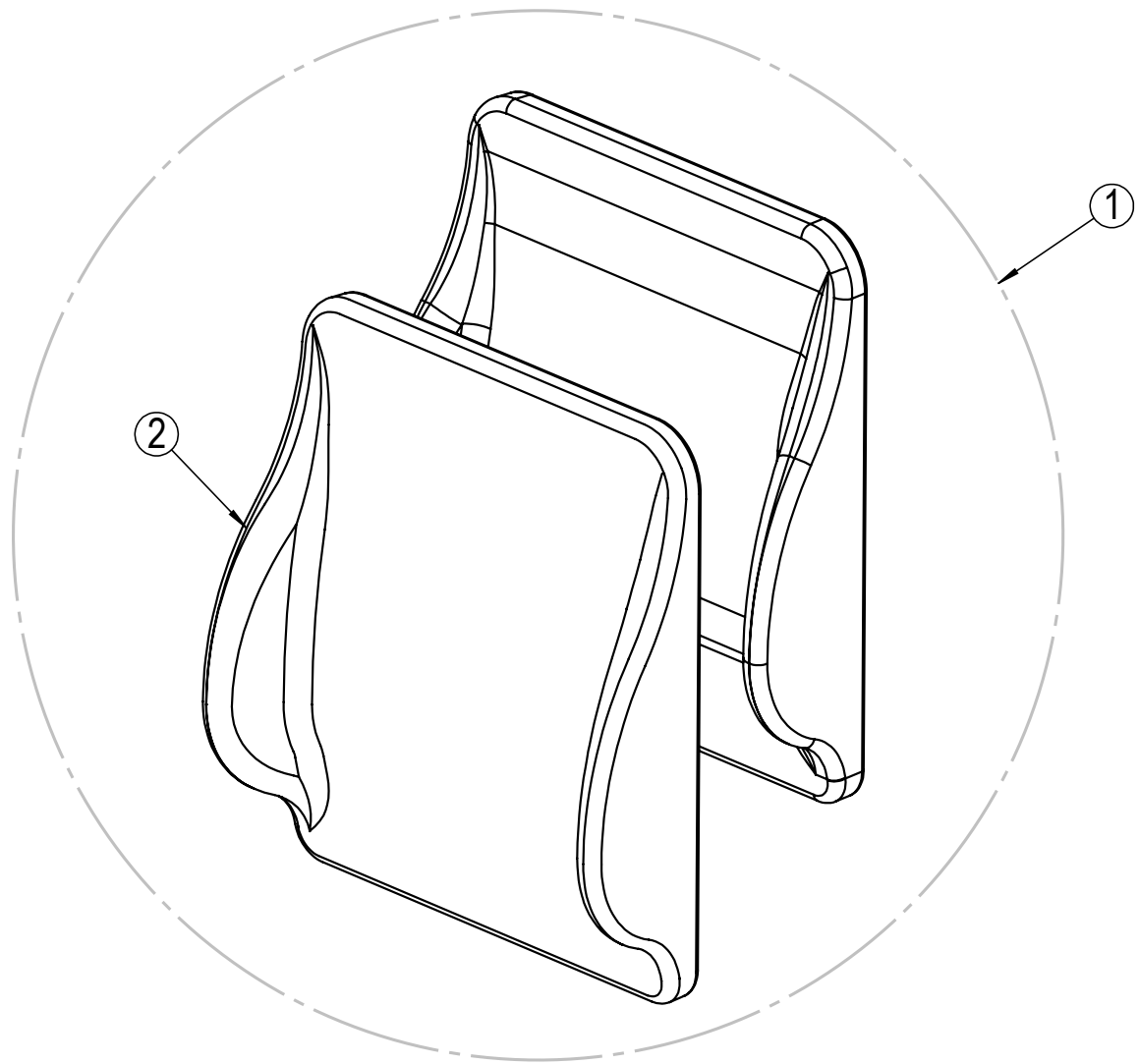

Backrest cushion


| Position | Description | Item no. | Pcs. pr. chair |
|----------|---------------------------------------------------------|----------|----------------|
| 1 | Backrest cushion, Uno Back Width:35cm Height:50cm, Mars | 3735035 | 1 |
| 1 | Backrest cushion, Uno Back Width:38cm Height:50cm, Mars | 3735038 | 1 |
| 1 | Backrest cushion, Uno Back Width:40cm Height:50cm, Mars | 3735040 | 1 |
| 1 | Backrest cushion, Uno Back Width:43cm Height:50cm, Mars | 3735043 | 1 |
| 1 | Backrest cushion, Uno Back Width:45cm Height:50cm, Mars | 3735045 | 1 |
| 1 | Backrest cushion, Uno Back Width:50cm Height:50cm, Mars | 3735050 | 1 |
| 2 | Outer cover, Uno Back Width:35cm Height:50cm, Mars | 3745035 | 1 |
| 2 | Outer cover, Uno Back Width:38cm Height:50cm, Mars | 3745038 | 1 |
| 2 | Outer cover, Uno Back Width:40cm Height:50cm, Mars | 3745040 | 1 |
| 2 | Outer cover, Uno Back Width:43cm Height:50cm, Mars | 3745043 | 1 |
| 2 | Outer cover, Uno Back Width:45cm Height:50cm, Mars | 3745045 | 1 |
| 2 | Outer cover, Uno Back Width:50cm Height:50cm, Mars | 3745050 | 1 |

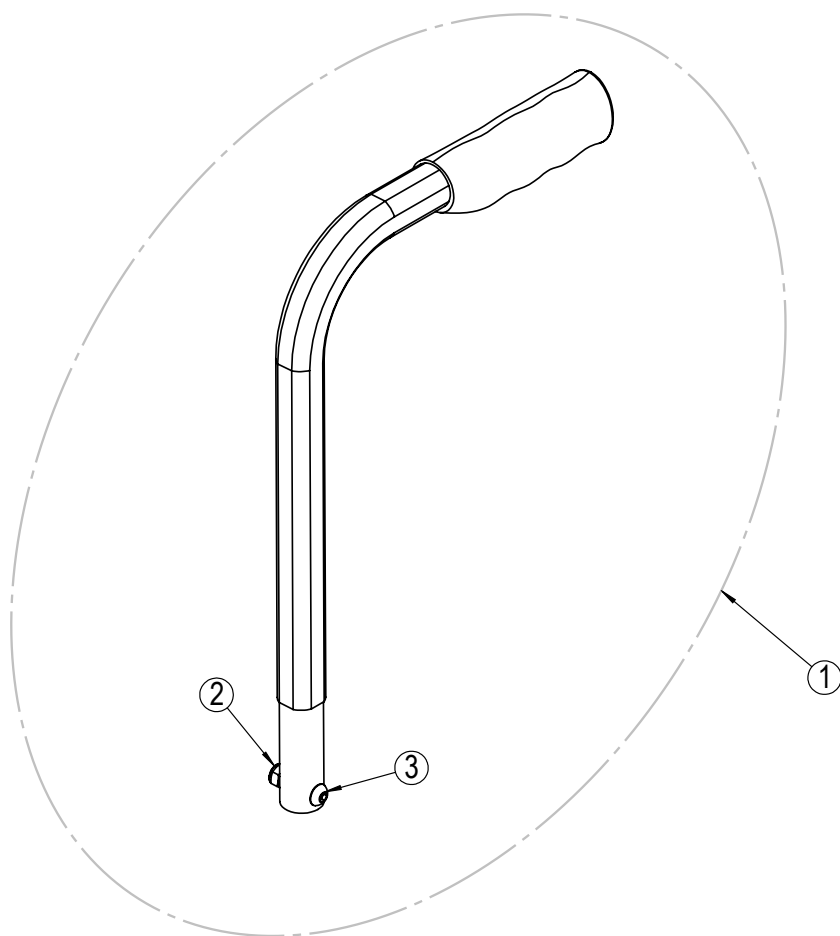

Push handle

| Position | Description | Item no. | Pcs. pr. chair |
|----------|-------------------------------------|----------|----------------|
| 1 | Push handle standard 300mm complete | 21754 | 2 |
| 1 | Push handle long 500mm complete | 21808 | 2 |
| 2* | Lock nut M6 | 20058 | 2 |
| 3* | Screw M6x25 | 20992 | 2 |


* Available upon request

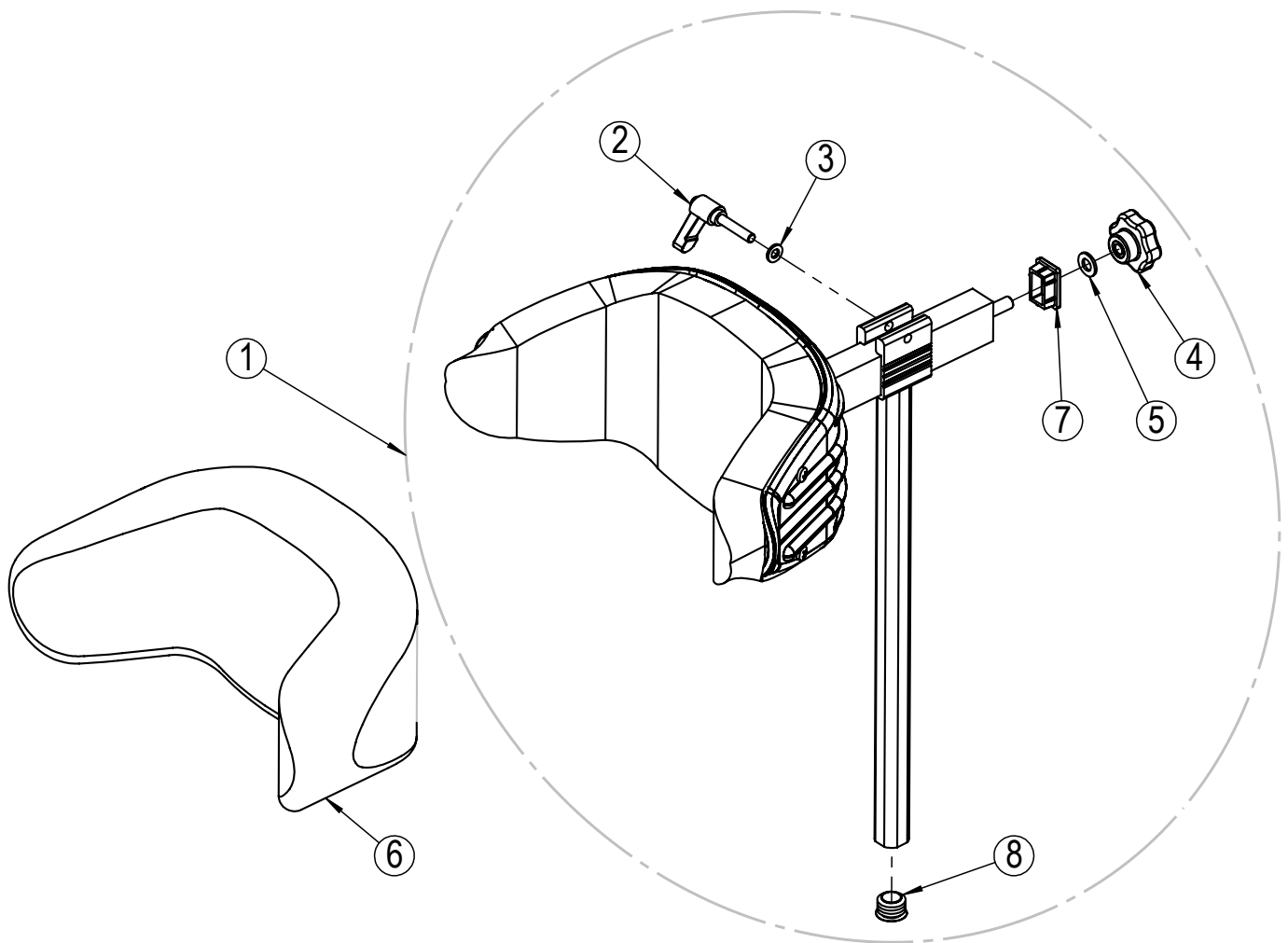

Head support A

| Position | Description | Item no. | Pcs. pr. chair |
|----------|-----------------------------------|----------|----------------|
| 1 | Head support A with Mars | 20575 | 1 |
| 2* | Clamp lever M6x30 | 20282 | 1 |
| 3* | Washer M6 | 20063 | 1 |
| 4* | Star wheel M8 | 26684 | 1 |
| 5* | Washer M8 | 29305 | 1 |
| 6 | Hygiene cover Mars Head support A | 20126 | 1 |
| 7* | Square plug horizontal bar | 20852 | 1 |
| 8* | Plastic plug vertical bar | 20509 | 1 |


* Available upon request.

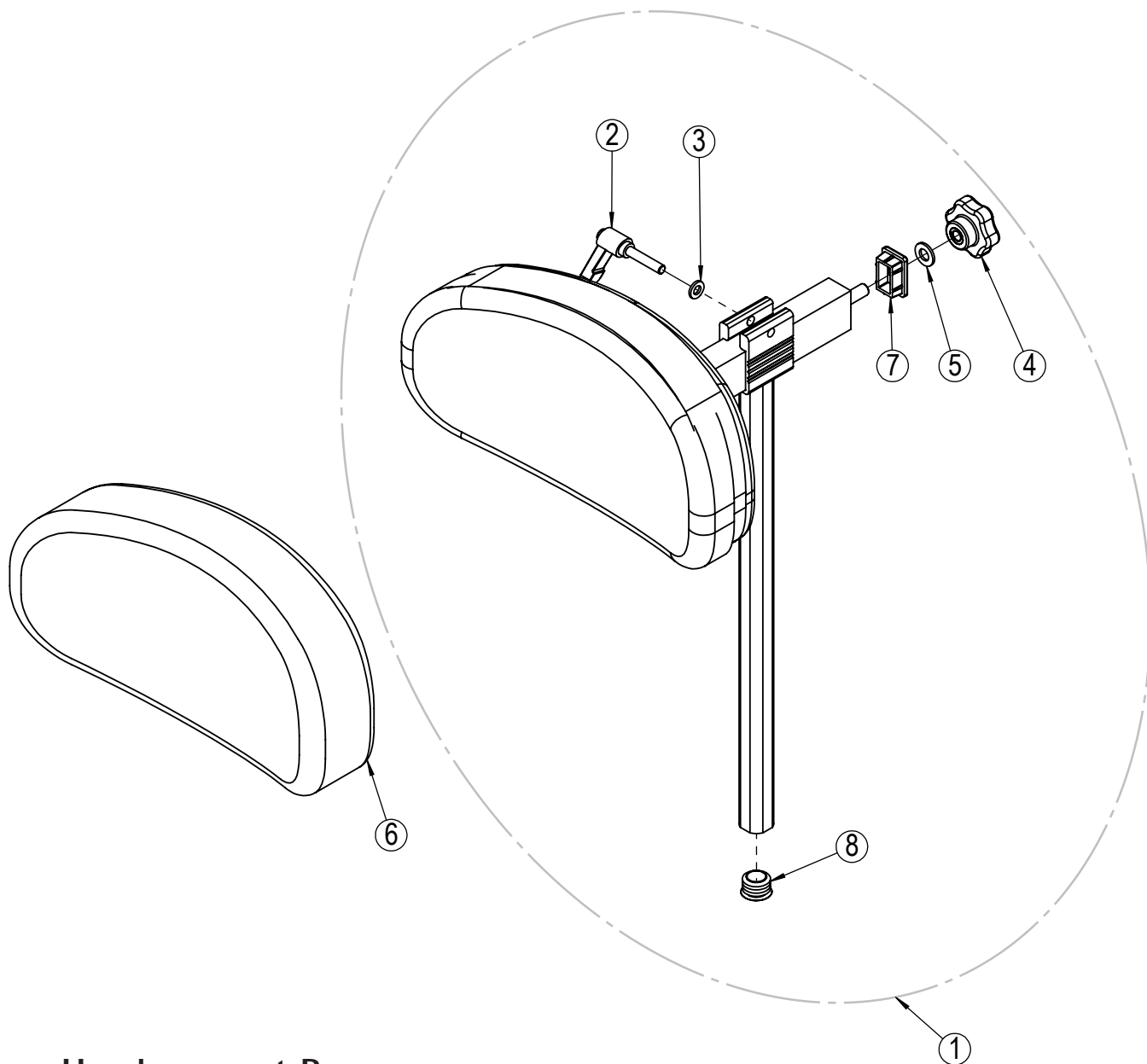

Head support B

| Position | Description | Item no. | Pcs. pr. chair |
|----------|-----------------------------------|----------|----------------|
| 1 | Head support B with Mars | 20574 | 1 |
| 2* | Clamp lever M6x30 | 20282 | 1 |
| 3* | Washer M6 | 20063 | 1 |
| 4* | Star wheel M8 | 26684 | 1 |
| 5* | Washer M8 | 29305 | 1 |
| 6 | Hygiene cover Mars Head support B | 20629 | 1 |
| 7* | Square plug horizontal bar | 20852 | 1 |
| 8* | Plastic plug vertical bar | 20509 | 1 |


* Available upon request.

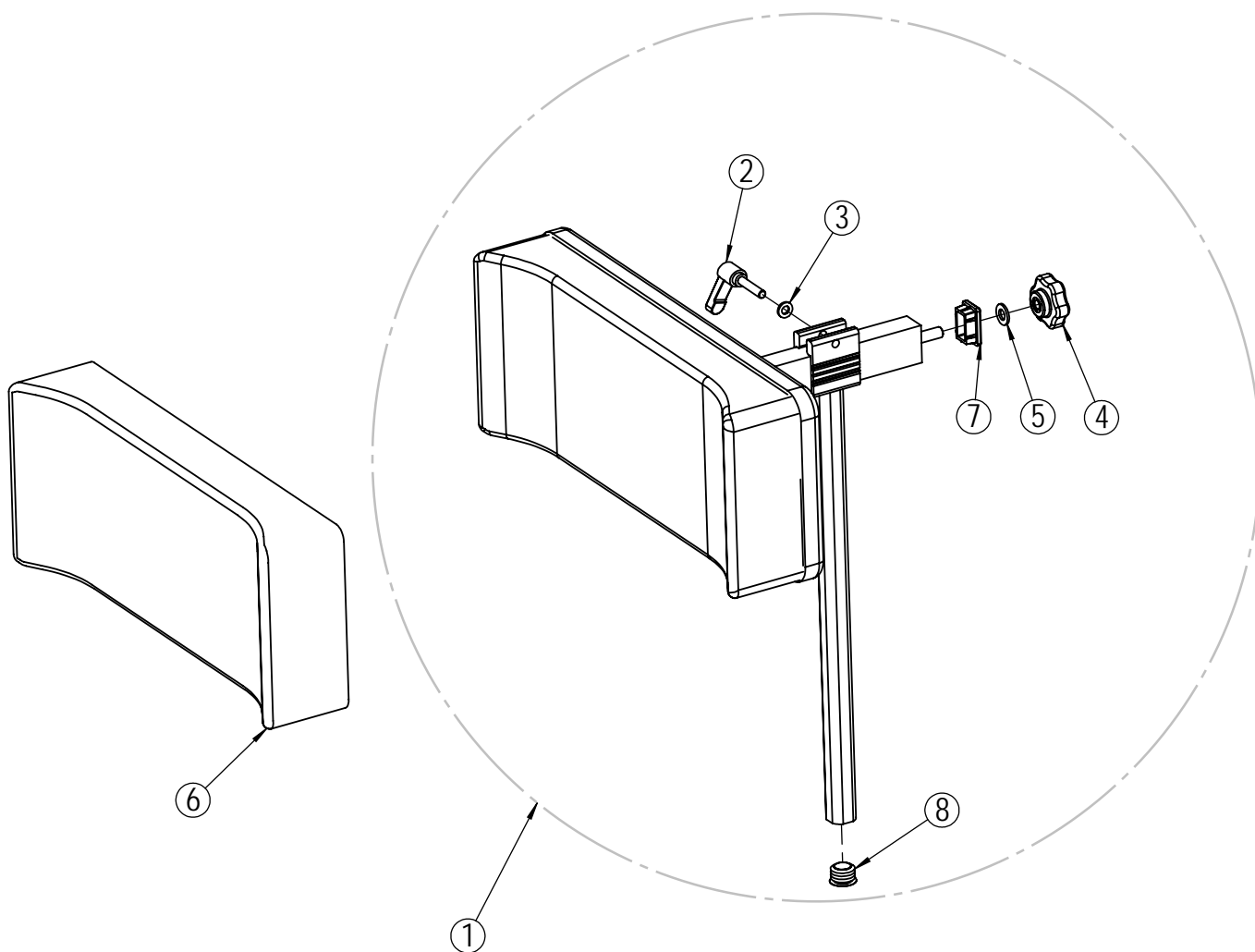

Head support C

| Position | Description | Item no. | Pcs. pr. chair |
|----------|-----------------------------------|----------|----------------|
| 1 | Head support C with Mars | 20573 | 1 |
| 2* | Clamp lever M6x30 | 20282 | 1 |
| 3* | Washer M6 | 20063 | 1 |
| 4* | Star wheel M8 | 26684 | 1 |
| 5* | Washer M8 | 29305 | 1 |
| 6 | Hygiene cover Mars Head support C | 20628 | 1 |
| 7* | Square plug horizontal bar | 20852 | 1 |
| 8* | Plastic plug vertical bar | 20509 | 1 |


* Available upon request.

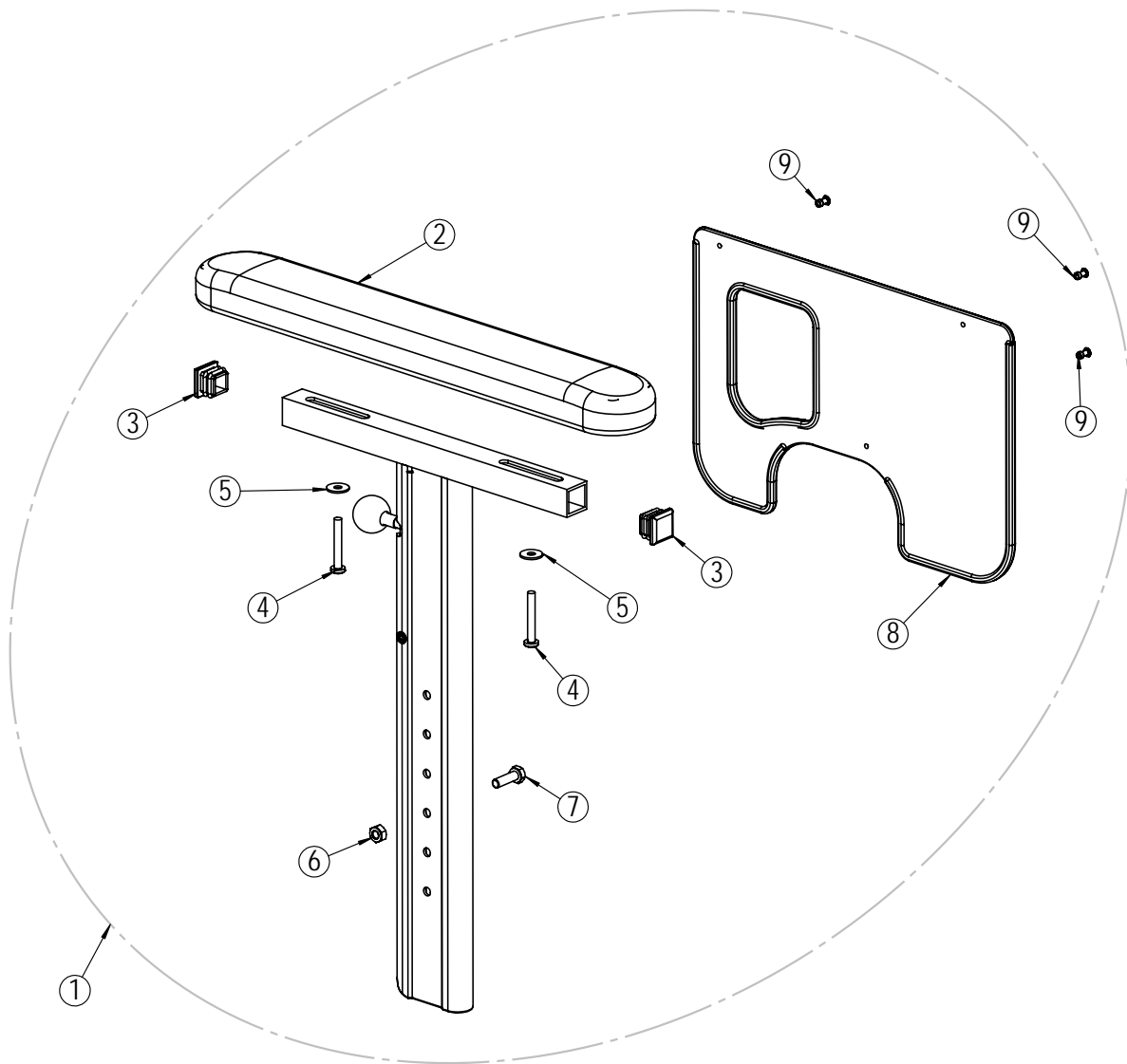

Arm support - left

| Position | Description | Item no. | Pcs. pr. chair |
|----------|---------------------------------------------------|----------|----------------|
| 1 | Arm support left complete | 21681 | 1 |
| 2 | Arm support pad standard 385x58 PU with screws | 20835 | 1 |
| 3* | Square plug 20x20x12 mm | 20633 | 2 |
| 4* | Screw M5x35 | 21323 | 2 |
| 5* | Washer M5 | 23594 | 2 |
| 6* | Screw M6x20 | 20406 | 1 |
| 7* | Nut M6 | 20407 | 1 |
| 8 | Skirt guard arm support | 20239 | 1 |
| 9* | Pop black $\varnothing 3.2 \times 9.7 \text{ mm}$ | 20060 | 3 |


* Available upon request

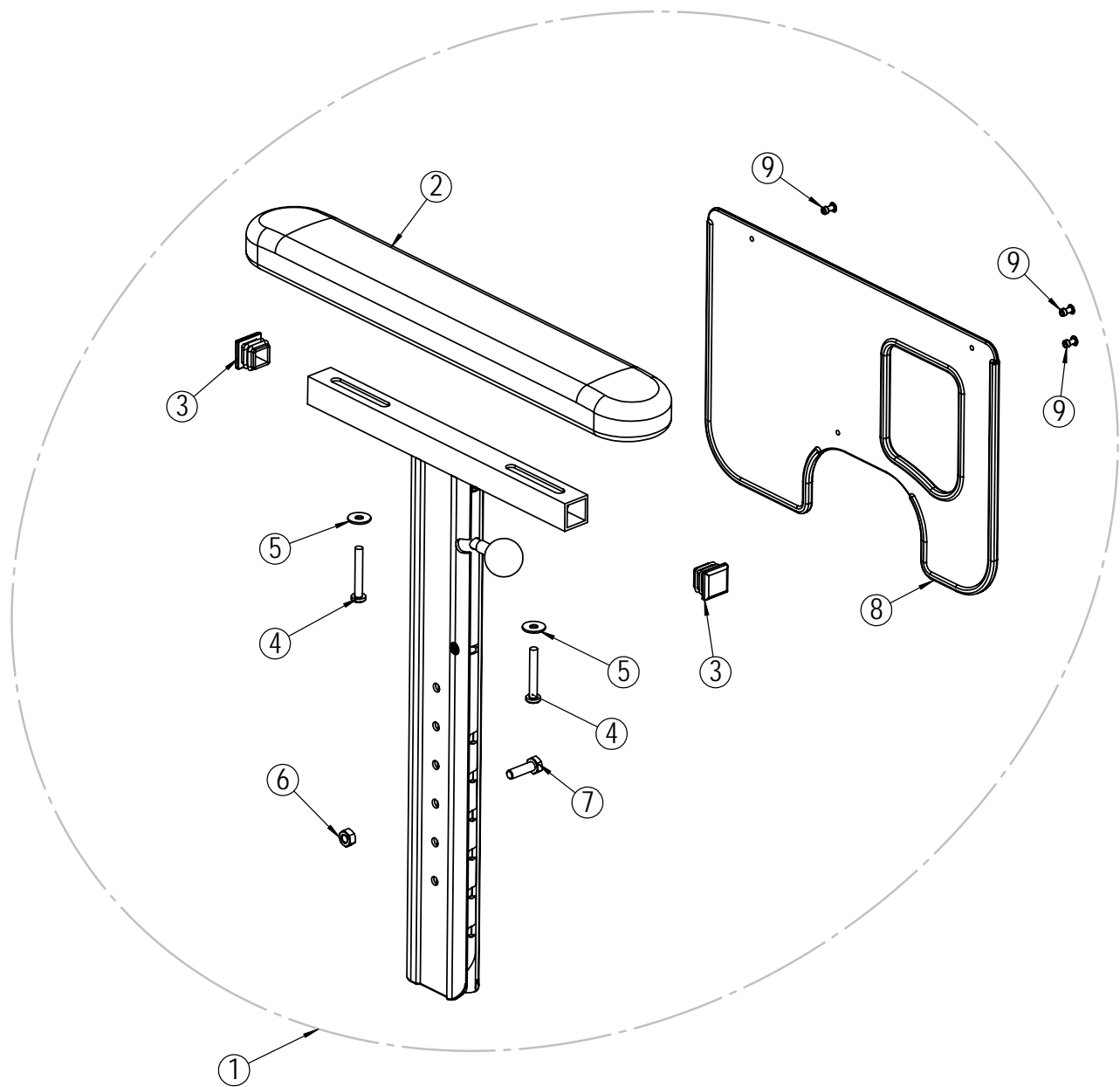

Arm support - right


| Position | Description | Item no. | Pcs. pr. chair |
|----------|---------------------------------------------------|----------|----------------|
| 1 | Arm support right complete | 21682 | 1 |
| 2 | Arm support pad standard 385x58 PU with screws | 20835 | 1 |
| 3* | Square plug 20x20x12 mm | 20633 | 2 |
| 4* | Screw M5x35 | 21323 | 2 |
| 5* | Washer M5 | 23594 | 2 |
| 6* | Screw M6x20 | 20406 | 1 |
| 7* | Nut M6 | 20407 | 1 |
| 8 | Skirt guard arm support | 20239 | 1 |
| 9* | Pop black $\varnothing 3.2 \times 9.7 \text{ mm}$ | 20060 | 3 |

* Available upon request

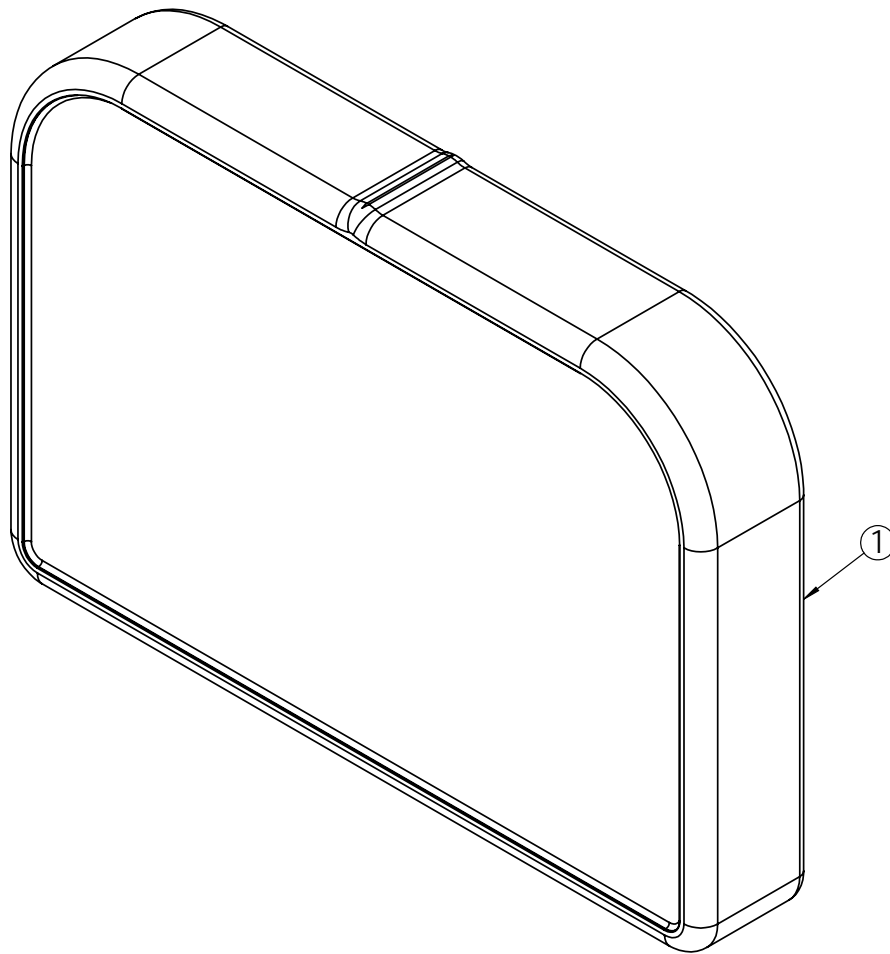

Comfort pads arm support


| Position | Description | Item no. | Pcs. pr. chair |
|----------|---------------------|----------|----------------|
| 1 | Comfort pads. 2,5cm | 22149 | 2 |
| 1 | Comfort pads. 3,5cm | 22150 | 2 |
| 1 | Comfort pads. 3,5cm | 22151 | 2 |

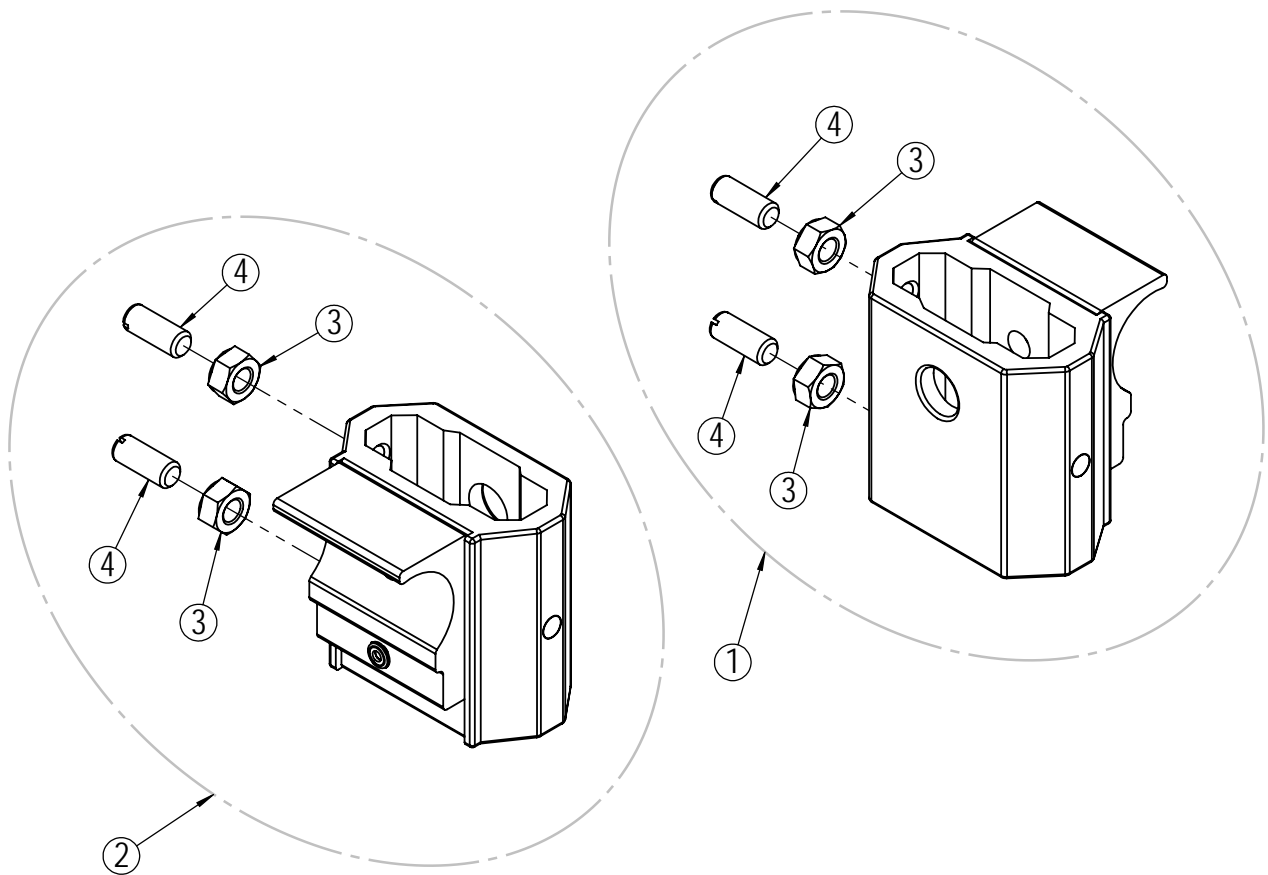

Arm support bracket


| Position | Description | Item no. | Pcs. pr. chair |
|----------|------------------------------------|----------|----------------|
| 1 | Arm support bracket right complete | 20404 | 1 |
| 2 | Arm support bracket left complete | 20401 | 1 |
| 3* | Nut M8 | 20408 | 4 |
| 4* | Nylon screw M8x20 | 20857 | 4 |

* Available upon request

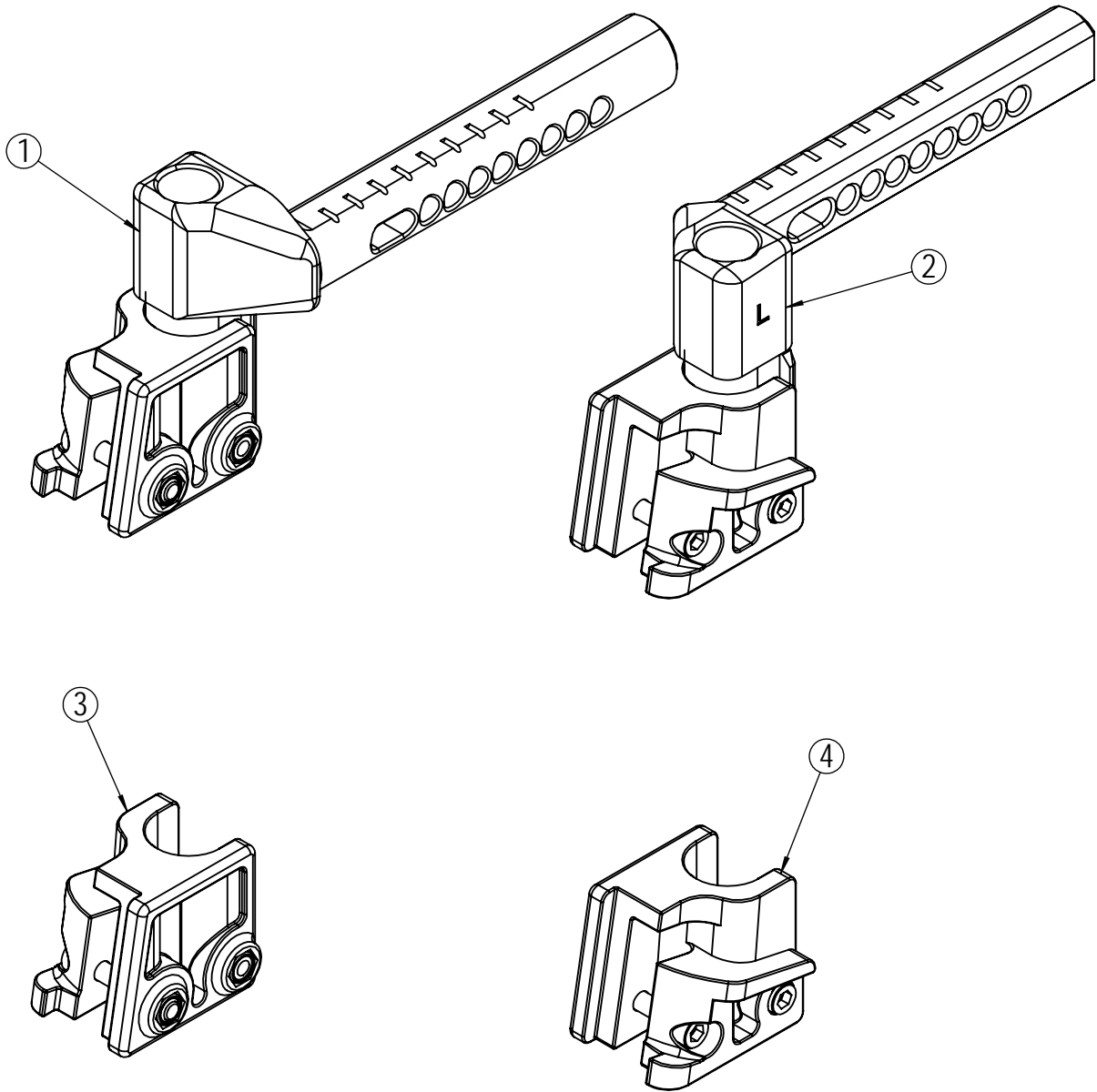


Extension piece - foot support

| Position | Description | Item no. | Pcs. pr. chair |
|----------|-------------------------------------------|----------|----------------|
| 1 | Extension piece. Silver. Right. Complete. | 27384 | 1 |
| 2 | Extension piece. Silver. Left. Complete. | 27385 | 1 |
| 3 | Locking bracket with screw. Right. | 29555 | 1 |
| 4 | Locking bracket with screw. Left. | 29556 | 1 |

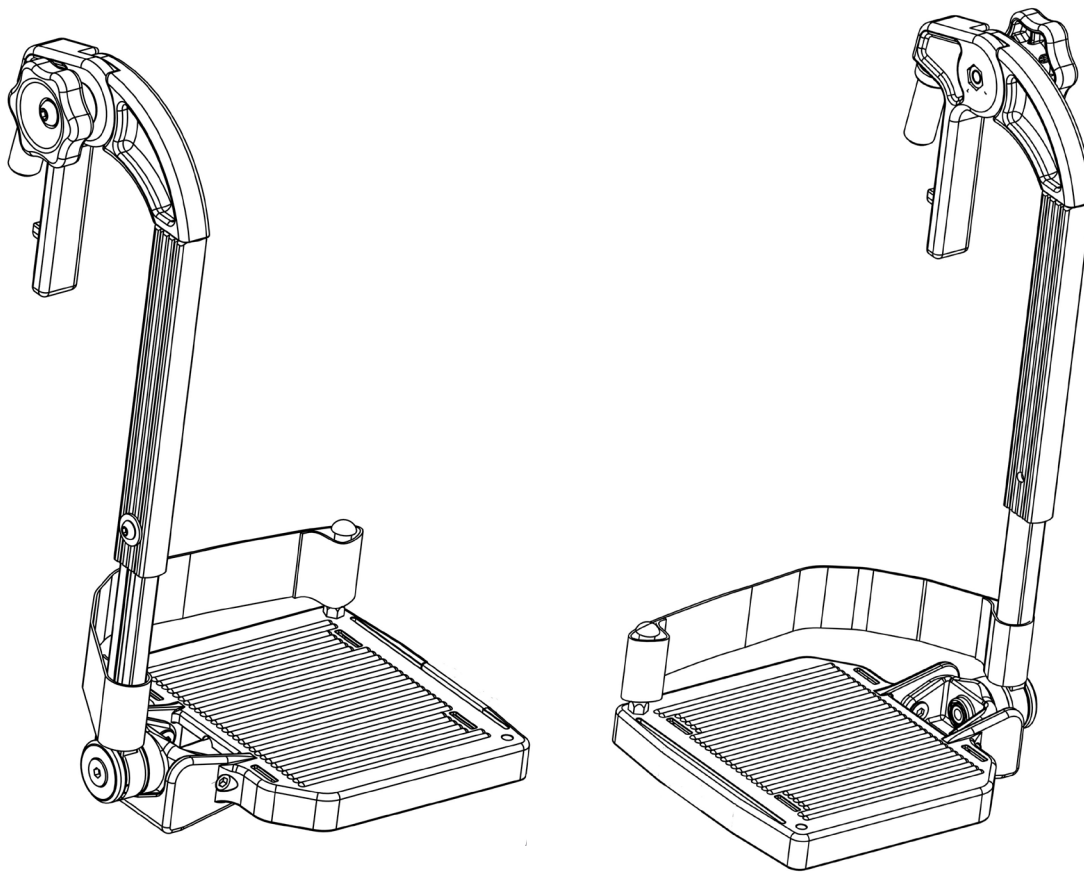


Universal foot supports - complete

| Position | Description | Item no. | Pcs. pr. chair |
|----------|----------------------------------------------|----------|----------------|
| 1 | Universal foot support. Silver. 35cm. Right. | 27220 | 1 |
| 2 | Universal foot support. Silver. 35cm Left. | 27221 | 1 |
| 1 | Universal foot support. Silver. 38cm. Right. | 27222 | 1 |
| 2 | Universal foot support. Silver. 38cm Left. | 27223 | 1 |
| 1 | Universal foot support. Silver. 40m. Right. | 27224 | 1 |
| 2 | Universal foot support. Silver. 40cm Left. | 27225 | 1 |
| 1 | Universal foot support. Silver. 43cm. Right. | 27226 | 1 |
| 2 | Universal foot support. Silver. 43cm Left. | 27227 | 1 |
| 1 | Universal foot support. Silver. 45cm. Right. | 27228 | 1 |
| 2 | Universal foot support. Silver. 45cm Left. | 27229 | 1 |
| 1 | Universal foot support. Silver. 50cm. Right. | 27230 | 1 |
| 2 | Universal foot support. Silver. 50cm Left. | 27231 | 1 |

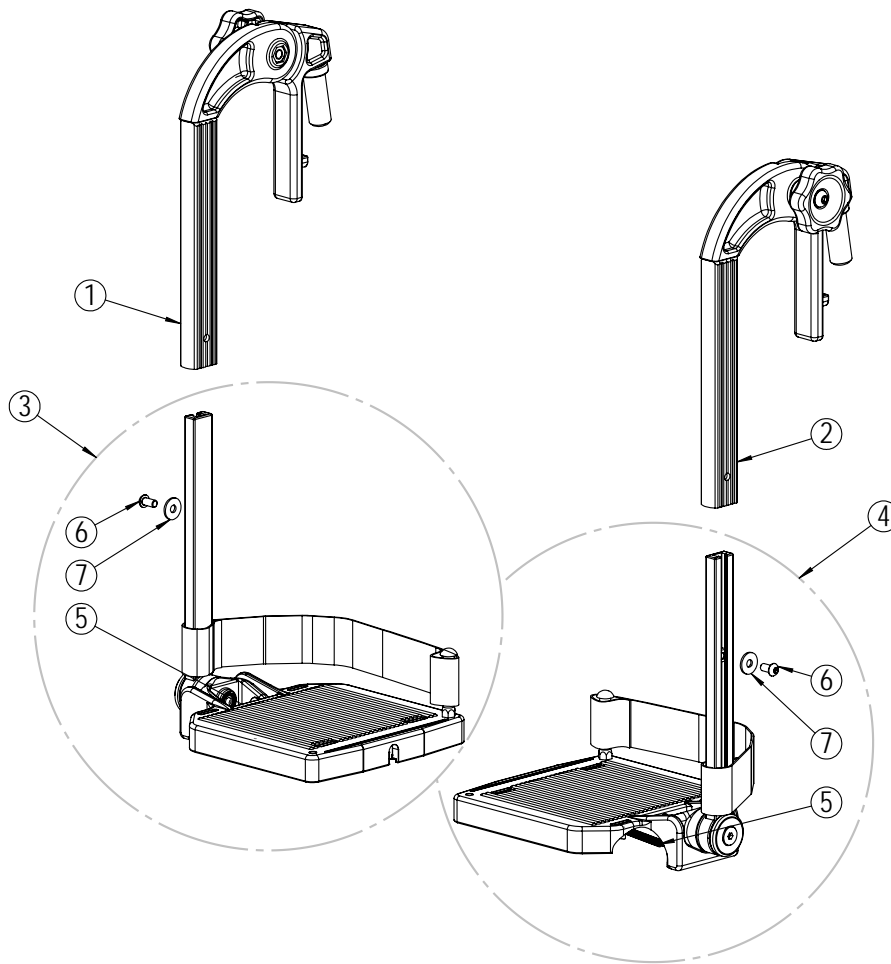

Universal foot supports - parts

| Position | Description | Item no. | Pcs. pr. chair |
|----------|----------------------------------------------|----------|----------------|
| 1 | Universal foot support. Silver. 35cm. Right. | 27220 | 1 |
| 2 | Universal foot support. Silver. 35cm Left. | 27221 | 1 |
| 1 | Universal foot support. Silver. 38cm. Right. | 27222 | 1 |
| 2 | Universal foot support. Silver. 38cm Left. | 27223 | 1 |
| 1 | Universal foot support. Silver. 40m. Right. | 27224 | 1 |
| 2 | Universal foot support. Silver. 40cm Left. | 27225 | 1 |
| 1 | Universal foot support. Silver. 43cm. Right. | 27226 | 1 |
| 2 | Universal foot support. Silver. 43cm Left. | 27227 | 1 |
| 1 | Universal foot support. Silver. 45cm. Right. | 27228 | 1 |
| 2 | Universal foot support. Silver. 45cm Left. | 27229 | 1 |
| 1 | Universal foot support. Silver. 50cm. Right. | 27230 | 1 |
| 2 | Universal foot support. Silver. 50cm Left. | 27231 | 1 |

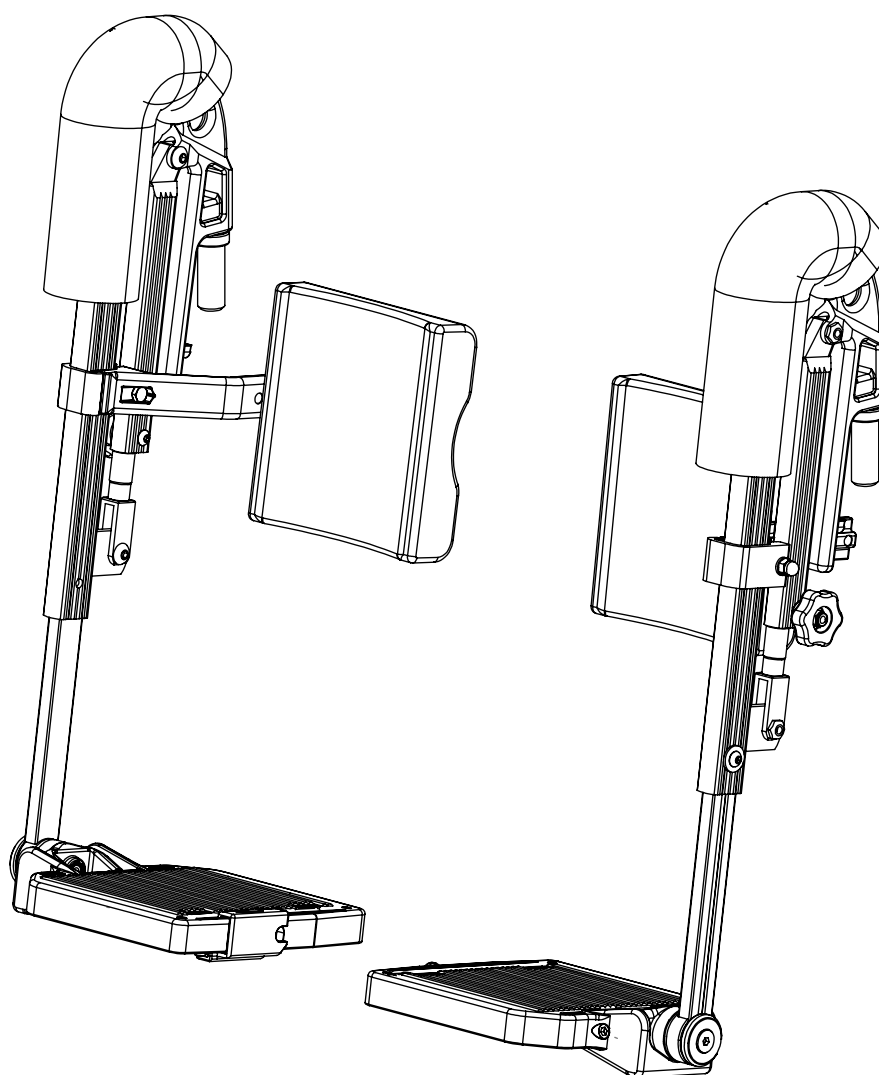


Angle adjustable foot supports - complete


| Position | Description | Item no. | Pcs. pr. chair |
|----------|----------------------------------------------------|----------|----------------|
| 1 | Angle adjustable foot support. Black. 35cm. Right. | 29500 | 1 |
| 2 | Angle adjustable foot support. Black. 35cm. Left. | 29508 | 1 |
| 1 | Angle adjustable foot support. Black. 40cm. Right. | 29502 | 1 |
| 2 | Angle adjustable foot support. Black. 40cm. Left. | 29510 | 1 |
| 1 | Angle adjustable foot support. Black. 43cm. Right. | 29503 | 1 |
| 2 | Angle adjustable foot support. Black. 43cm. Left. | 29511 | 1 |
| 1 | Angle adjustable foot support. Black. 45cm. Right. | 29504 | 1 |
| 2 | Angle adjustable foot support. Black. 45cm. Left. | 29512 | 1 |
| 1 | Angle adjustable foot support. Black. 50cm. Right. | 29505 | 1 |
| 2 | Angle adjustable foot support. Black. 50cm. Left. | 29513 | 1 |

Angle adjustable foot supports - parts

| Position | Description | Item no. | Pcs. pr. chair |
|----------|-------------------------------------------------------|----------|----------------|
| 3 | Angle adjustable foot support. Black. Right. | 29489 | 1 |
| 4 | Angle adjustable foot support. Black. Left. | 29490 | 1 |
| 5 | Knee upholstery. Mars. | 23864 | 2 |
| 6 | Calf pad with bracket. 35cm | 27964 | 1 |
| 6 | Calf pad with bracket. 40cm | 27965 | 1 |
| 6 | Calf pad with bracket. 43cm | 27966 | 1 |
| 6 | Calf pad with bracket. 45cm | 27966 | 1 |
| 6 | Calf pad with bracket. 50cm | 27967 | 1 |
| 7 | Foot plate. 35cm Right. | 28621 | 1 |
| 8 | Foot plate. 35cm Left. | 28649 | 1 |
| 7 | Foot plate. 40cm Right. | 28623 | 1 |
| 8 | Foot plate. 40cm Left. | 28651 | 1 |
| 7 | Foot plate. 43cm Right. | 28624 | 1 |
| 8 | Foot plate. 43cm Left. | 28652 | 1 |
| 7 | Foot plate. 45cm Right. | 28625 | 1 |
| 8 | Foot plate. 45cm Left. | 28653 | 1 |
| 7 | Foot plate. 50cm Right. | 28626 | 1 |
| 8 | Foot plate. 50cm Left. | 28654 | 1 |
| 9* | Foot plate spring | 27858 | 2 |
| 10* | Screw M6x12 | 24272 | 2 |
| 11* | Washer | 20356 | 2 |
| 12 | Mechanical adjustment piece with screw. Black. Right. | 29558 | 1 |
| 13 | Mechanical adjustment piece with screw. Black. Left. | 29559 | 1 |
| 14* | Screw M6x40 | 20103 | 2 |
| 15* | Adjustment wheel | 27388 | 2 |
| 16 | Foot plate hinge complete | 28523 | 2 |
| 17 | Calf support bracket with screw, washer and lock nut | 25353 | 2 |
| 18* | Nut M8 | 29761 | 2 |
| 19* | Foot support locking cone | 20539 | 2 |
| 20* | Nut M6 | 20058 | 2 |

* Available upon request.

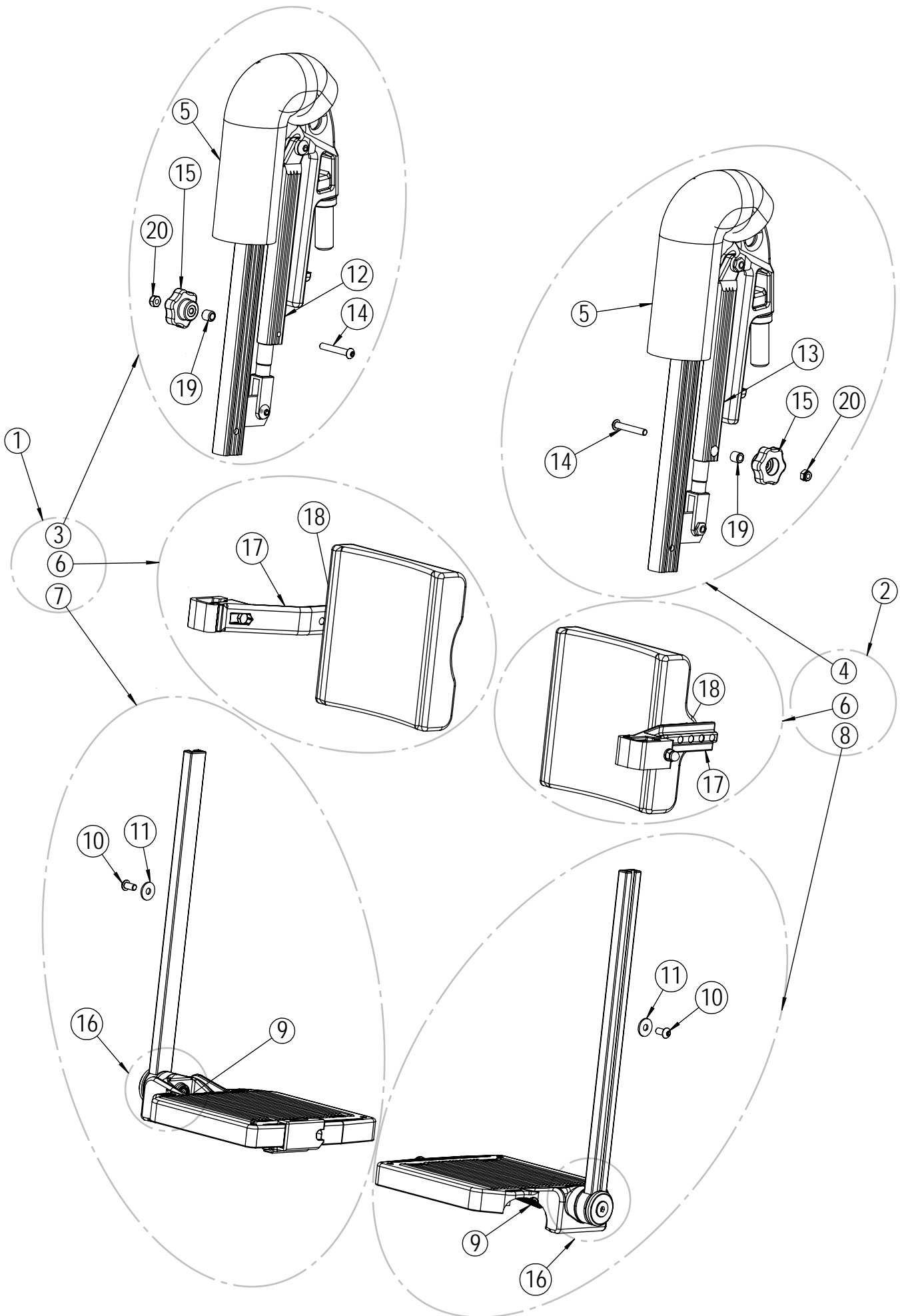


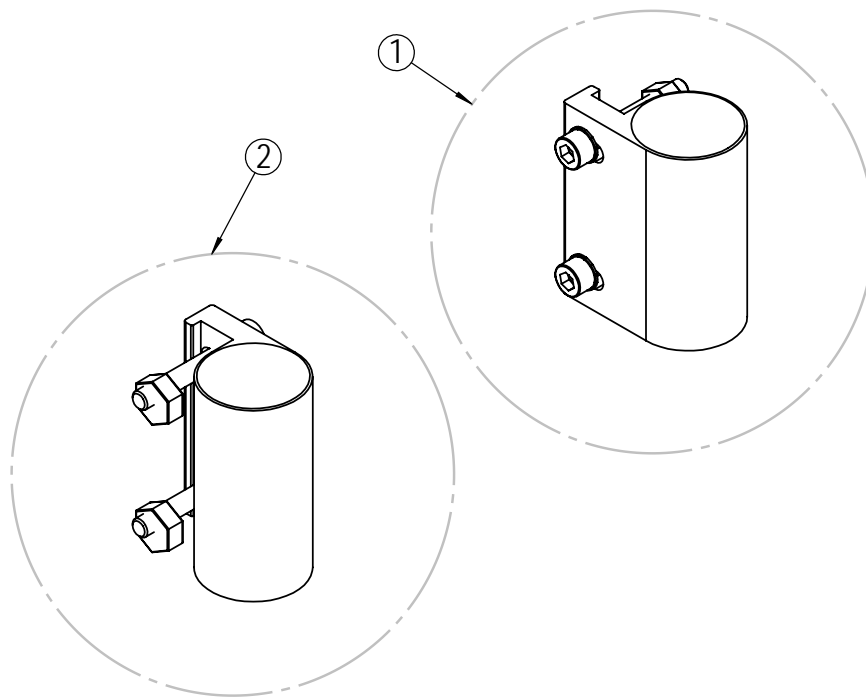

Bearing house

| Position | Description | Item no. | Pcs. pr. chair |
|----------|--------------------------|----------|----------------|
| 1 | Bearing house A complete | 26445 | 1 |
| 2 | Bearing house B complete | 26446 | 1 |



* Available upon request

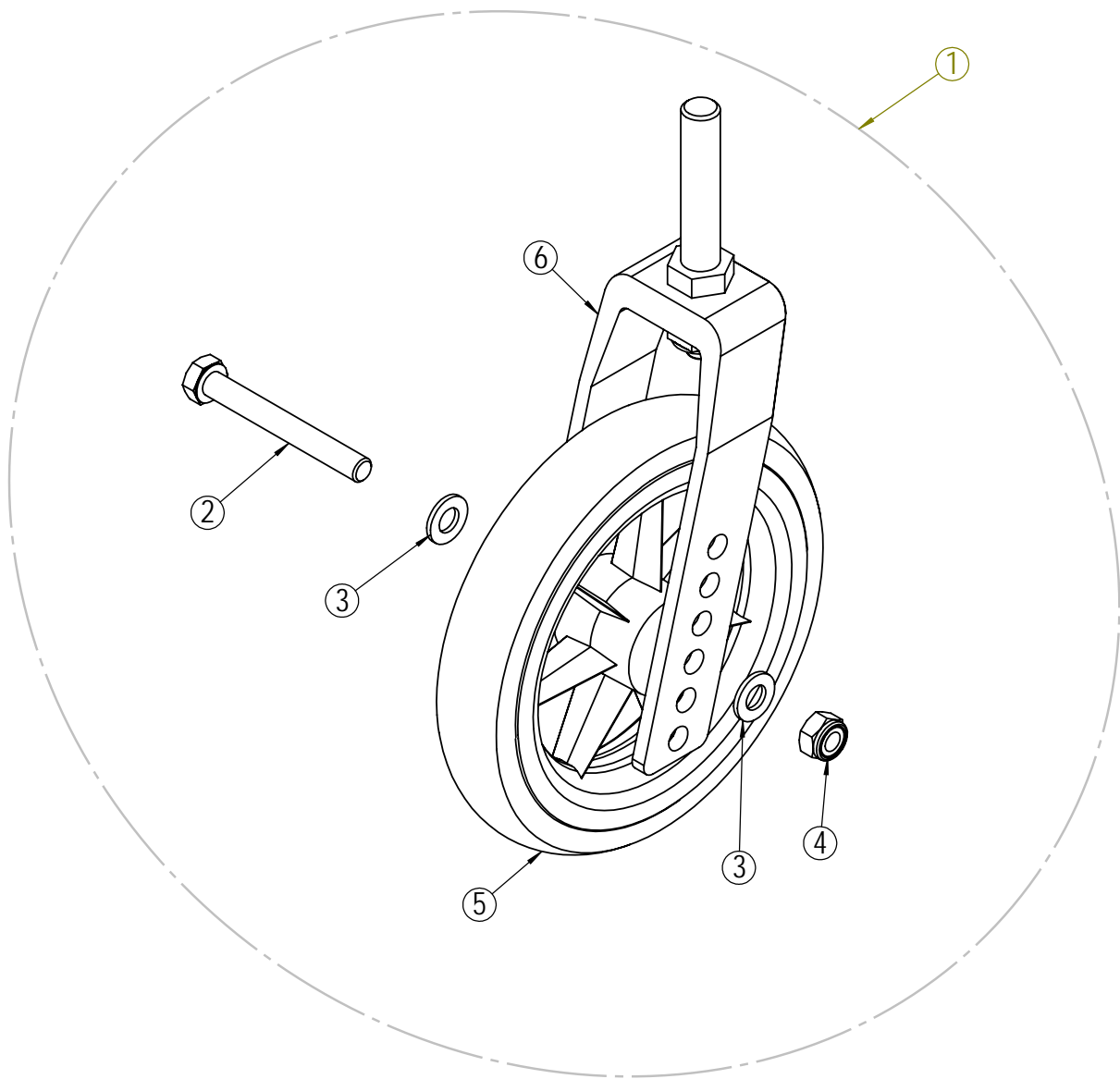

Front castors


| Position | Description | Item no. | Pcs. pr. chair |
|----------|----------------------------------------------|----------|----------------|
| 1 | Fork standard with 6" front castors complete | 20538 | 2 |
| 1 | Fork standard with 7" front castors complete | 20535 | 2 |
| 2* | Screw M8x65 | 20419 | 2 |
| 3* | Washer M8 | 20068 | 4 |
| 4* | Lock nut M8 | 20412 | 2 |
| 5 | Front castor 6" | 20817 | 2 |
| 5 | Front castor 7" | 20976 | 2 |
| 6 | Front castor standard with axle | 20429 | 2 |

* Available upon request.

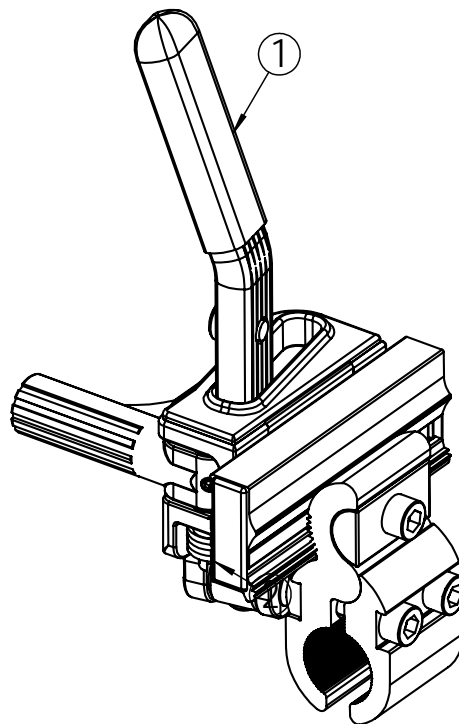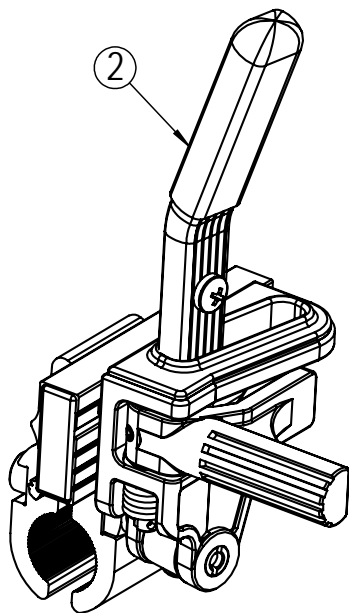


Brake

| Position | Description | Item no. | Pcs. pr. chair |
|----------|-------------------------------------|----------|----------------|
| 1 | Brake Universal Netti I/Svipp Left | 26824 | 1 |
| 2 | Brake Universal Netti I/Svipp Right | 26817 | 1 |

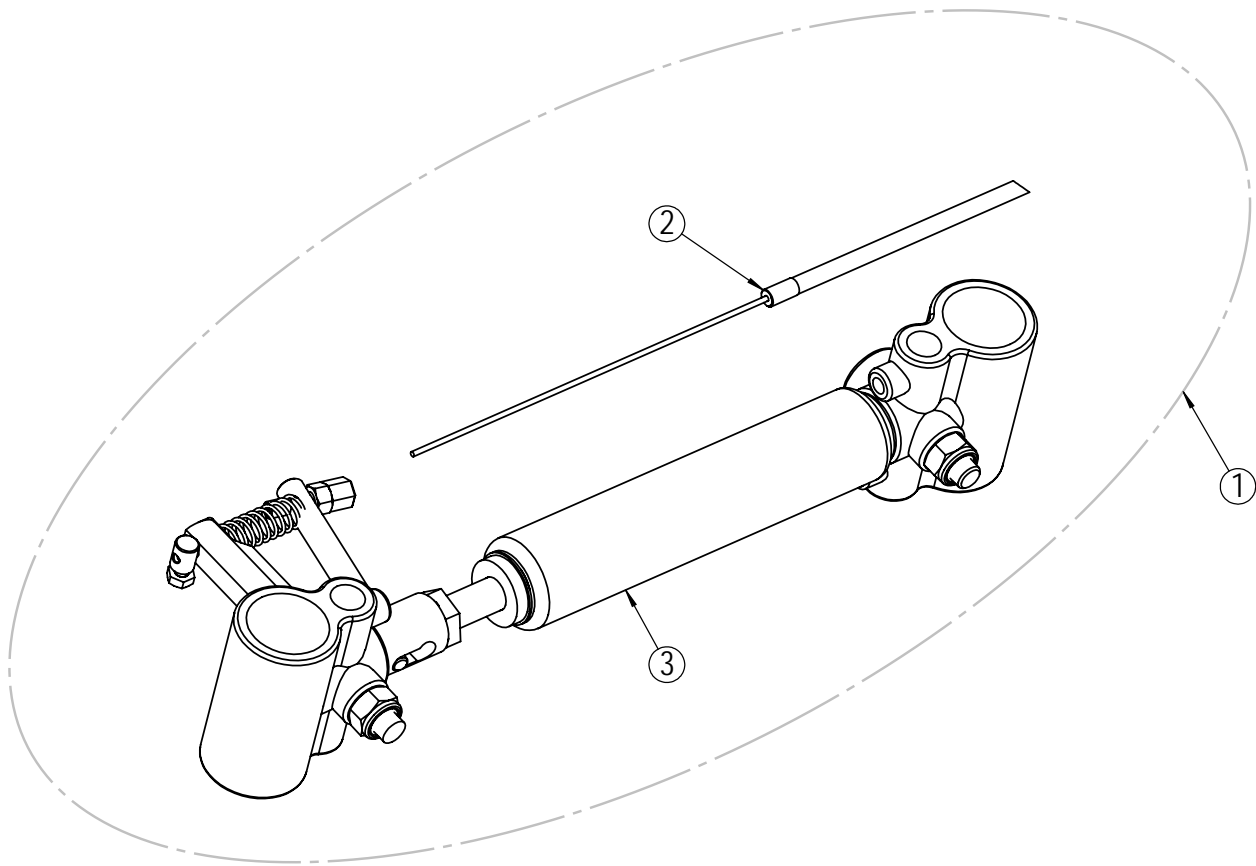


Gas cylinder - recline

| Position | Description | Item no. | Pcs. pr. chair |
|----------|----------------------------------------------|----------|----------------|
| 1 | Tilt cylinder with wire complete | 25514 | 1 |
| 2 | Wire with sleeve, clamps and spring complete | 27628 | 1 |
| 3 | Tilt cylinder complete | 23273 | 1 |

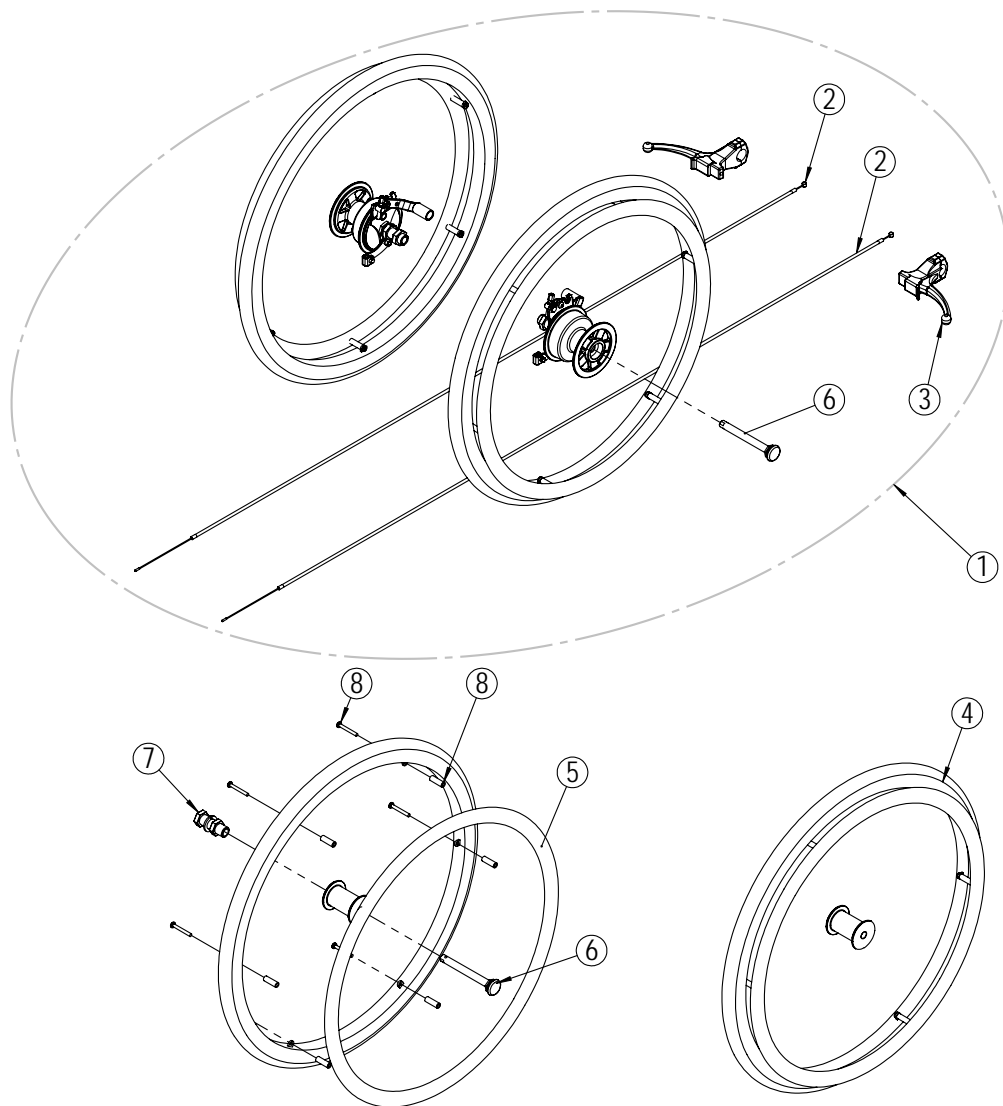

Main wheels

| Position | Description | Item no. | Pcs. pr. chair |
|----------|--------------------------------------------------|----------|----------------|
| 1 | Main wheel 22" x 1" with drum brake - kit | 24365 | 1 |
| 1 | Main wheel 24" x 1" with drum brake - kit | 24310 | 1 |
| 2 | Wire 1250 mm complete | 41310 | 2 |
| 3 | Brake handle | 41313 | 2 |
| 4 | Main wheel 22" x 1" with quick release | 25371 | 2 |
| 4 | Main wheel 24" x 1" with quick release | 25372 | 2 |
| 5 | Push rim 22" plast | 20451 | 2 |
| 5 | Push rim 24" plast | 20450 | 2 |
| 6 | Quick release Ø12 x 98mm | 20449 | 2 |
| 7 | Wheel bushing complete | 20329 | 2 |
| 8 | Screw (6 pcs.) & distance sleeves (6 pcs.) - kit | 21111 | 2 |

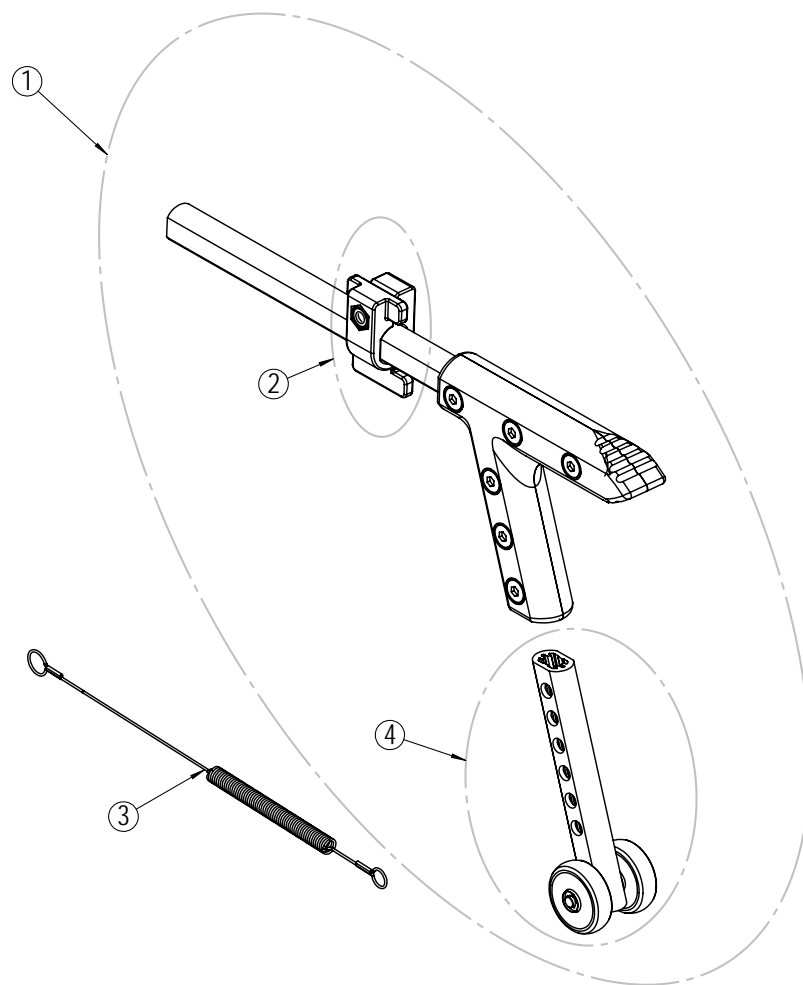

Anti tip

| Position | Description | Item no. | Pcs. pr. chair |
|----------|--------------------------------------------|----------|----------------|
| 1 | Anti tip long/short | 25419 | 2 |
| 2 | Anti tip lock bracket complete | 26980 | 2 |
| 3* | Anti tip spring 445mm | 24242 | 2 |
| 4* | Anti tip vertical bar with wheels standard | 26118 | 2 |

* Available upon request