

SPARE PART CATALOG NETTI SVIPP

Version 2/08 -

INDEX

	Page
Seat frame and backrest	3
Seat cover - adjustable front	4
Seat frame and backrest	5
Backrest	6
Seat cushion - complete	7
Seat cushion - outer cover	8
Backrest - recline kit	9
Push handle	10
Head support rack	11
Head support A	12
Head support B	13
Head support C	14
Arm support left	15
Arm support right	16
Extension piece foot support	17
Universal foot supports - complete	18
Universal foot supports - parts	19
Angle adjustable foot supports - complete	20
Angle adjustable foot supports - parts	21
Bearing house	23
Front castors	24
Brake	25
Main wheels	26
Anti tip	27


Produced by:

Alu Rehab AS, Bedriftsveien 23, NO-4352 Klepp stasjon, Norway

Phone: + 47 51 78 62 20 - Fax + 47 51 78 62 21


E-mail: post@alurehab.com

www.alurehab.com

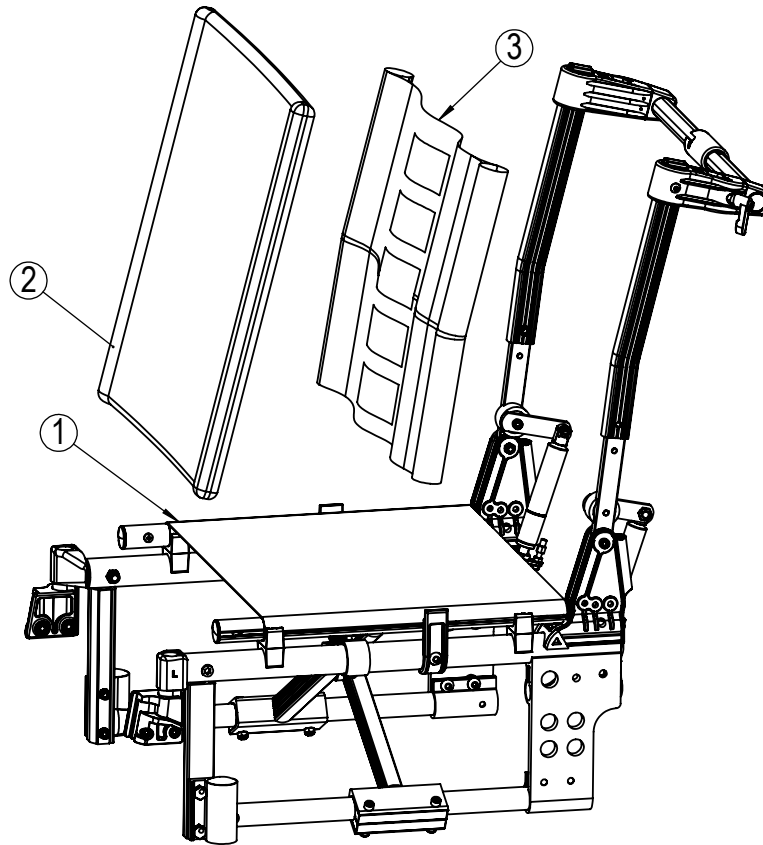


Seat frame and backrest

Position	Description	Item no.	Pcs. pr. chair
1	Cross Seat cover 35cm Netti Svipp long	22020	1
1	Cross Seat cover 38cm Netti Svipp long	22021	1
1	Cross Seat cover 40cm Netti Svipp long	22022	1
1	Cross Seat cover 43cm Netti Svipp long	22023	1
1	Cross Seat cover 45cm Netti Svipp long	22024	1
1	Cross Seat cover 50cm Netti Svipp long	22025	1
1	Cross Seat cover 55cm Netti Svipp long	22026	1
1	Cross Seat cover 35cm Netti Svipp short	22170	1
1	Cross Seat cover 38cm Netti Svipp short	22171	1
1	Cross Seat cover 40cm Netti Svipp short	22172	1
1	Cross Seat cover 43cm Netti Svipp short	22173	1
1	Cross Seat cover 45cm Netti Svipp short	22174	1

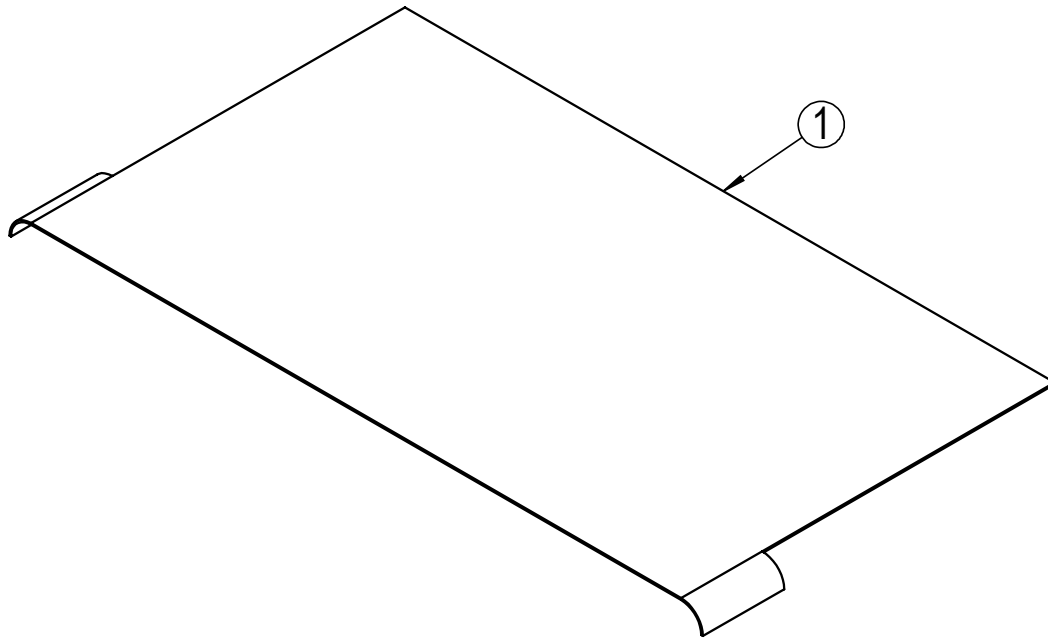


Seat cover - adjustable front

Position	Description	Item no.	Pcs. pr. chair
1	Seat cover adjustable front Netti Svipp 35cm	22391	1
1	Seat cover adjustable front Netti Svipp 38cm	22392	1
1	Seat cover adjustable front Netti Svipp 40cm	22393	1
1	Seat cover adjustable front Netti Svipp 43cm	22394	1
1	Seat cover adjustable front Netti Svipp 45cm	22395	1
1	Seat cover adjustable front Netti Svipp 50cm	22396	1
1	Seat cover adjustable front Netti Svipp 55cm	22397	1

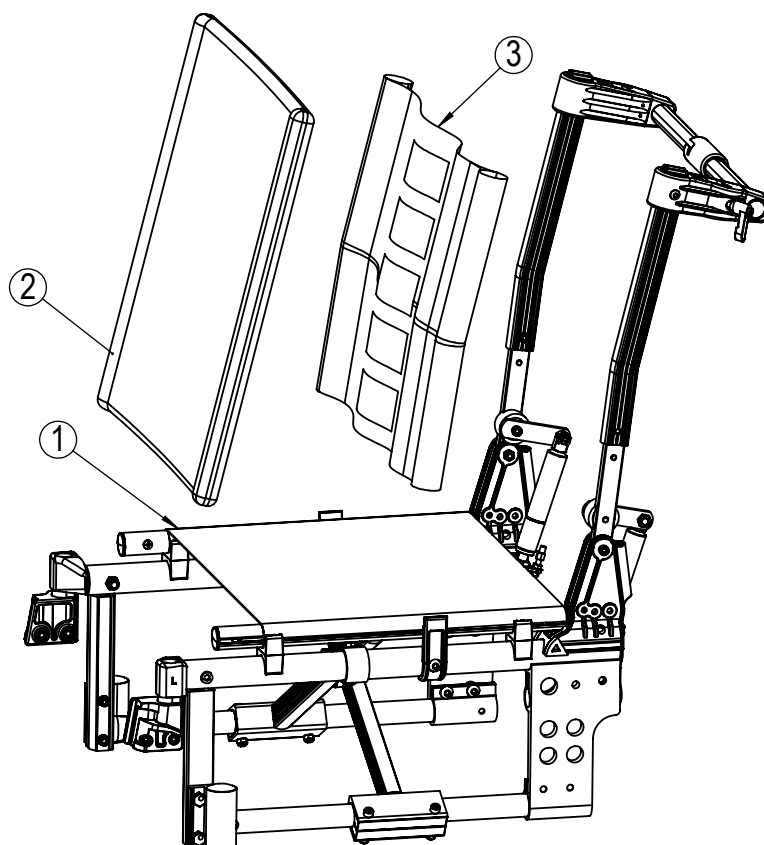

Seat frame and backrest


Position	Description	Item no.	Pcs. pr. chair
2	Backrest cushion Flip Mars 35cm	22052	1
2	Backrest cushion Flip Mars 38cm	22053	1
2	Backrest cushion Flip Mars 40cm	22054	1
2	Backrest cushion Flip Mars 43cm	22055	1
2	Backrest cushion Flip Mars 45cm	22056	1
2	Backrest cushion Flip Mars 50cm	22057	1
2	Backrest cushion Flip Mars 55cm	22058	1
3	Backrest Velcro 35-50 cm Height: 35cm	20699	1
3	Backrest Velcro 55-60 cm Height: 35cm	2865	1

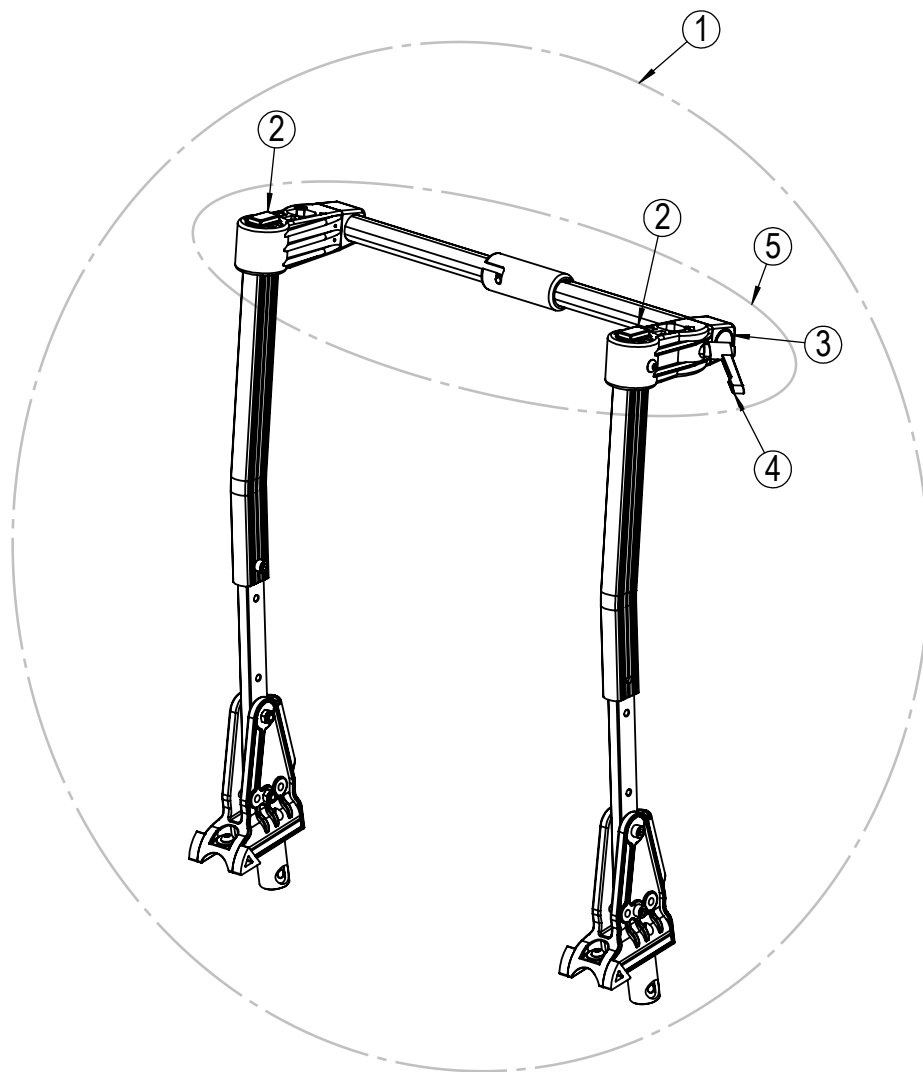

Backrest


Position	Description	Item no.	Pcs. pr. chair
1	Backrest Netti Svipp 35cm komplett	26125	1
1	Backrest Netti Svipp 38cm komplett	26126	1
1	Backrest Netti Svipp 40cm komplett	26127	1
1	Backrest Netti Svipp 43cm komplett	26128	1
1	Backrest Netti Svipp 45cm komplett	26129	1
1	Backrest Netti Svipp 50cm komplett	26130	1
1	Rygg Netti Svipp 55cm komplett	26131	1
2*	Square plug 25x15	21847	2
3*	Plastic plug	20509	2
4*	Clamp lever M6x16	20097	2
5	Backrest stabilization bar complete 35cm	23419	1
5	Backrest stabilization bar complete 38cm	23420	1
5	Backrest stabilization bar complete 40cm	23421	1
5	Backrest stabilization bar complete 43cm	23422	1
5	Backrest stabilization bar complete 45cm	23423	1
5	Backrest stabilization bar complete 50cm	23424	1
5	Backrest stabilization bar complete 55cm	23425	1

* Available upon request

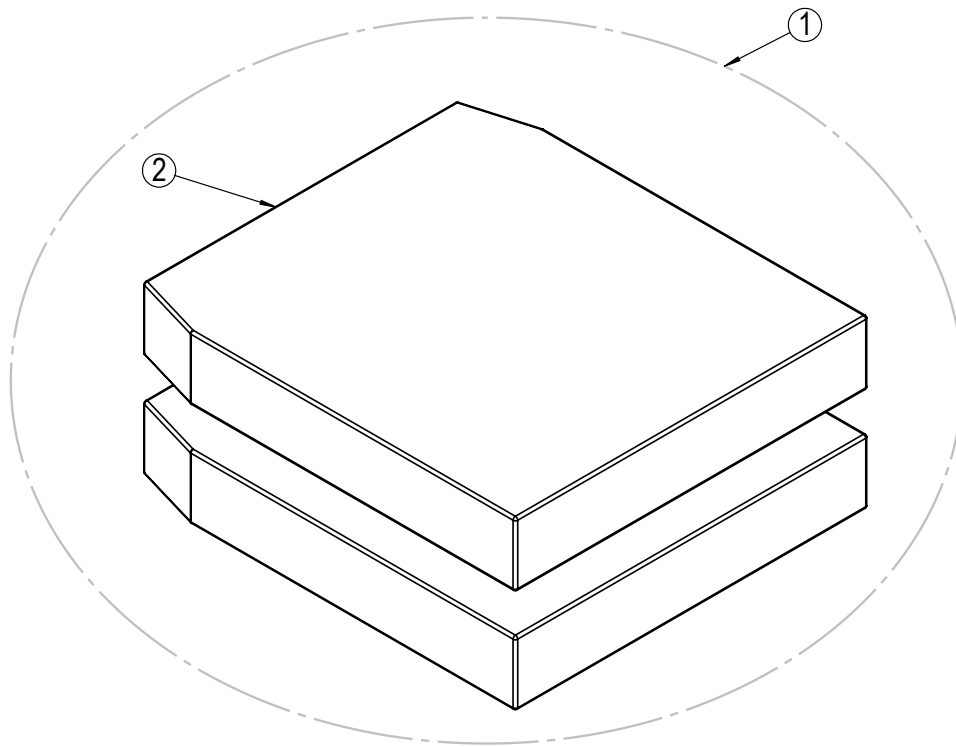


Seat cushion - complete

Position	Description	Item no.	Pcs. pr. chair
1	Seat cushion, Uno Seat Width:35cm Depth:40cm, Mars	840377	1
1	Seat cushion, Uno Seat Width:35cm Depth:45cm, Mars	840378	1
1	Seat cushion, Uno Seat Width:35cm Depth:50cm, Mars	840379	1
1	Seat cushion, Uno Seat Width:38cm Depth:40cm, Mars	840380	1
1	Seat cushion, Uno Seat Width:38cm Depth:45cm, Mars	840381	1
1	Seat cushion, Uno Seat Width:38cm Depth:50cm, Mars	840382	1
1	Seat cushion, Uno Seat Width:40cm Depth:40cm, Mars	840385	1
1	Seat cushion, Uno Seat Width:40cm Depth:45cm, Mars	840386	1
1	Seat cushion, Uno Seat Width:40cm Depth:50cm, Mars	840387	1
1	Seat cushion, Uno Seat Width:43cm Depth:40cm, Mars	840390	1
1	Seat cushion, Uno Seat Width:43cm Depth:45cm, Mars	840391	1
1	Seat cushion, Uno Seat Width:43cm Depth:50cm, Mars	840392	1
1	Seat cushion, Uno Seat Width:45cm Depth:40cm, Mars	840395	1
1	Seat cushion, Uno Seat Width:45cm Depth:45cm, Mars	840396	1
1	Seat cushion, Uno Seat Width:45cm Depth:50cm, Mars	840397	1
1	Seat cushion, Uno Seat Width:50cm Depth:40cm, Mars	840400	1
1	Seat cushion, Uno Seat Width:50cm Depth:45cm, Mars	840401	1
1	Seat cushion, Uno Seat Width:50cm Depth:50cm, Mars	840402	1
1	Seat cushion, Uno Seat Width:55cm Depth:40cm, Mars	840405	1
1	Seat cushion, Uno Seat Width:55cm Depth:45cm, Mars	840406	1
1	Seat cushion, Uno Seat Width:55cm Depth:50cm, Mars	840407	1

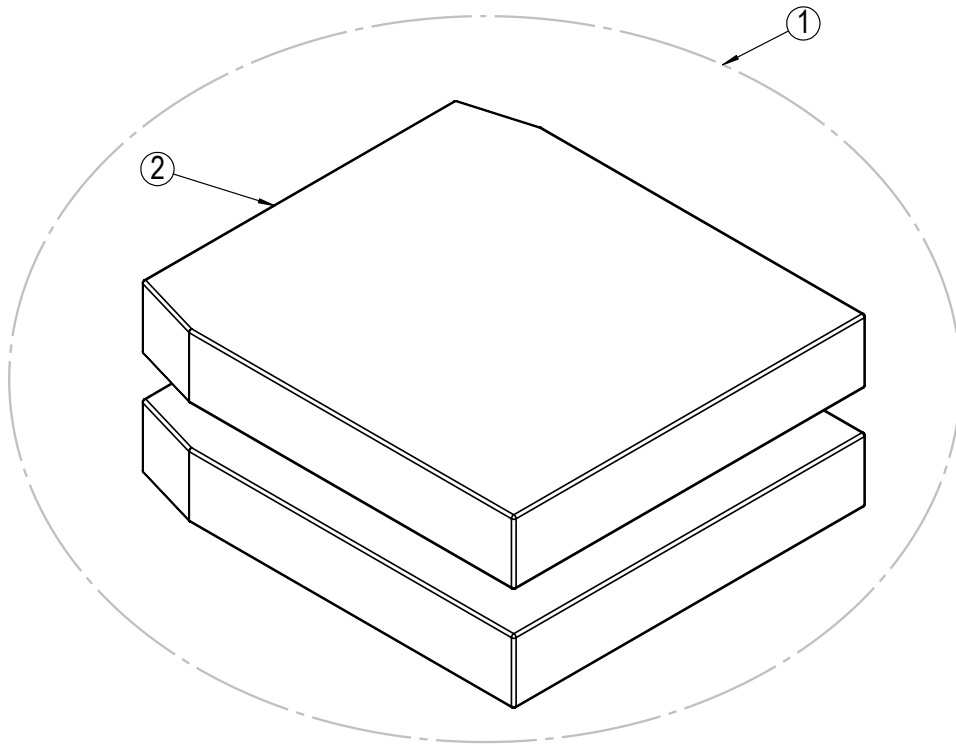

Seat cushion - outer cover


Position	Description	Item no.	Pcs. pr. chair
2	Outer cover, Uno Seat Width:35cm Depth:40cm, Mars	3543540	1
2	Outer cover, Uno Seat Width:35cm Depth:45cm, Mars	3543545	1
2	Outer cover, Uno Seat Width:35cm Depth:50cm, Mars	3543550	1
2	Outer cover, Uno Seat Width:38cm Depth:40cm, Mars	3543840	1
2	Outer cover, Uno Seat Width:38cm Depth:45cm, Mars	3543845	1
2	Outer cover, Uno Seat Width:38cm Depth:50cm, Mars	3543850	1
2	Outer cover, Uno Seat Width:40cm Depth:40cm, Mars	3544040	1
2	Outer cover, Uno Seat Width:40cm Depth:45cm, Mars	3544045	1
2	Outer cover, Uno Seat Width:40cm Depth:50cm, Mars	3544050	1
2	Outer cover, Uno Seat Width:43cm Depth:40cm, Mars	3544340	1
2	Outer cover, Uno Seat Width:43cm Depth:45cm, Mars	3544345	1
2	Outer cover, Uno Seat Width:43cm Depth:50cm, Mars	3544550	1
2	Outer cover, Uno Seat Width:45cm Depth:40cm, Mars	3544540	1
2	Outer cover, Uno Seat Width:45cm Depth:45cm, Mars	3544545	1
2	Outer cover, Uno Seat Width:45cm Depth:50cm, Mars	3544550	1
2	Outer cover, Uno Seat Width:50cm Depth:40cm, Mars	3545040	1
2	Outer cover, Uno Seat Width:50cm Depth:45cm, Mars	3545045	1
2	Outer cover, Uno Seat Width:50cm Depth:50cm, Mars	3545050	1
2	Outer cover, Uno Seat Width:55cm Depth:40cm, Mars	3545540	1
2	Outer cover, Uno Seat Width:55cm Depth:45cm, Mars	3545545	1
2	Outer cover, Uno Seat Width:55cm Depth:50cm, Mars	3545550	1

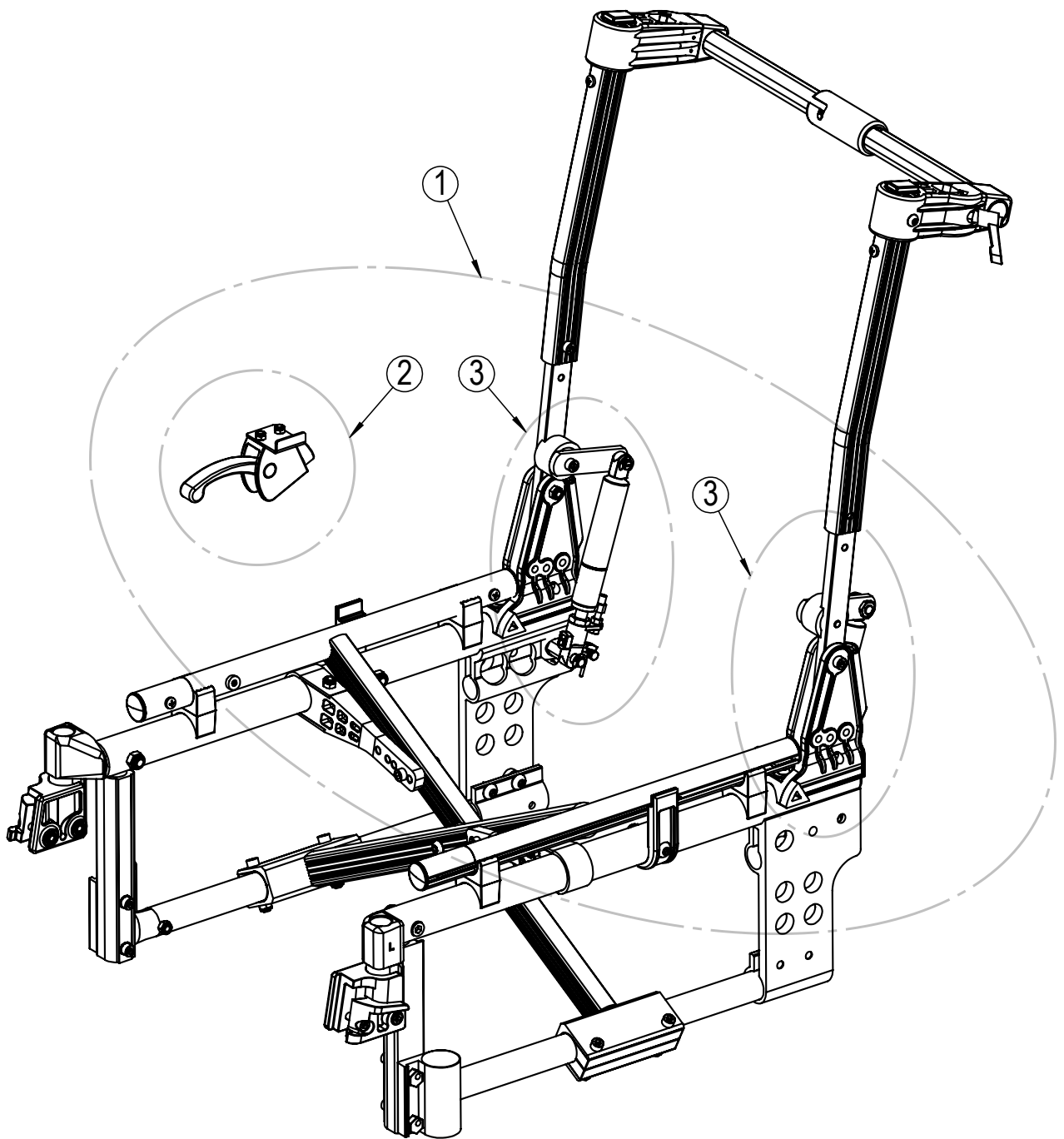

Backrest- recline kit*

Position	Description	Item no.	Pcs. pr. chair
1	Recline kit complete	25400	1
2	Release handle recline right	26190	1
2	Release handle recline left	26191	1
3	Gas cylinder recline Netti Svipp complete	24471	2


* Available upon request

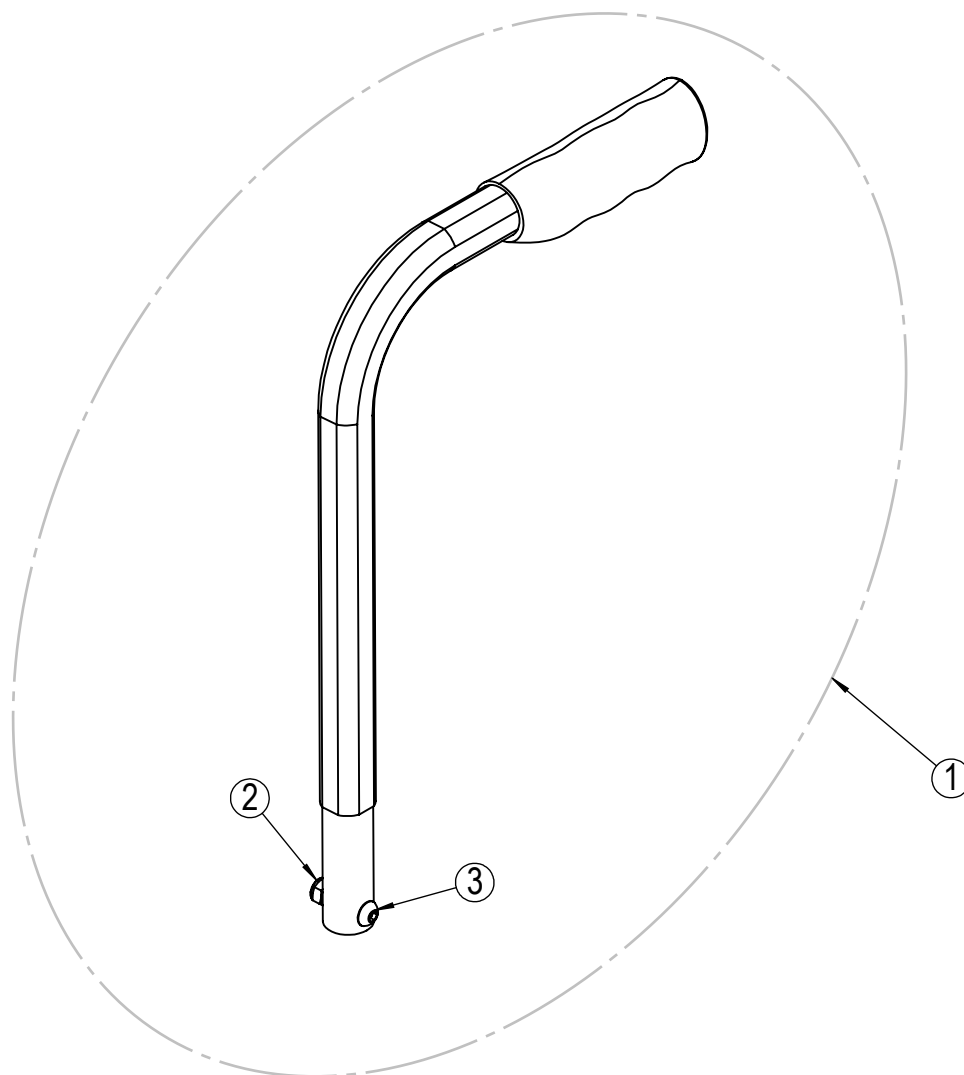

Push handle


Position	Description	Item no.	Pcs. pr. chair
1	Push handle standard 300mm complete	21754	2
1	Push handle long 500mm complete	21808	2
2*	Lock nut M6	20058	2
3*	Screw M6x25	20992	2

* Available upon request

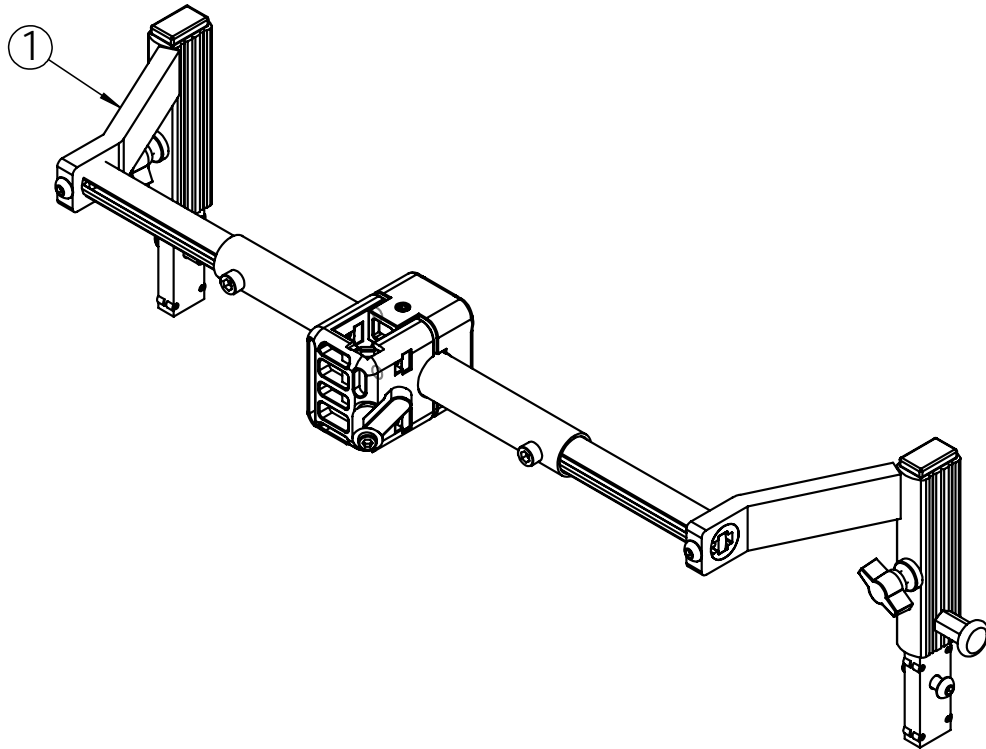

Head support rack


Position	Description	Item no.	Pcs. pr. chair
1	Head support rack Netti Svipp 35-45cm complete	27214	1
1	Head support rack Netti Svipp 35-45cm complete	27215	1

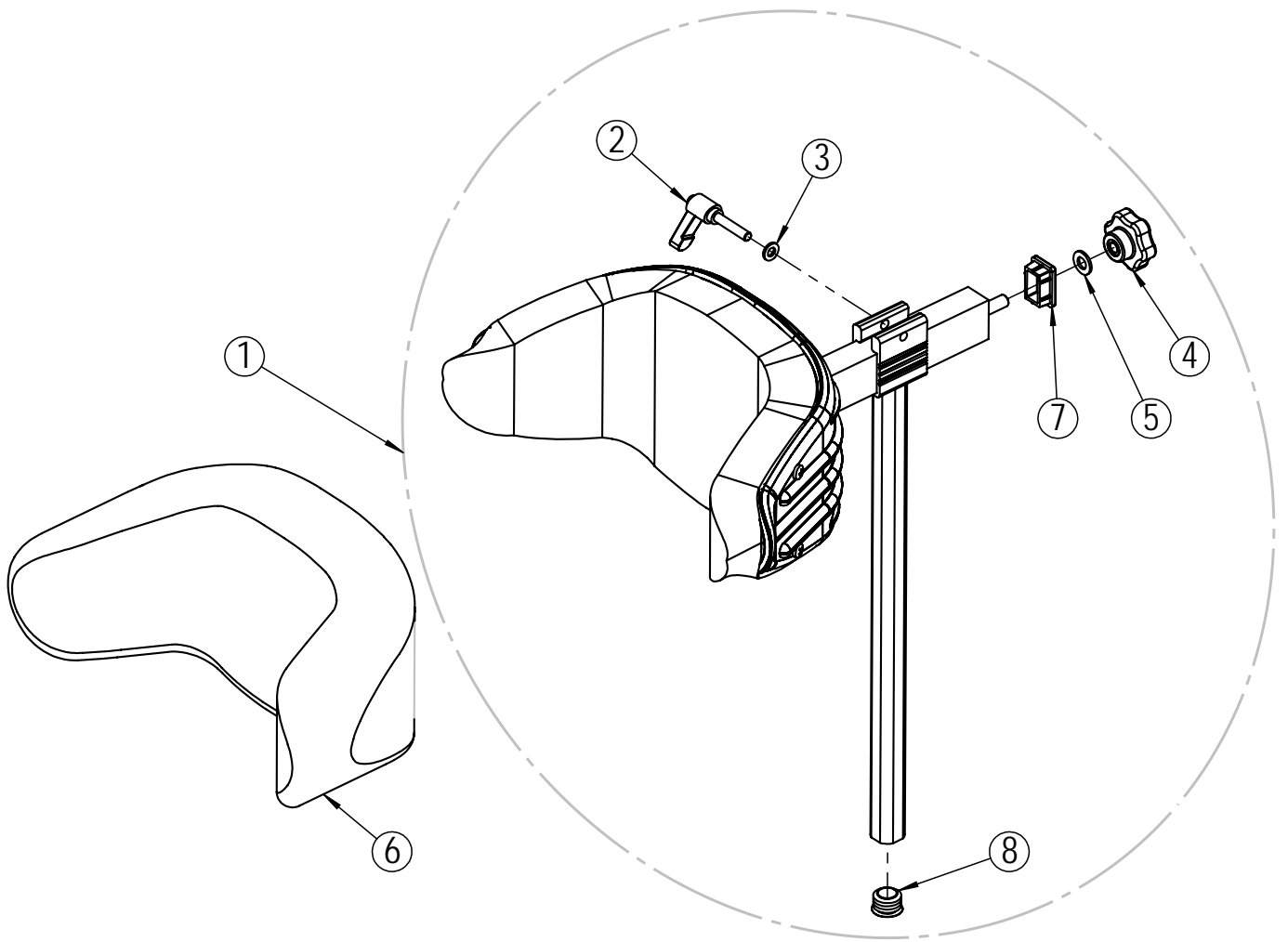

Head support A

Position	Description	Item no.	Pcs. pr. chair
1	Head support A with Mars	20575	1
2*	Clamp lever M6x30	20282	1
3*	Washer M6	20063	1
4*	Star wheel M8	26684	1
5*	Washer M8	29305	1
6	Hygiene cover Mars Head support A	20126	1
7*	Square plug horizontal bar	20852	1
8*	Plastic plug vertical bar	20509	1


* Available upon request.

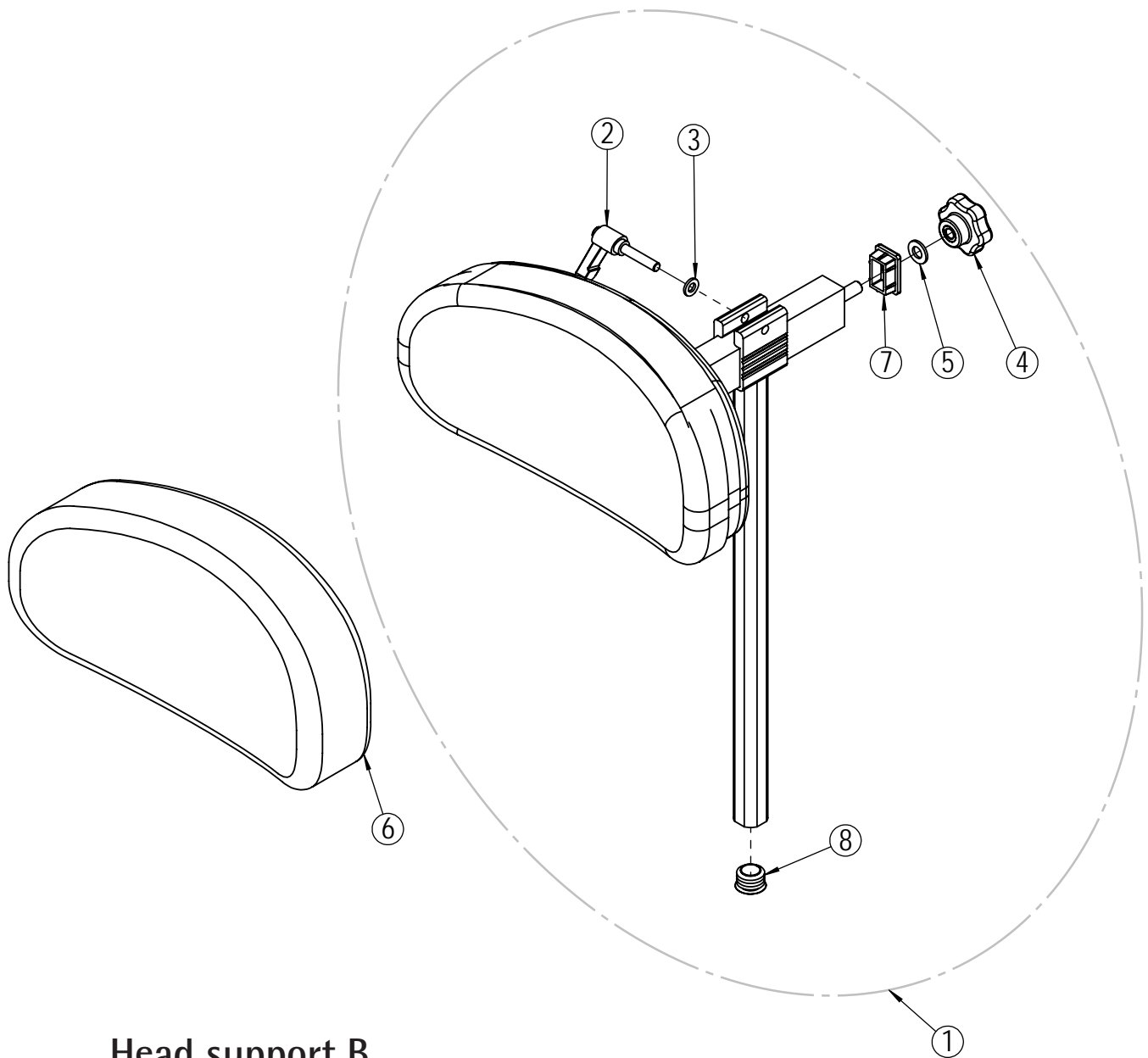

Head support B

Position	Description	Item no.	Pcs. pr. chair
1	Head support B with Mars	20574	1
2*	Clamp lever M6x30	20282	1
3*	Washer M6	20063	1
4*	Star wheel M8	26684	1
5*	Washer M8	29305	1
6	Hygiene cover Mars Head support B	20629	1
7*	Square plug horizontal bar	20852	1
8*	Plastic plug vertical bar	20509	1


* Available upon request.

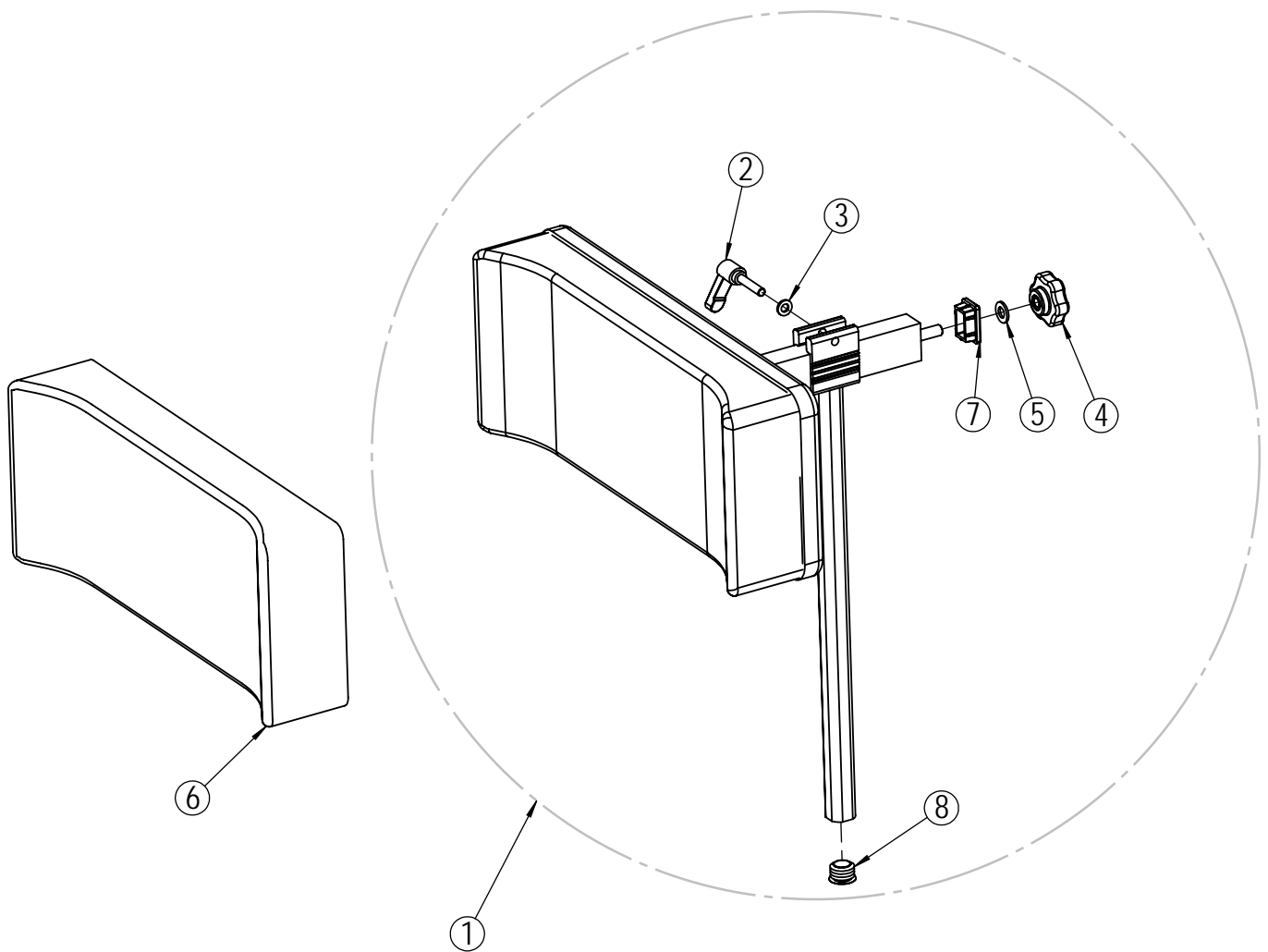

Head support C

Position	Description	Item no.	Pcs. pr. chair
1	Head support C with Mars	20573	1
2*	Clamp lever M6x30	20282	1
3*	Washer M6	20063	1
4*	Star wheel M8	26684	1
5*	Washer M8	29305	1
6	Hygiene cover Mars Head support C	20628	1
7*	Square plug horizontal bar	20852	1
8*	Plastic plug vertical bar	20509	1


* Available upon request.

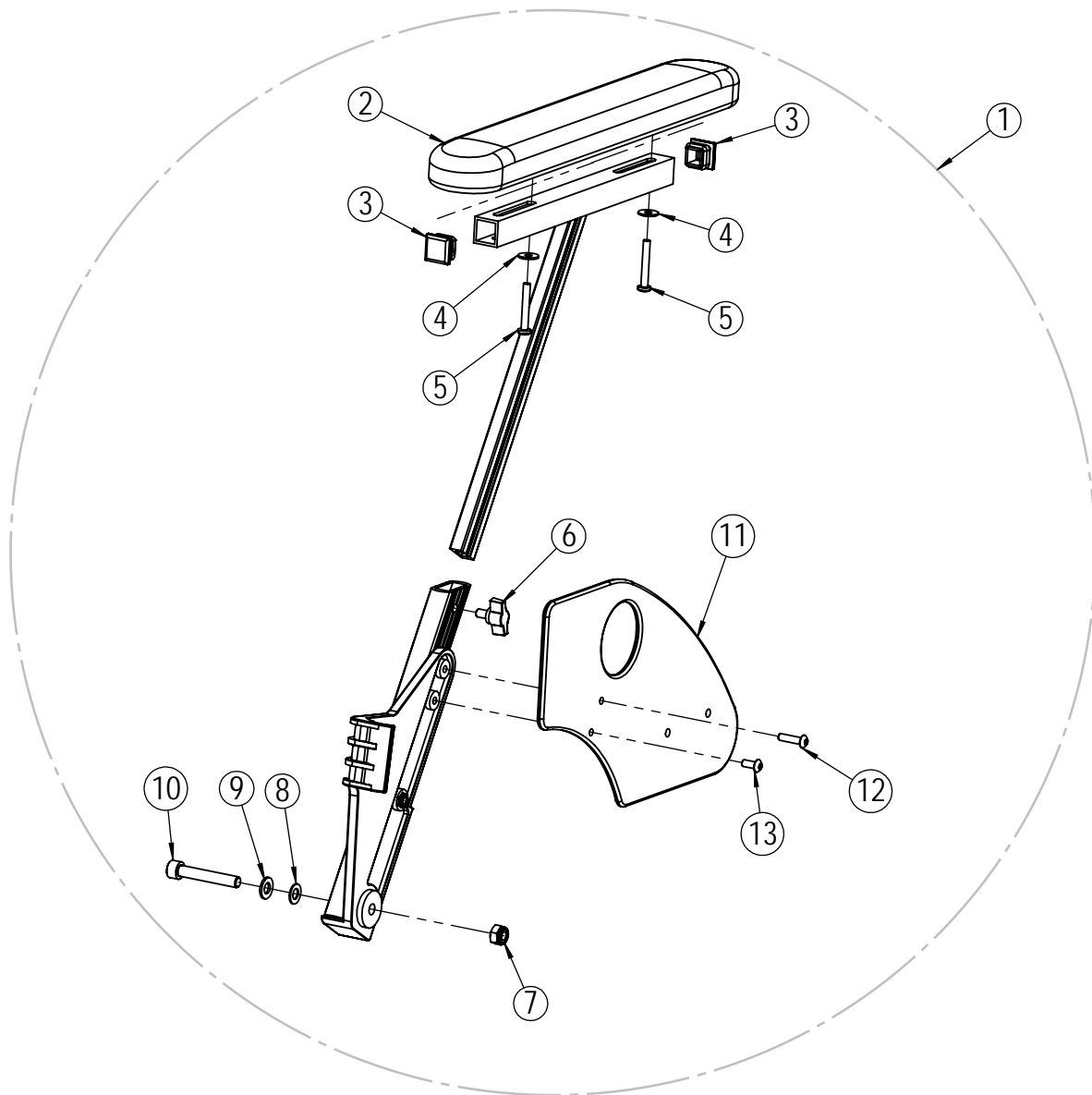

Arm support - left

Position	Description	Item no.	Pcs. pr. chair
1	Arm support Netti Svipp left complete	23234	1
2	Arm support pad 385x58 PU with screws	20835	1
3*	Square plug	20633	2
4*	Washer M5	23594	2
5*	Screw M5x35	21323	2
6*	Wing screw M6x10	23256	1
7*	Lock nut M8	20412	1
8*	Washer nylon Ø15	23362	1
9*	Washer M8	20068	1
10*	Screw M8x50	25311	1
11*	Skirt guard	25503	1
12*	Screw M5x20	25438	1
13*	Screw M5x12	25310	1


* Available upon request

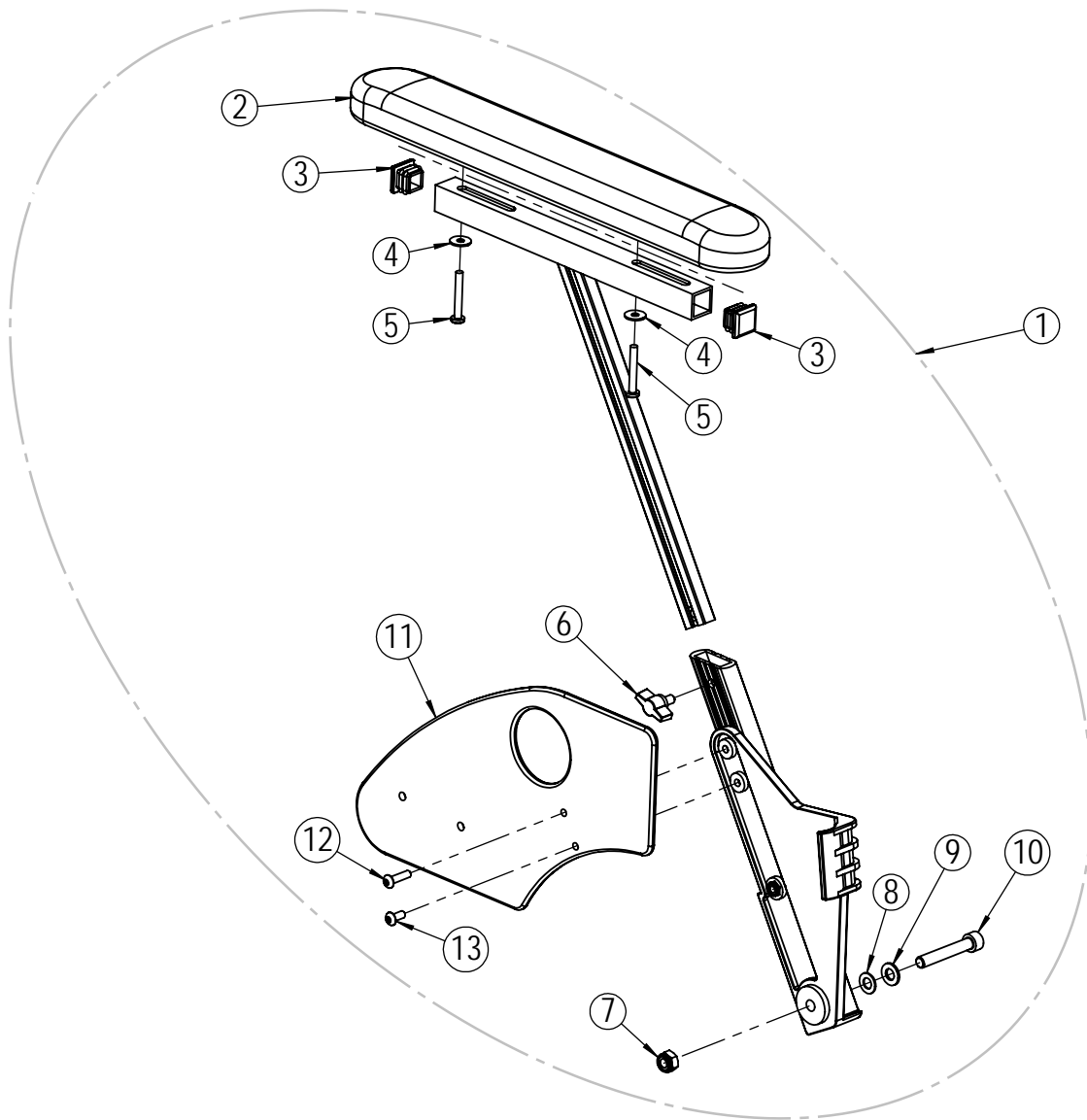

Arm support - right


Position	Description	Item no.	Pcs. pr. chair
1	Arm support Netti Svipp left complete	23226	1
2	Arm support pad 385x58 PU with screws	20835	1
3*	Square plug	20633	2
4*	Washer M5	23594	2
5*	Screw M5x35	21323	2
6*	Wing screw M6x10	23256	1
7*	Lock nut M8	20412	1
8*	Washer nylon Ø15	23362	1
9*	Washer M8	20068	1
10*	Screw M8x50	25311	1

* Available upon request

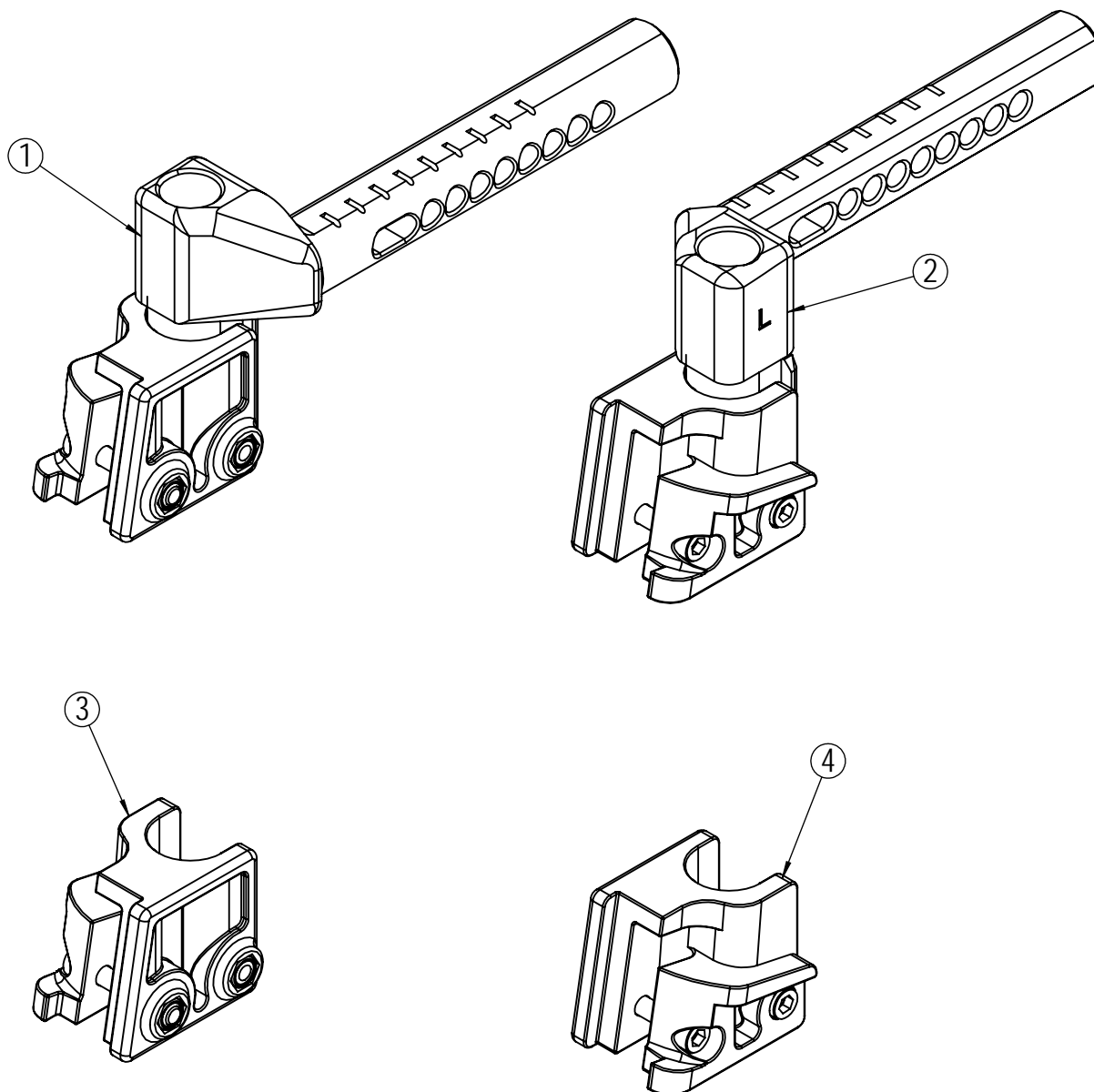


Extension piece - foot support

Position	Description	Item no.	Pcs. pr. chair
1	Extension piece. Silver. Right. Complete.	27384	1
2	Extension piece. Silver. Left. Complete.	27385	1
3	Locking bracket with screw. Right.	29555	1
4	Locking bracket with screw. Left.	29556	1

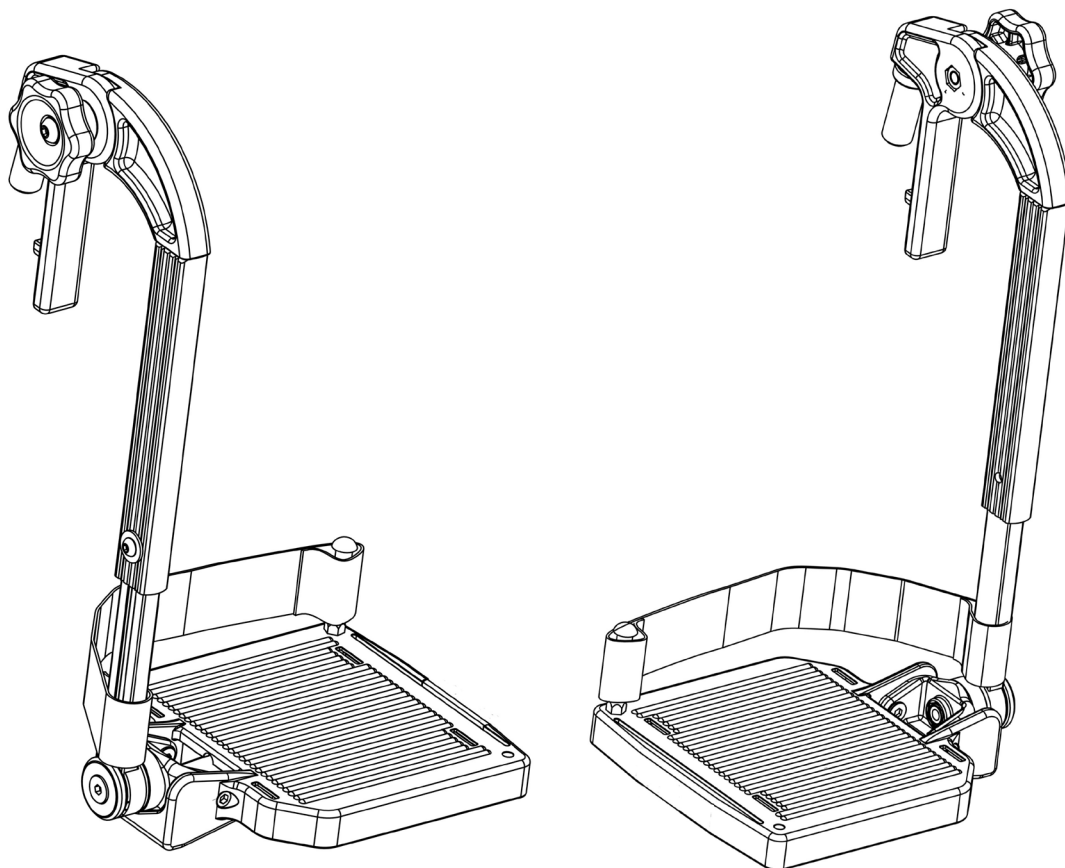

Universal foot supports - complete


Position	Description	Item no.	Pcs. pr. chair
1	Universal foot support. Silver. 35cm. Right.	27220	1
2	Universal foot support. Silver. 35cm Left.	27221	1
1	Universal foot support. Silver. 38cm. Right.	27222	1
2	Universal foot support. Silver. 38cm Left.	27223	1
1	Universal foot support. Silver. 40cm. Right.	27224	1
2	Universal foot support. Silver. 40cm Left.	27225	1
1	Universal foot support. Silver. 43cm. Right.	27226	1
2	Universal foot support. Silver. 43cm Left.	27227	1
1	Universal foot support. Silver. 45cm. Right.	27228	1
2	Universal foot support. Silver. 45cm Left.	27229	1
1	Universal foot support. Silver. 50cm. Right.	27230	1
2	Universal foot support. Silver. 50cm Left.	27231	1
1	Universal foot support. Silver. 55cm. Right.	27232	1
2	Universal foot support. Silver. 55cm Left.	27233	1

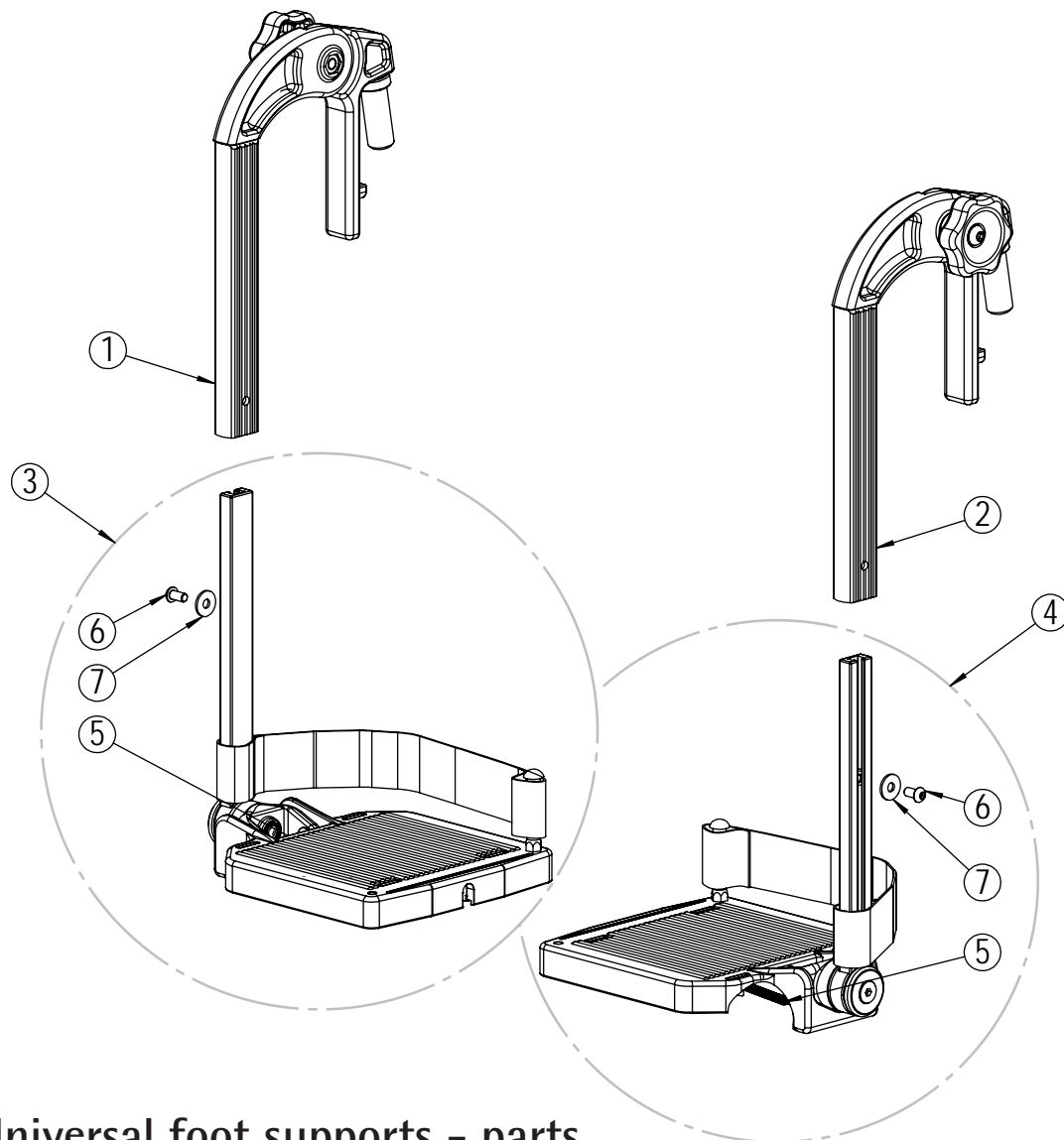

Universal foot supports - parts

Position	Description	Item no.	Pcs. pr. chair
1	Universal foot support. Swing element. Silver. Right.	25824	1
2	Universal foot support. Swing element. Silver. Left.	26723	1
3	Footplate with heelstrap. 35cm. Right.	27875	1
4	Footplate with heelstrap. . 35cm. Left.	27899	1
3	Footplate with heelstrap. . 38cm. Right.	27876	1
4	Footplate with heelstrap. . 38cm. Left.	27900	1
3	Footplate with heelstrap. . 40cm. Right.	27877	1
4	Footplate with heelstrap. . 40cm. Left.	27901	1
3	Footplate with heelstrap. . 43cm. Right.	27878	1
4	Footplate with heelstrap. . 43cm. Left.	27902	1
3	Footplate with heelstrap. . 45cm. Right.	27879	1
4	Footplate with heelstrap. . 45cm. Left.	27903	1
3	Footplate with heelstrap. . 50cm. Right.	27880	1
4	Footplate with heelstrap. . 50cm. Left.	27904	1
3	Footplate with heelstrap. . 55cm. Right.	27881	1
4	Footplate with heelstrap. . 55cm. Left.	27905	1
5*	Footplate with heelstrap. fjær 1x6x65	27858	2
6*	Screw M6x12	24272	2
7*	Washer 6,4x18x1,5	20356	2

* Available upon request

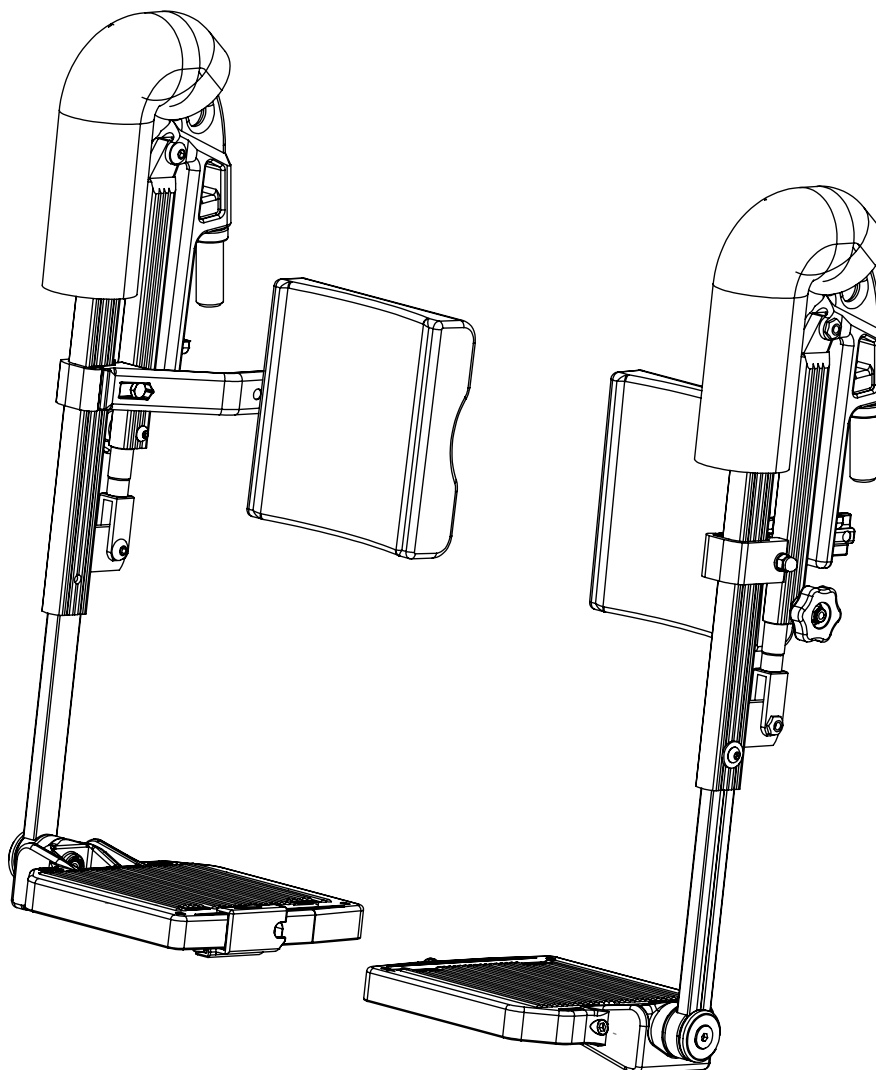


Angle adjustable foot supports - complete


Position	Description	Item no.	Pcs. pr. chair
1	Angle adjustable foot support. Black. 35cm. Right.	29500	1
2	Angle adjustable foot support. Black. 35cm. Left.	29508	1
1	Angle adjustable foot support. Black. 40cm. Right.	29502	1
2	Angle adjustable foot support. Black. 40cm. Left.	29510	1
1	Angle adjustable foot support. Black. 43cm. Right.	29503	1
2	Angle adjustable foot support. Black. 43cm. Left.	29511	1
1	Angle adjustable foot support. Black. 45cm. Right.	29504	1
2	Angle adjustable foot support. Black. 45cm. Left.	29512	1
1	Angle adjustable foot support. Black. 50cm. Right.	29505	1
2	Angle adjustable foot support. Black. 50cm. Left.	29513	1
1	Angle adjustable foot support. Black. 55cm. Right.	29506	1
2	Angle adjustable foot support. Black. 55cm. Left.	29514	1

Angle adjustable foot supports - parts

Position	Description	Item no.	Pcs. pr. chair
3	Angle adjustable foot support. Black. Right.	29489	1
4	Angle adjustable foot support. Black. Left.	29490	1
5	Knee upholstery. Mars.	23864	2
6	Calf pad with bracket. 35cm	27964	1
6	Calf pad with bracket. 40cm	27965	1
6	Calf pad with bracket. 43cm	27966	1
6	Calf pad with bracket. 45cm	27966	1
6	Calf pad with bracket. 50cm	27967	1
6	Calf pad with bracket. 55cm	27968	1
7	Foot plate. 35cm Right.	28621	1
8	Foot plate. 35cm Left.	28649	1
7	Foot plate. 40cm Right.	28623	1
8	Foot plate. 40cm Left.	28651	1
7	Foot plate. 43cm Right.	28624	1
8	Foot plate. 43cm Left.	28652	1
7	Foot plate. 45cm Right.	28625	1
8	Foot plate. 45cm Left.	28653	1
7	Foot plate. 50cm Right.	28626	1
8	Foot plate. 50cm Left.	28654	1
7	Foot plate. 55cm Right.	28627	1
8	Foot plate. 55cm Left.	28655	1
9*	Foot plate spring	27858	2
10*	Screw M6x12	24272	2
11*	Washer	20356	2
12	Mechanical adjustment piece with screw. Black. Right.	29558	1
13	Mechanical adjustment piece with screw. Black. Left.	29559	1
14*	Screw M6x40	20103	2
15*	Adjustment wheel	27388	2
16	Foot plate hinge complete	28523	2
17	Calf support bracket with screw, washer and lock nut	25353	2
18*	Nut M8	29761	2
19*	Foot support locking cone	20539	2
20*	Nut M6	20058	2

* Available upon request.

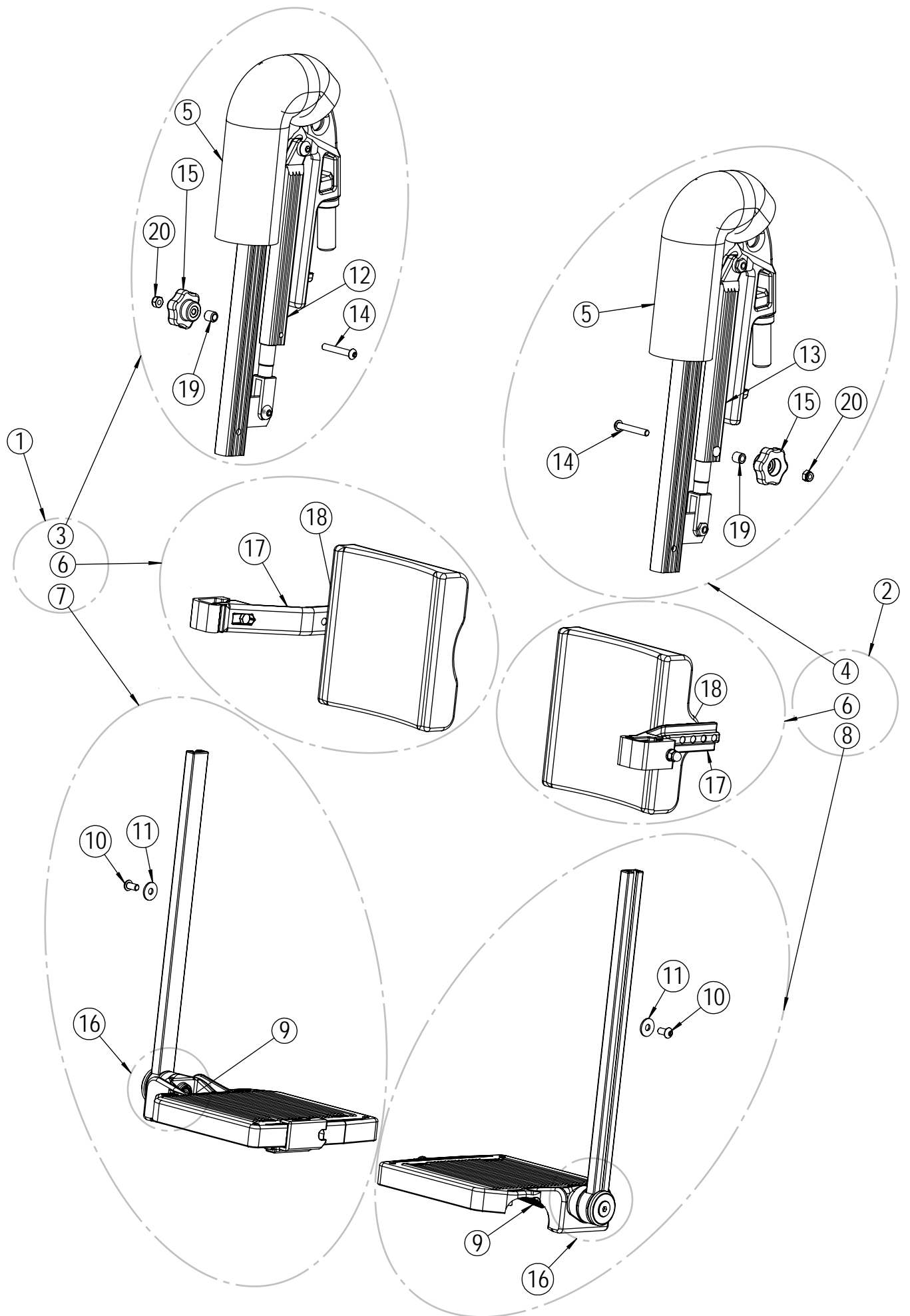


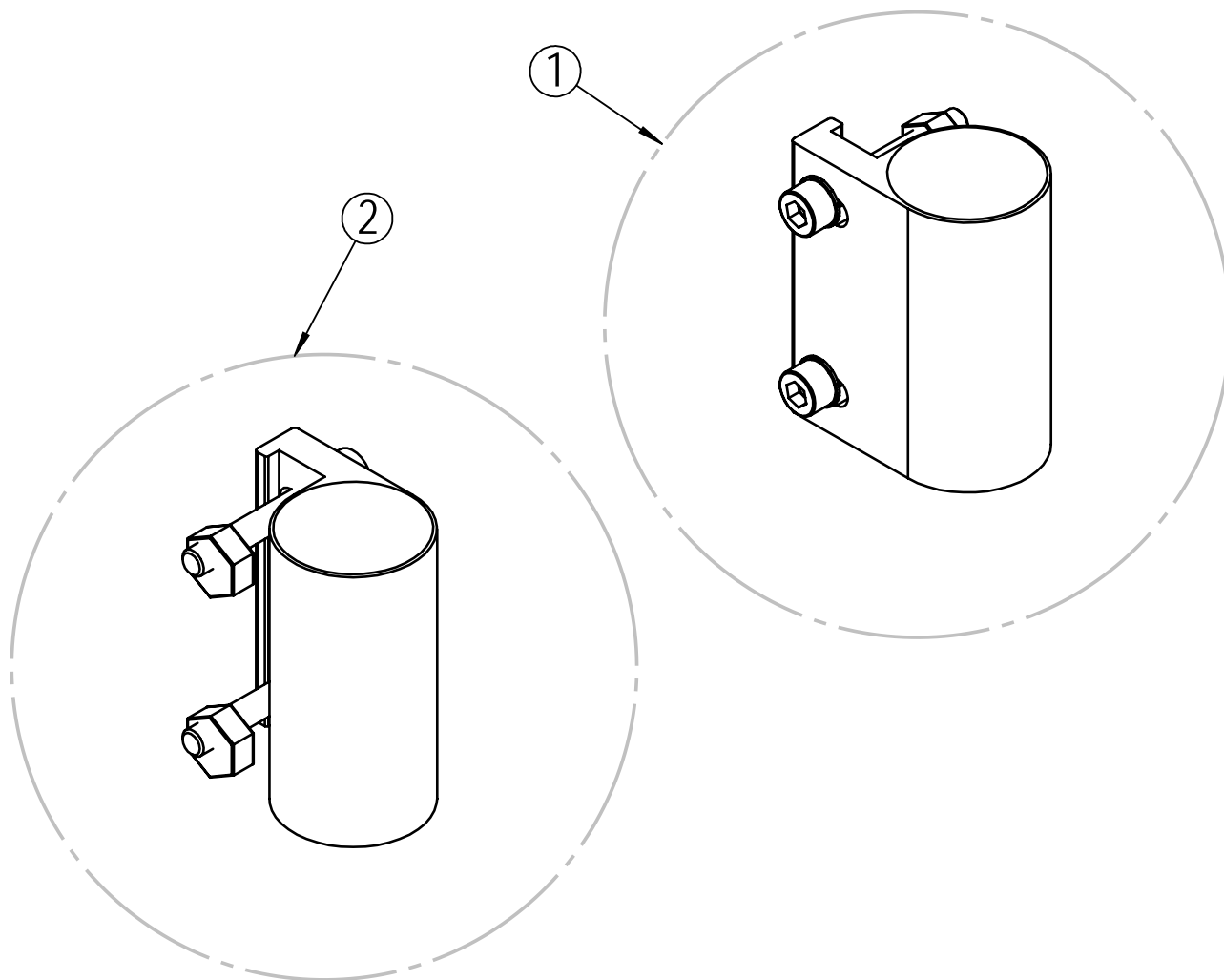

Bearing house

Position	Description	Item no.	Pcs. pr. chair
1	Bearing house A complete	26445	1
2	Bearing house B complete	26446	1


* Available upon request

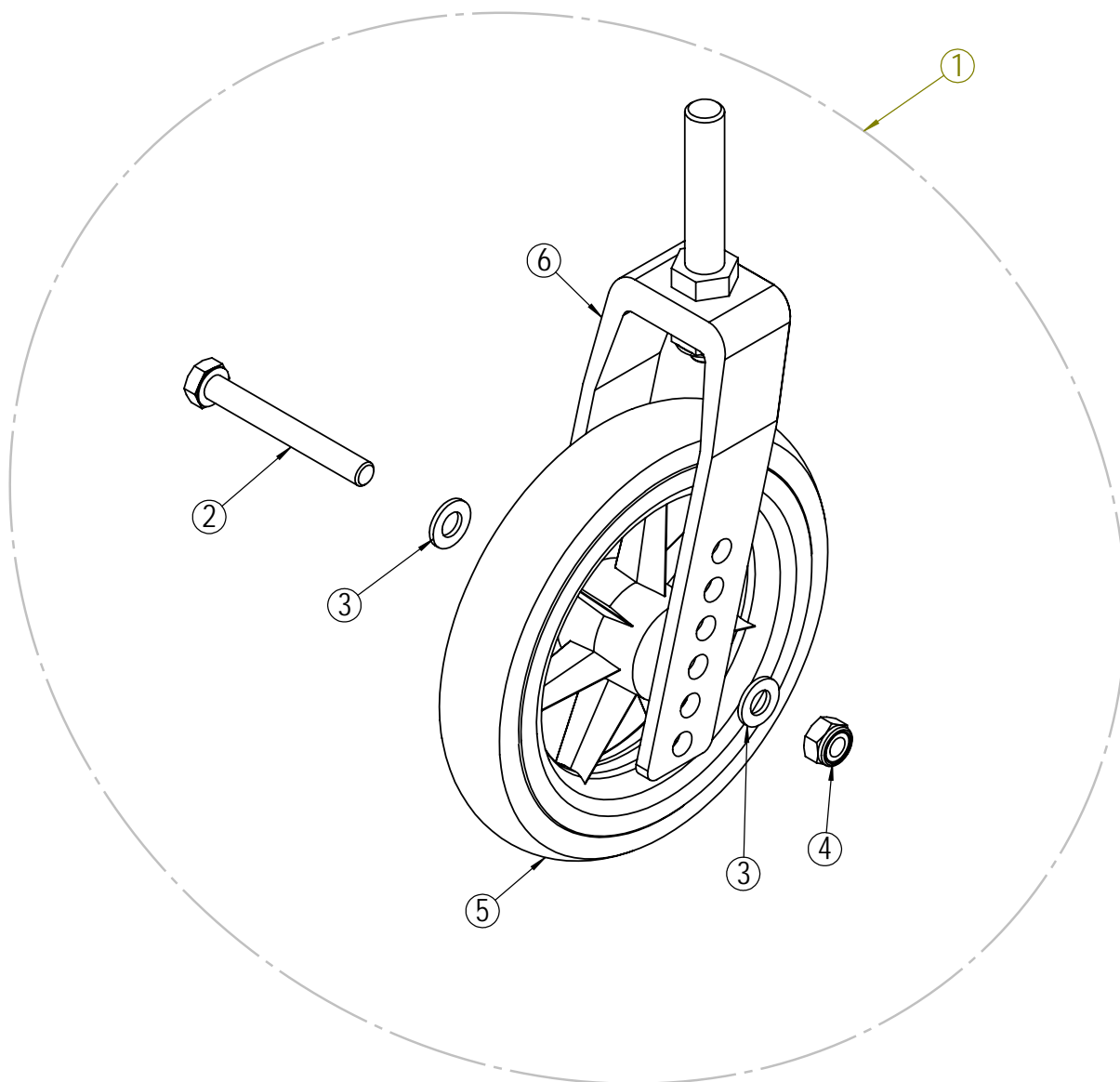

Front castors


Position	Description	Item no.	Pcs. pr. chair
1	Fork standard with 6" front castors complete	20538	2
1	Fork standard with 7" front castors complete	20535	2
2*	Screw M8x65	20419	2
3*	Washer M8	20068	4
4*	Lock nut M8	20412	2
5	Front castor 6"	20817	2
5	Front castor 7"	20976	2
6	Front castor standard with axle	20429	2

* Available upon request.

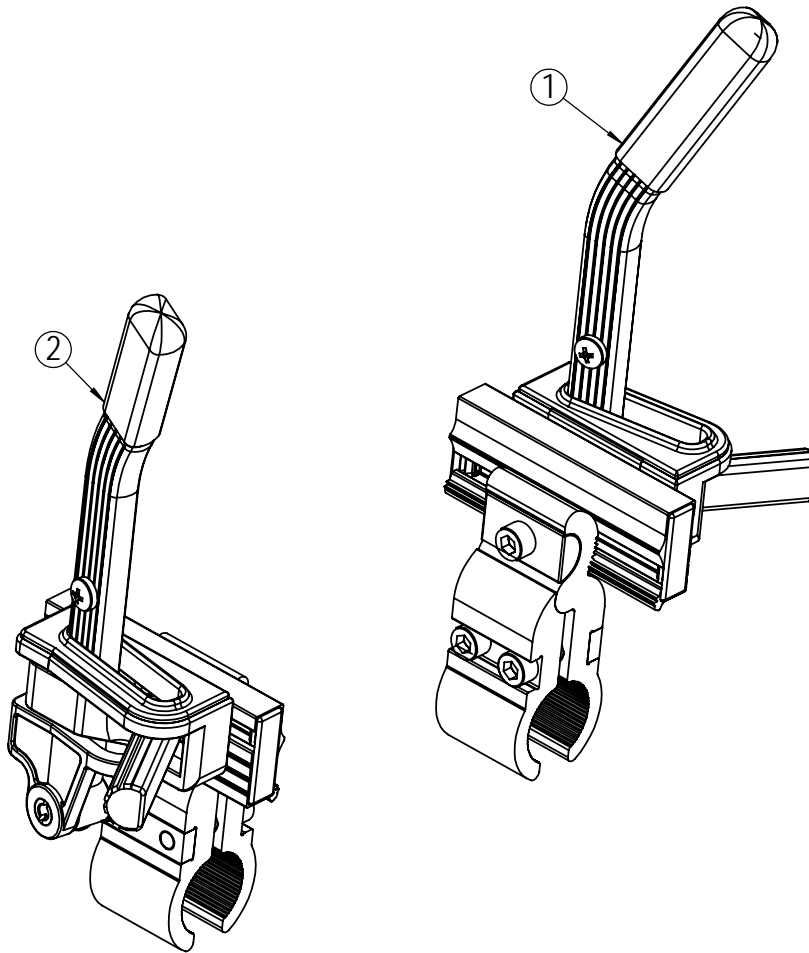


Brake

Position	Description	Item no.	Pcs. pr. chair
1	Brake Universal Netti I/Svipp Left	26824	1
2	Brake Universal Netti I/Svipp Right	26817	1

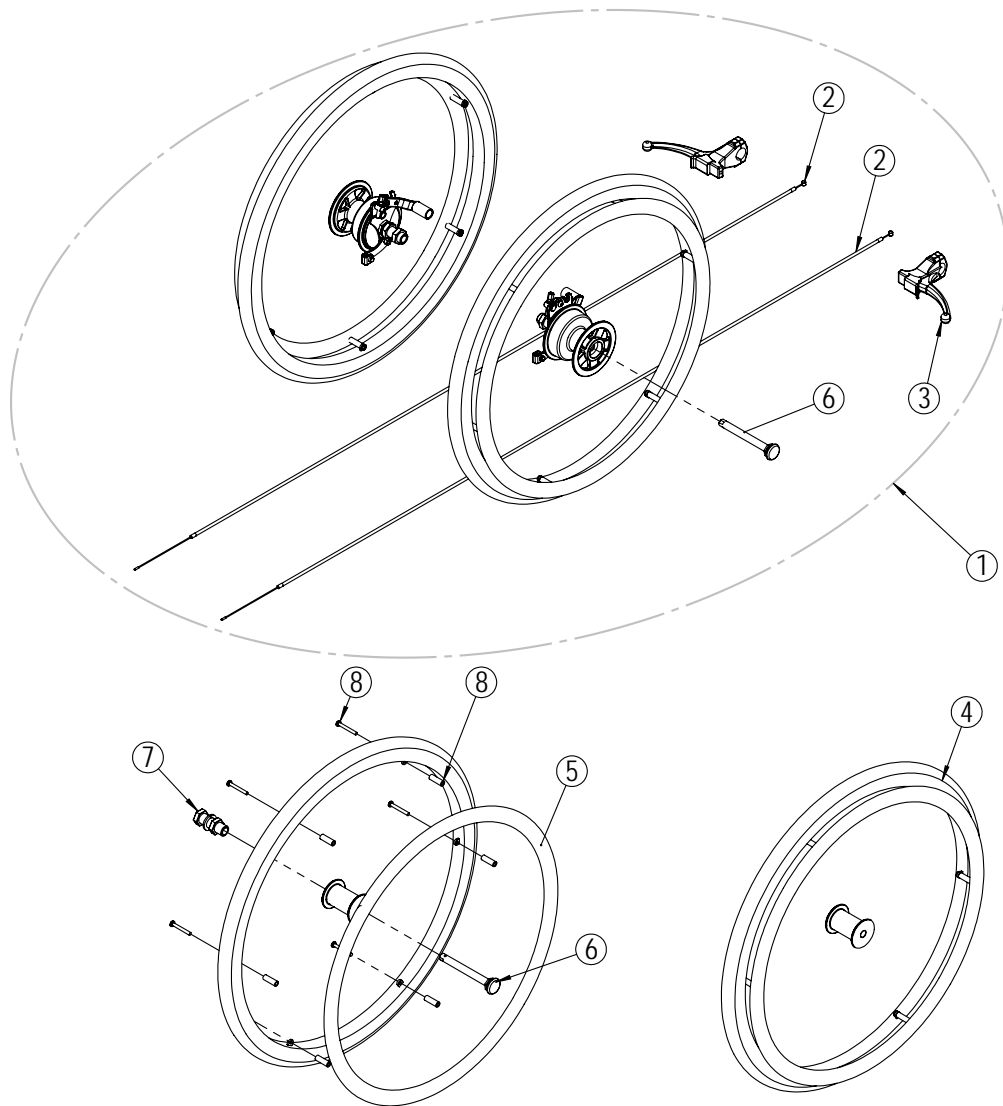

Main wheels

Position	Description	Item no.	Pcs. pr. chair
1	Main wheel 22" x 1" with drum brake - kit	24365	1
1	Main wheel 24" x 1" with drum brake - kit	24310	1
2	Wire 1250 mm complete	41310	2
3	Brake handle	41313	2
4	Main wheel 22" x 1" with quick release	25371	2
4	Main wheel 24" x 1" with quick release	25372	2
5	Push rim 22" plast	20451	2
5	Push rim 24" plast	20450	2
6	Quick release Ø12 x 98mm	20449	2
7	Wheel bushing complete	20329	2
8	Screw (6 pcs.) & distance sleeves (6 pcs.) - kit	21111	2

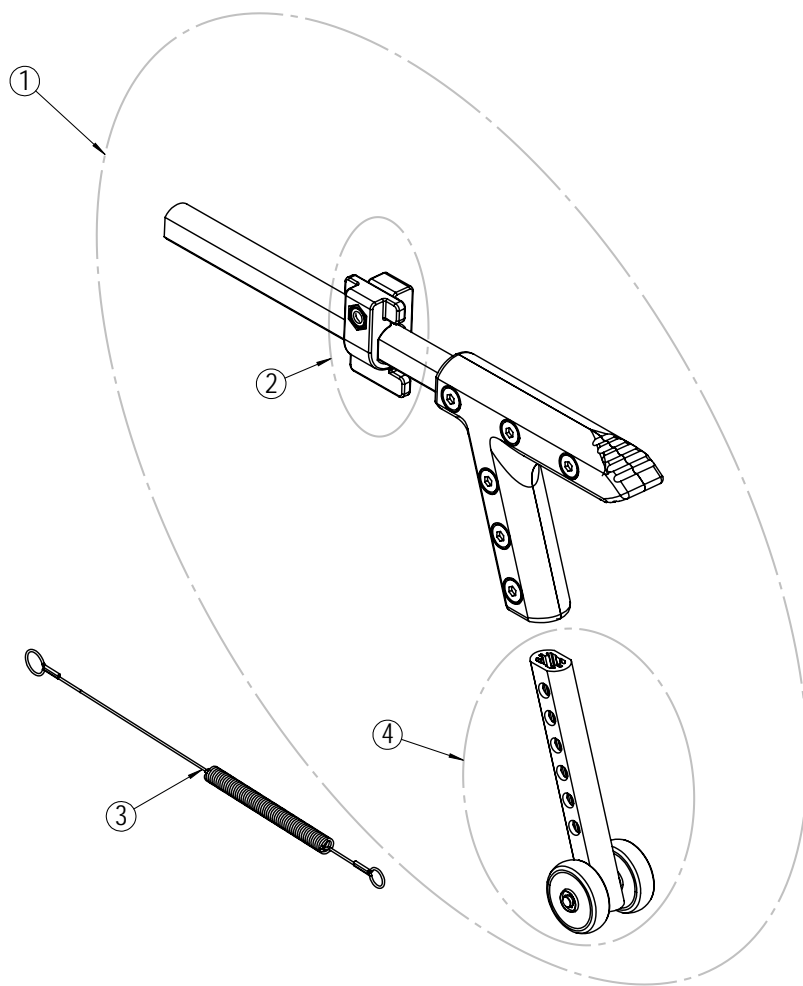

Anti tip

Position	Description	Item no.	Pcs. pr. chair
1	Anti tip long/short	25419	2
2	Anti tip lock bracket complete	26980	2
3*	Anti tip spring 445mm	24242	2
4*	Anti tip vertical bar with wheels standard	26118	2

* Available upon request