NETTI 4U CED/CEDS

Artikkelnr. / Item no. / Artikel nr.	
43799	35-60 cm

N Verktøy: Stjerneskrudjern, saks

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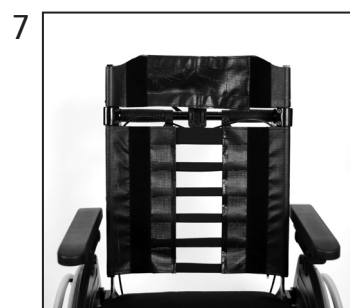
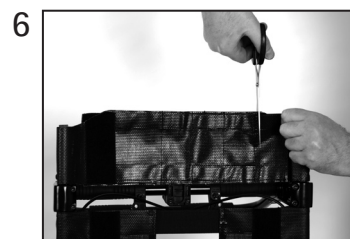
GB Tools: Cross-headed screw driver, scissors

1. Remove plastic plugs from backrest tube ends.
2. Remove screw attaching the velcro to the backrest tubes.
3. Insert the backrest extender into the backrest tube.
4. Re-mount the screw attaching the velcro to the backrest tubes. The screw should enter the pre-drilled holes in the backrest extender in required height position.
NB! Ensure that the extender velcro is covering the top section of the cushion. The top end of the backrest extender profiles should be placed right below the cut corners.
5. Attach the two loose velcro flips in centre.
6. Cut surplus material to release the attachment velcro against the backrest cushion.
7. Mount the backrest. See separate mounting description.

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GB Tools: Cross-headed screw driver, 4 mm Allen key, pliers, scissors

1. Remove plastic plugs from backrest tube ends.
2. Remove the screw attaching the push bow bracket to the backrest tubes with an Allen key, while fixing the nut on the inside with a pair of pliers.
3. Remove the nut.
4. Remove screw attaching the velcro to the backrest tubes.
5. Re-mount the screw in the push bow bracket. It is fixed to the loose square nut in the profile of the backrest extender. Let it be loose.
6. Re-mount the screw attaching the velcro to the backrest tubes. The screw should enter the pre-drilled holes in the backrest extender in required height position.
NB! Ensure that the extender velcro is covering the top section of the cushion. The top end of the backrest extender profiles should be placed right below the cut corners. Fix the screw described in point 5.
7. Attach the two loose velcro flips in centre, and cut surplus material to release the attachment velcro against the backrest cushion.
8. Mount the backrest. See separate mounting description.

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